

Stainless Steel Trolley Cleaning Instructions

Please use a neutral cleaner (no strong acid, alkali or oxidizing cleaners) to wipe the trolley surfaces down.

Once cleaned it is important no residual cleaner is left on the trolley surface which can form scale deposits or staining. To remove please use a clean soft cloth to wipe the trolley dry.

Should your trolley experience scale deposits or staining as illustrated in the image below.



Please utilize a specific cleaner and polisher suited to 304 stainless steel or gently scrub the trolley with a scourer as shown below to remove the buildup. Once removed commence your usual cleaning regime.





Phone: 1300 076 332 Email: info@pacificmedical.com.au www.pacificmedical.com.au