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# Bovie

**Bovie Medical Corporation**  
5115 Ulmerton Road  
Clearwater, Florida 33760 USA

EC REP

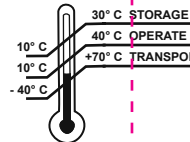
**Emergo Europe**  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands


Rx ONLY

STORAGE  
10 to 75 % non-condensing  
OPERATE  
30 to 75 % non-condensing  
TRANSPORT  
10 to 100 % including condensation

Illustration
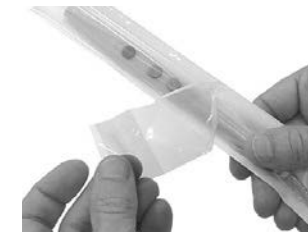
## EN HANDPIECE SHEATH

REF A910

### WARNING:

This medical device cannot be effectively cleaned and/or sterilized by the user and therefore cannot be safely reused. It is intended for single use only. Any attempt by the user to clean and resterilize this device may result in biocompatibility, infection, or other risks of device failure to the patient.

1. INSERT HANDPIECE: To insert the handpiece, first remove the electrode and slide the electrosurgical handpiece beneath the WHITE tab into the sheath opening. Push the handpiece so that the tip meets the end of the sheath.
2. REMOVE PROTECTIVE COVER: To remove the protective cover, hold the handpiece at the bottom and pull on the BLUE tab as shown. Throw away the cover.
3. REMOVE PAPER BACKING: Beginning at the tip end of the handpiece, peel the paper backing in the direction shown and discard.
4. Insert electrode through the sheath into the tip of the handpiece.

**IMPORTANT** - Visually inspect sheath for damage after application to the handpiece and prior to placing in sterile field.

5. After use, remove sheath and dispose of properly.

**NOTICE:** Any serious incident that has occurred in relation to the device should be reported to Bovie Medical Corporation., the governing body, and the Competent authority of the member state in which the incident occurred

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