

CONFY CARE®

Medical Stool

MA13BR / MA14BR / MA15AR

INSTALLATION INSTRUCTION

Contents

MA13BR round stool with backrest	-----	P01 - 03
MA14BR saddle stool with backrest	-----	P04 - 06
MA15AR round stool with armrest	-----	P07 - 09




MA13BR
Round Stool with Backrest
INSTALLATION INSTRUCTION

Installation Steps


1. Install wheels

Insert the wheels into the five-star base.

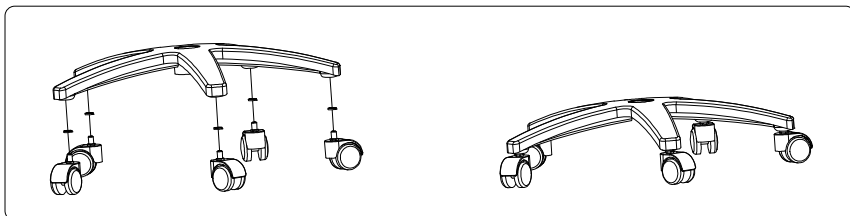


2. Install spring gas

Insert the large end of the spring gas into the circular hole in the center of the five-star base.

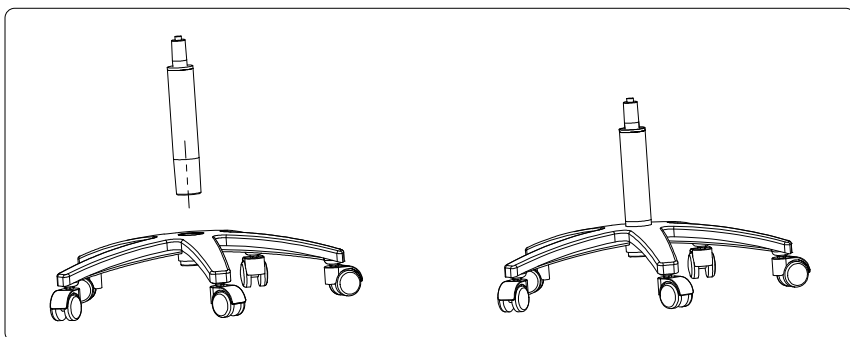


Insert the small end of the spring gas into the circular hole in the center of the chair chassis.

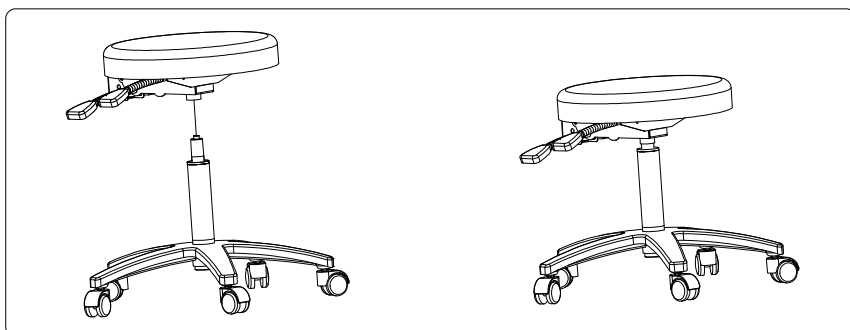


3. Install backrest

Insert the backrest rod into the base hole and tighten the knob to complete the installation

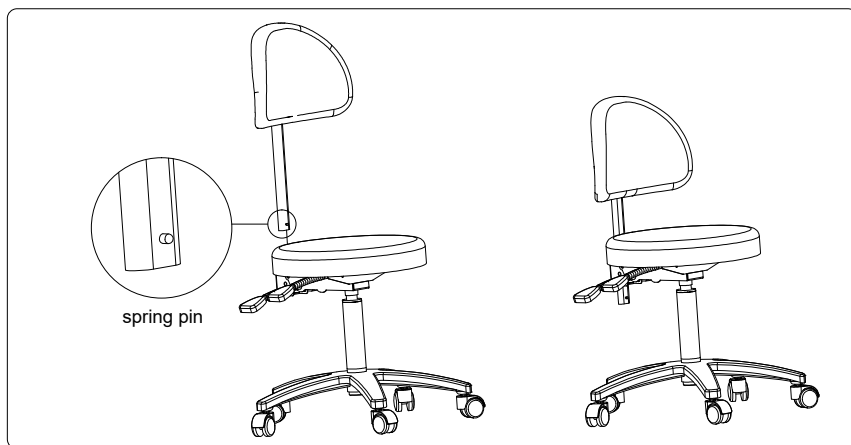

Operation

1. Adjust the height of the seat by adjusting the lever.
2. Adjust the front and rear angles of the backrest through the adjustment lever.
3. Adjust the height of the backrest through the knob.

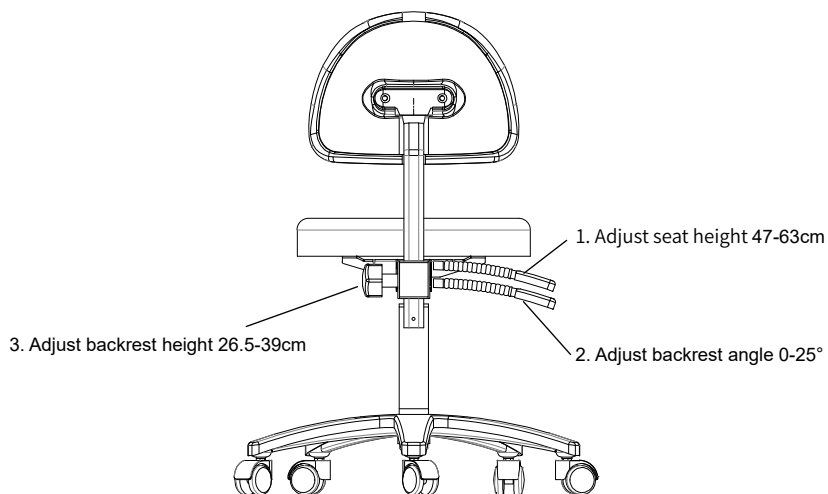




MA14BR
Saddle Stool with Backrest
INSTALLATION INSTRUCTION

Installation Steps


1. Install wheels

Insert the wheels into the five-star base.

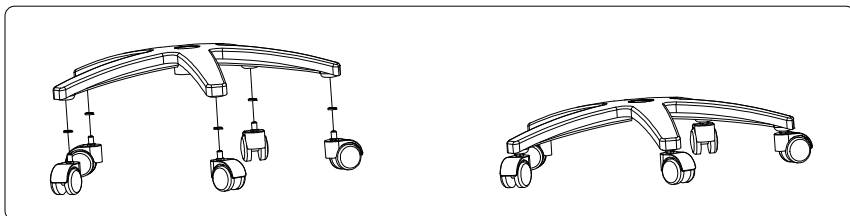


2. Install spring gas

Insert the large end of the spring gas into the circular hole in the center of the five-star base.

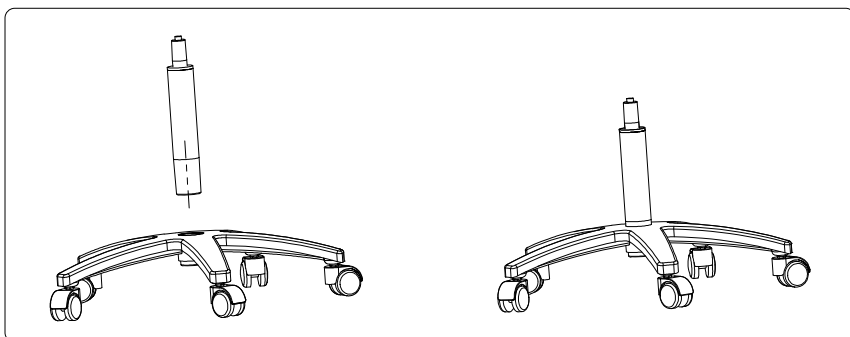


Insert the small end of the spring gas into the circular hole in the center of the chair chassis.

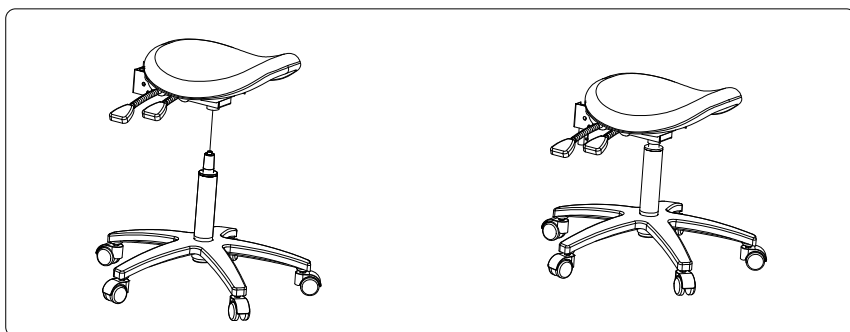


3. Install backrest


Insert the backrest rod into the base hole and tighten the knob to complete the installation

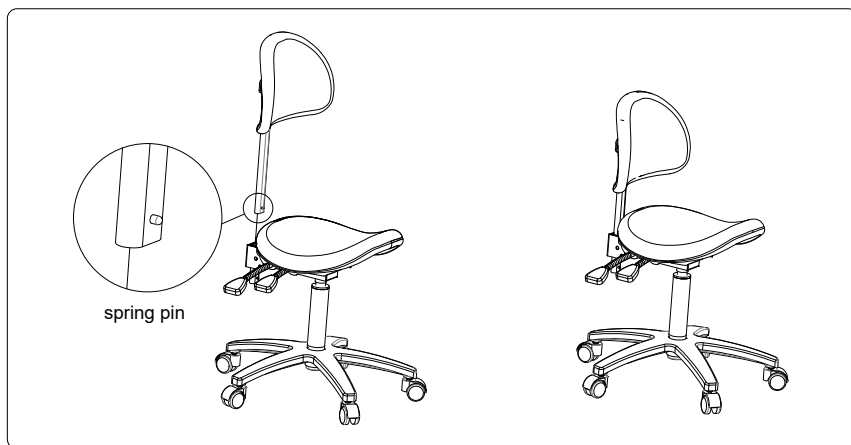

Operation

1. Adjust the height of the seat by adjusting the lever.
2. Adjust the front and rear angles of the backrest through the adjustment lever.
3. Adjust the height of the backrest through the knob.
4. Adjust the front and rear angles of the seat cushion by adjusting the lever.

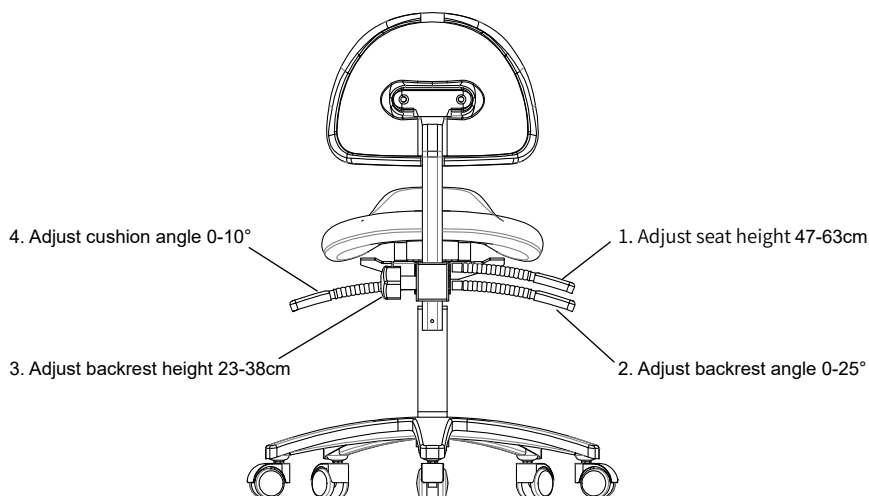

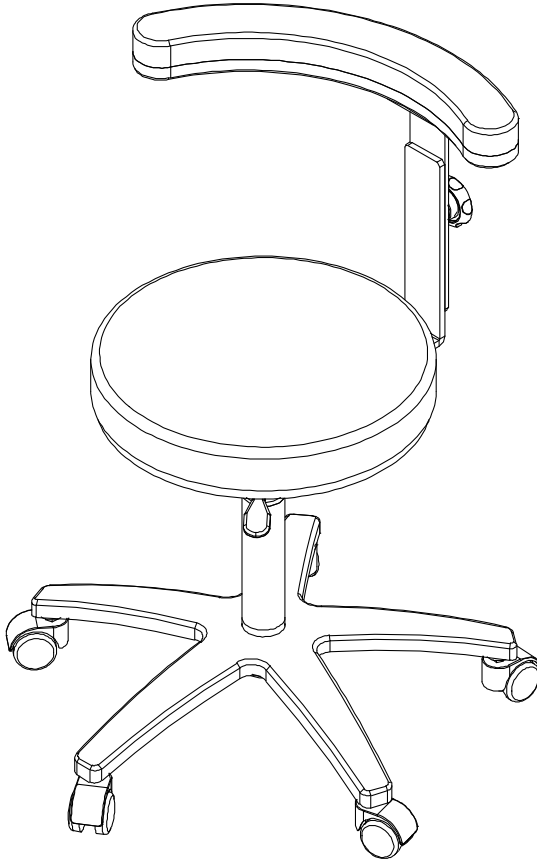


MA15AR
Round Stool with Armrest
INSTALLATION INSTRUCTION

Installation Steps


1. Install wheels

Insert the wheels into the five-star base.

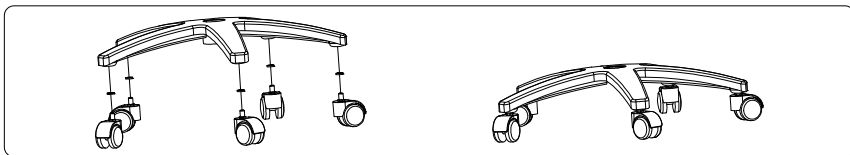


2. Install spring gas

Insert the large end of the spring gas into the circular hole in the center of the five-star base.

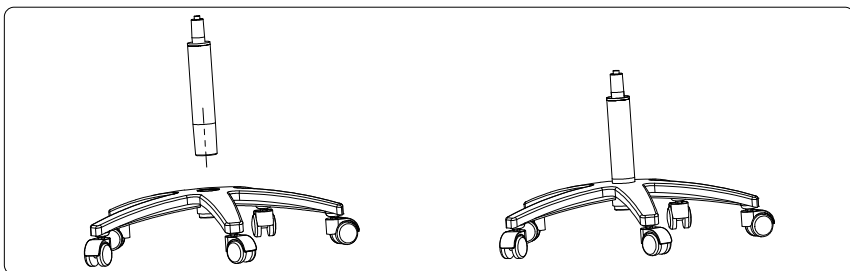


Insert the small end of the spring gas into the circular hole in the center of the chair chassis.

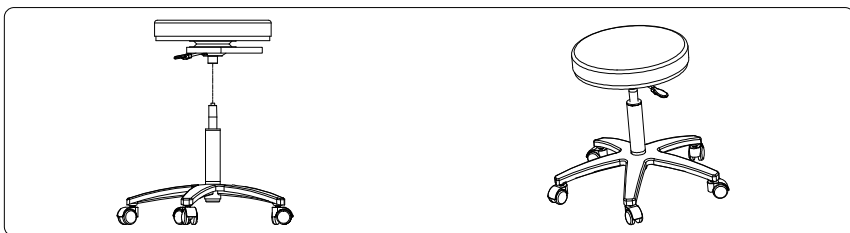


3. Install armrest


Install the L-shaped backrest rod on the chassis and tighten the screws

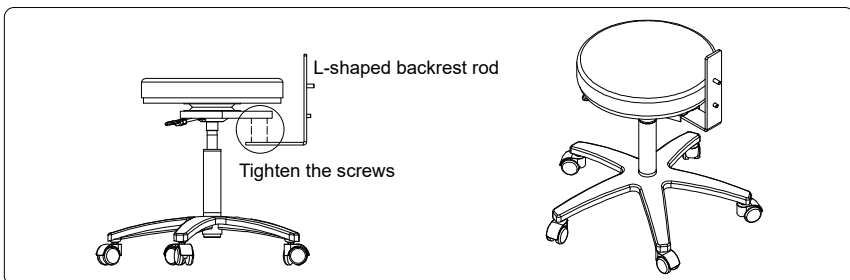

Install the T-shaped backrest rod at the bottom of the armrest and tighten the screws

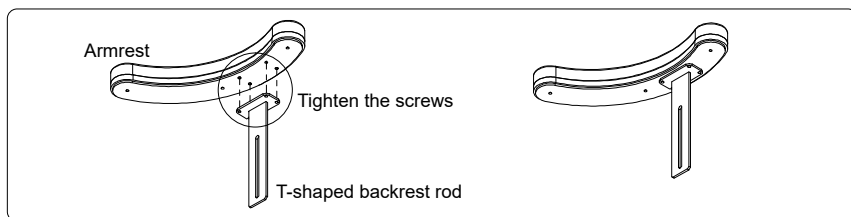


Fix the armrest and backrest rod together. Align the screws and washers, and fix the height with the knob to complete the installation

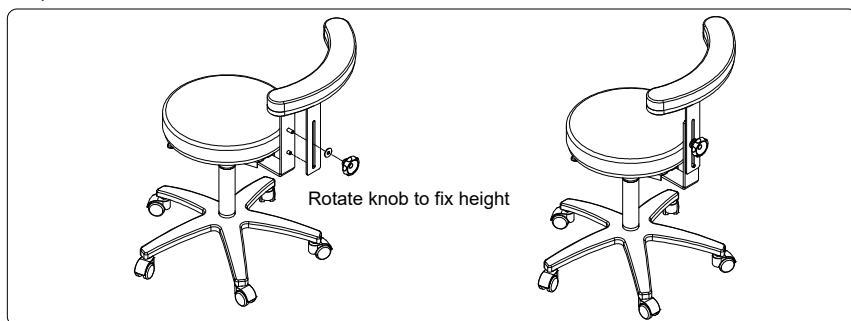

Operation

1. Adjust the height of the seat by adjusting the lever.
2. Adjust the height of the armrest through the knob.
3. Armrest can be rotated and adjusted 360 °

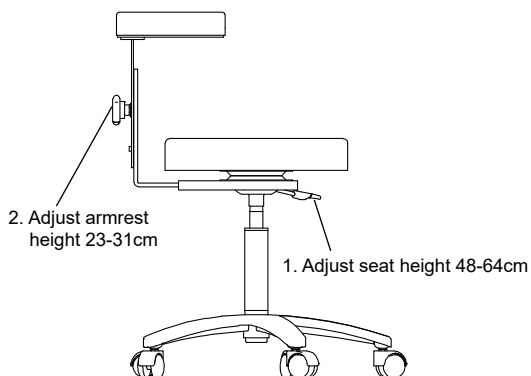

3. Armrest 360 ° rotation adjustment

