



# Office Medic™

## User's Manual

For use with: Orbit™ • Universal SmartECG™

**EMERGO EUROPE**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands



**VectraCor, Inc.**  
785 Totowa Road, Suite 100  
Totowa, NJ 07512



# Table of Contents

<b>General Cautions &amp; Warnings .....</b>	<b>4</b>
Glossary of Symbols.....	4
Important Information Regarding Instruction for Use.....	5
Warnings .....	5
Cautions .....	7
Electrical Safety Classifications.....	9
<b>Accessories .....</b>	<b>10</b>
Spirometry Accessories .....	10
Electrocardiogram Accessories .....	12
<b>Office Medic Basics .....</b>	<b>13</b>
System Requirements .....	13
Installation.....	14
Backing-up and Restoring the Database .....	15
Navigation .....	16
File Menu.....	17
Test Menu .....	19
Options Menu.....	20
Tools Menu.....	22
Help Menu .....	22
<b>Spirometry .....</b>	<b>24</b>
Spirometry Cautions & Warnings.....	24
Spirometry Getting Started .....	25
Proper Patient Preparation .....	26
Proper Testing Procedure .....	26
Effort Quality Messages for Adult Subjects.....	27
Test Session Grades .....	27
Unacceptable Spirometry Tests.....	28
Repeatability.....	28
Performing a Spirometry Test using ATS/ERS (2019) Interpretation .....	29
About the Spirometry Test Session Window .....	33
Spirometry Options.....	35
Spirometry Tools .....	40
Predicted Value Equations .....	42
Lung Age Calculation .....	48
Spirometry Interpretation.....	49
<b>Electrocardiography.....</b>	<b>58</b>
ECG Cautions, Warnings, and Other Information .....	58
ECG Getting Started.....	60
Performing an ECG Test: Quick Start Guide.....	60
About the Acquisition Window .....	62
ECG Options .....	64
Reviewing an ECG .....	67
Printing an ECG.....	73
ECG Device Verification.....	74
ECG Analysis Program.....	74

<b>Troubleshooting and Tips.....</b>	<b>75</b>
Office Medic shows "Error connecting to Database" message.....	75
How to hide patient names from the patient tree in Office Medic?.....	75
How to reduce artifacts in an ECG tracing? .....	76
How to troubleshoot Leads Off Indication?.....	76
Office Medic shows "Cannot find ECG cable" message.....	77
Other Office Medic messages to users .....	78
<b>Service Information .....</b>	<b>79</b>
Device Care & Maintenance .....	79
Service .....	80
<b>Glossary of Terms .....</b>	<b>82</b>
<b>Device Specifications .....</b>	<b>87</b>
Universal SmartECG Specifications .....	87
Orbit Portable Spirometer Specifications .....	88

## General Cautions & Warnings

Before conducting tests read the General Caution & Warnings and the specific Cautions & Warnings pertaining to your particular medical device.

If you need further assistance, see [Service](#).

### Glossary of Symbols

**Attention**

Consult Accompanying Documents

**Consult Instructions For Use**

Consult Accompanying Documents

**Consult Instructions For Use**

Consult Accompanying Documents

**Defibrillator proof type CF equipment**

Defibrillator proof type CF equipment complying with IEC Publication 60601.

**CE Mark**

Indicates this device is in compliance with MDR (2017/745). 2797 is the Notified Body Number.



Do not reuse.



Class II, Electrical Equipment.

**REF**


Catalogue or Model Number

**S/N**

Serial Number



Manufacturer

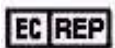

Authorized representative in the European community.



Waste Electronic Electrical Equipment (WEEE). Separate collection for waste electrical and electronic equipment.

**Rx only**

Federal (USA) law restricts this device to sale by or on the order of a physician.

**Latex-Free**

Product is Latex-Free



Non-Sterile



Batch Code



Use By



Upper Limit of Temperature

**Important Information Regarding Instruction for Use**

- Instructions for use are only provided in electronic format and are provided in all Member States of the EU where product is available.
- Customers can request a hard copy of the instruction for use by contacting VectraCor within 30 days of receiving equipment at no additional cost.
- Any request for a hard copy of the instruction for use after 30 days upon receiving the equipment can be provided at a cost.
- Customers can contact VectraCor either by phone or email to request a hard copy of the instruction for use.
- When a hard copy of the instructions for use are requested, a hard copy will be provided with 7 days of receiving the request.
- Current and previous revisions of instructions for use are available on [www.VectraCor.com](http://www.VectraCor.com).

**Warnings**

- Do not use VectraCor Medical Devices in presence of flammable anesthetic mixture.
- Do not operate VectraCor Medical Devices in an explosive atmosphere.
- Use of accessory equipment not complying with EN60601-1 and/or UL2601-1 or equivalent safety standard may lead to a reduced level of safety of the resulting system.
- Computers and printers used with VectraCor Medical Devices should be evaluated to EN 60950-1, EN60601-1 or equivalent safety standard to maintain the safety of VectraCor Medical Devices.
- When using a Networked database: verify virus protection software and firewall software are installed and not disabled.
- Do not use any VectraCor Medical Device on children or vulnerable adults without proper supervision.
- Ensure patient cabling or tubing is carefully routed on all VectraCor Medical Devices to reduce the possibility of patient entanglement or strangulation.
- All numerical, graphical and interpretive data should be evaluated with respect to the patient's clinical and historical picture.

- Do not attempt to insert any VectraCor Medical Device (including patient cables) directly into an electrical outlet.



- Restoring the database erases all of the data located in Office Medic and replaces it with the data contained in the back-up file. Data that was acquired after the date of the last back-up will be lost and cannot be recovered.
- Once deleted, data can only be recovered from the date of your last back-up. Maintain regular back-ups to ensure data is not lost.
- The computer regulates the battery and will provide a warning message to inform the user that the battery is low in order to prevent data loss.
- Do not load any other manufacturer's SCP files. The Office Medic program is designed to work only with VectraCor SCP files.
- Do not use 3<sup>rd</sup> party applications to review or analyze VectraCor SCP files.
- Use only VectraCor approved accessories with VectraCor devices.

---

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.


---

## Cautions


### Disposal Instructions:

Due to the potential presence of hazardous substances in electrical or electronic equipment, DO NOT dispose of VectraCor medical devices with municipal waste. Improper disposal could have an adverse effect on the environment and human health.



For VectraCor products NOT marked with  please contact your local municipal waste company for proper disposal instructions.

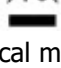

For VectraCor products MARKED with  please contact your local sales representative (from whom you purchased the product) or your local municipal waste company for proper disposal instructions.

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- All VectraCor Devices are intended for use by a physician or by trained personnel under a physician's supervision. Read all instructions for use and specifications provided prior to use.

---

**Important!** VectraCor medical devices are intended for use in the electromagnetic environment(s) specified below. Users of this equipment should ensure that it is used in such environment(s).

---


### Security precautions:

- User is responsible for protection of the login credentials/ access controls used to access the PC hosting Office Medic and patient database. VectraCor will not be able to give access to any user who has lost their access to their host PC.
- User is responsible for equipping the PC hosting Office Medic and patient database with necessary protection against external attacks (virus, malware etc.) and protection configuration.
- VectraCor strongly recommends the user utilize credible antivirus, malware protection, firewall, etc. software on each piece of equipment where patient information is stored. Please keep in mind that VectraCor software will need to be given appropriate permissions to operate properly.
- User is responsible for creating/maintaining logs of login or Office Medic usage information.
- User is responsible for patient database including patient demographic information and patient medical data. VectraCor will not receive or maintain any patient identifiable data.
- User is responsible for scheduled backup of the database to prevent data loss due to unforeseen circumstances. Refer BACKING UP AND RESTORING DATABASE section for instructions.
- User is responsible for the integrity of Office Medic software and its components residing in the PC.
- VectraCor does not access, store or modify the patient demographic and medical data stored in the Office Medic database residing in the user's PC.
- Use of other equipment that connects to the network may result in unidentified risks to patients, operators or third parties. The User is responsible for identifying, evaluating, and controlling these risks.
- Changes to the network (including, but no limited to changes in network configuration, connection of additional items, disconnection of items, update of equipment, or upgrade of equipment) may introduce new risks that requires additional analysis.

Attention should be paid to the following EMC information prior to installing or using VectraCor medical devices.

- Portable and mobile Radio Frequency (RF) communication equipment may interfere with the operation of VectraCor medical devices. RF equipment should only be used no closer than 30 cm (12 inches) to any part of VectraCor medical devices.
- VectraCor medical devices have been tested and found to comply with IEC/EN 60601-1-2.
- Computers, cables and accessories not tested to 60601-1-2 may result in increased emissions or decreased immunity of VectraCor devices.
- Verify normal operation if utilizing VectraCor medical devices adjacent to or stacked with other electrical equipment.

Guidance and manufacturer's declaration - electromagnetic emissions and immunity		
Emissions Test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	VectraCor equipment uses RF energy only for its internal function. Therefore, its RF emissions are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	VectraCor medical devices are suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable for VectraCor devices	
Voltage Fluctuations/flicker emissions IEC 61000-3-3	Not applicable for VectraCor devices	

Immunity Test	IEC 60601 Test Level		Compliance	Electromagnetic Environment Guidance
	Universal SmartECG	Orbit Spirometer		
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±2 kV, ±4 kV, ±6 kV contact ±2 kV, ±4 kV, ±8 kV air	✓	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	3 A/m	✓	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m 80 MHz to 2.5 GHz	✓	<p>Portable and mobile RF Communications equipment should be used no closer to any part of VectraCor medical devices, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance:</b>  <math>d = 1.2\sqrt{P}</math>      80 MHz to 800 MHz  <math>d = 2.3\sqrt{P}</math>      800 MHz to 2.5 GHz</p> <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interface may occur in the vicinity of equipment marked with the following symbol:</p> 

NOTE: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which VectraCor medical devices are used exceeds the applicable RF compliance level above, VectraCor medical devices should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating VectraCor medical devices.
- b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

**Recommended separation distances between portable and mobile RF communications equipment and VectraCor medical devices.**

VectraCor medical devices are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of VectraCor medical devices can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and VectraCor medical devices as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter  W	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## Electrical Safety Classifications

Note: These classifications currently apply only to VectraCor Medical Devices.

- Class II Equipment
- Type CF Equipment. Note: Universal SmartECG is Type CF with defibrillator-proof applied part.
- IPXO - Ordinary Equipment.
- Continuous Operation.
- Not suitable for use in presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.

# Accessories

## Spirometry Accessories

### Mouth Pieces (Z-5000-2608)



All QRS spirometers use pre-calibrated, disposable mouthpieces. These mouthpieces are individually calibrated during production using the same equipment LDS Hospital uses to validate spirometers against the American Thoracic Society's Recommendations for the Standardization of Spirometry. These pneumotachs are single patient use only.

### Pressure Tubes (Z-7000-2032)



Pressure tubes connect the pre-calibrated mouthpieces to the QRS spirometers. They are re-usable, but need to be replaced if kinked or if condensation forms inside the tube. Pressure tubes measure 48" long.

### Nose Clips (724050-00)



QRS, in line with ATS Guidelines, recommends the use of nose-clips when performing a Spirometry test with a QRS spirometer.

### **Calibration Syringe (723000-00)**



QRS sells a three-liter Volume Calibration Syringe designed to fit the QRS spirometers. Although QRS spirometers cannot be calibrated in the field, the syringe allows you to check your calibration. According to ATS Guidelines, the volume accuracy of the spirometer must be checked at least daily. If calibration is off, please contact QRS for recalibration of your spirometer.

### **Viral/Bacterial Filter (724100)**



Filter to minimize bacterial/viral transmission. It is to be secured onto the end of the mouthpiece.

## Electrocardiogram Accessories

### Tab Electrodes



- Gel: Adhesive Gel
- Chloride Content: 4%
- Substrate: Synthetic Paper or Vinyl
- Sensor: Ag/AgCl
- Shape/Size: Rectangle, 1" x 7/8"
- Adhesive Performance: Usable up to 1 hour
- Shelf Life: 2 years, when stored between 10-32°C

### Wet Gel Electrodes



- Superior Conductivity
- Up to 72 hours of continuous use
- Better Skin Contact
- Recommended for Vectraplex ECG System

### Snap Electrode Adaptors



- Universal Adapter fits both snap and tab electrodes: Fits 3 mm to 4 mm pin leads
- Latch Lock System: Securely locks on electrodes, leads won't detach
- Secure Connection: Reduces false alarms
- Re-usable: Adaptors can be cleaned/sterilized and re-used

# Office Medic Basics

## System Requirements

Operating System:	Microsoft® Windows®: 10 or 11
Free Disk Space:	10GB
Internet Requirements:	Internet Explorer 6.0 SP1 or later
RAM:	4 GB or higher
Processor:	x86 & Dual-core 2 Ghz or higher (ARM CPU Architecture is not supported)
Screen Resolution:	1024x768 (EKG Requirement)
Interface:	Available USB port

### IT-Network information:

- Purpose of networking: Networked databases can be accessed by multiple computers and integrate Office Medic test sessions to an EMR/HER
- Required characteristics: Windows NT Client/Server domain.
- Required configuration: Windows NT Active Directory
- Technical specifications: Microsoft SQL trusted connect string
- Intended information flow: The Office Medic software utilizes a Software Developer Kit (SDK) and calls Office Medic to perform tests and return test data.


\*\*Recommended system specifications: PC running Windows 10 or 11, Dual core CPU, 4 gigs of RAM, 300 gig HDD or better with an available USB port.

## Installation

---

Important! Do not connect the medical device to the PC prior to installing the software. The device drivers (step #9) must be installed prior to testing.

---

1. Ensure you are logged in with Administrator rights.
  2. Remove all VectraCor devices from the computer.
  3. Log out and close all programs.
  4. Insert the Office Medic CD-ROM.  
If the autorun feature on your computer is disabled go to the next instruction. If not follow the on screen prompts.
  5. On the lower Windows toolbar select **Start | Run** or simultaneously press the Windows logo  and R key. Type d:\setup.exe in the Open dialog box. Note: substitute the letter of your CD/DVD- ROM drive if it is different from d:.
  6. Select Program(s) to be installed
    - I. Office Medic
    - II. Office Medic & VectraplexECG
  7. Select a language.  
\*\* Language package is available for Office Medic only. VectraplexECG is currently in English only.
- 


Note: If you need to change the language, you will need to uninstall Office Medic. To do this go to your control panel, click on "Programs and Features" and then find the "Programs" and select "Uninstall a program. Find Office Medic on the list and uninstall. Finally, reinstall Office Medic using the setup program and select the correct language. Any data that was recorded will be preserved because uninstallation doesn't delete data.

---

8. Follow the on-screen instructions.
- 

Note: You will be given a choice to install a local or network database. The Network option requires an Office Medic Network Database formally called IDMS database. To learn more about obtaining a Network database, and networking Office Medic, contact Customer Care at VectraCor.

---

An Office Medic shortcut will appear on your desktop when the installation is complete. 

9. Once the installation is complete, connect the medical device to the PC with the CD-ROM still inserted. Follow the software prompts for installing the device driver.

## Backing-up and Restoring the Database

### Database Back-up Instructions

Backing-up your database protects you from losing your patient data should a catastrophic event occur. Regular back-ups of the database should be maintained. Follow the steps below to back-up the database:

1. Close Office Medic.
2. Open folder: C:\Vectraplex\Database.
3. Copy the two files VectraplexECG.MDF and VectraplexECG\_Log.LDF to a secure location. This is the back-up copy of your Office Medic database. Copy these files as often as needed to maintain a current back-up file.

### Database Restore Instructions

---

**Warning!** Restoring the database erases all of the data located in Office Medic and replaces it with the data contained in the back-up file. Data that was acquired after the date of the last back-up will be lost and cannot be recovered.

---

Follow the steps below to restore the database:


1. Close Office Medic.
2. Copy and paste the two back-up files into the following location: C:\Vectraplex\Database.
3. Open Office Medic.

The database should look exactly as it did on the date of the last back-up.

## Navigation



Select the Office Medic icon to open the software. The initial screen displays the directory of patients, sessions and tests. Contact VectraCor Technical Support for instructions on how to hide patient names.

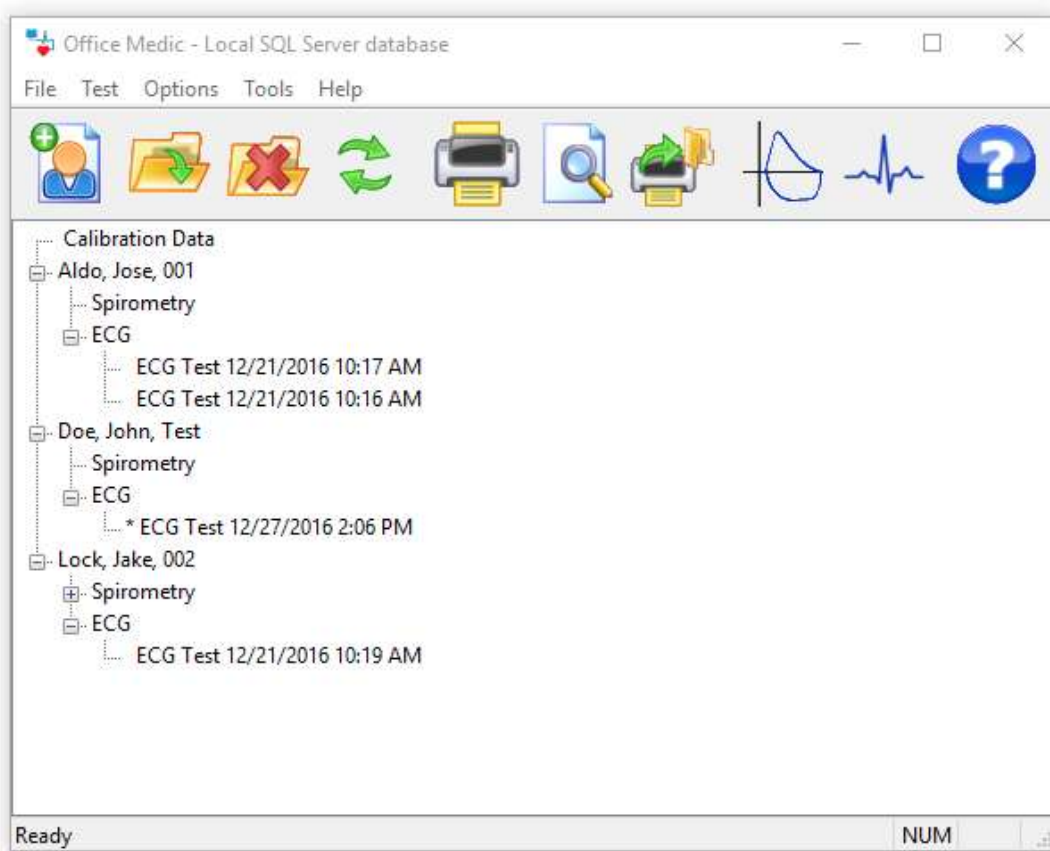

---

Note: ECG tests with an asterisk symbol are acquired by VectraplexECG. ECG's with an asterisk symbol will be displayed in VectraplexECG when opened.

Note: VectraplexECG only requires account number and gender to create a new patient. Some functions in Office Medic require more information.


---

---

Note: If an ECG was first acquired using Office Medic and the user changed the default ECG program from Office Medic to VectraplexECG, the ECG will be displayed in VectraplexECG when opened. Upon closing the ECG in VectraplexECG, the user will be prompted to save the ECG. If the user clicks yes to save the ECG, an ECG test will be created in the Office Medic patient tree under that specific patient. The original ECG from Office Medic will remain. If you do not save, a new file will not be created. When viewing these cases in the future open the appropriate file according to your default ECG program.

---

## File Menu

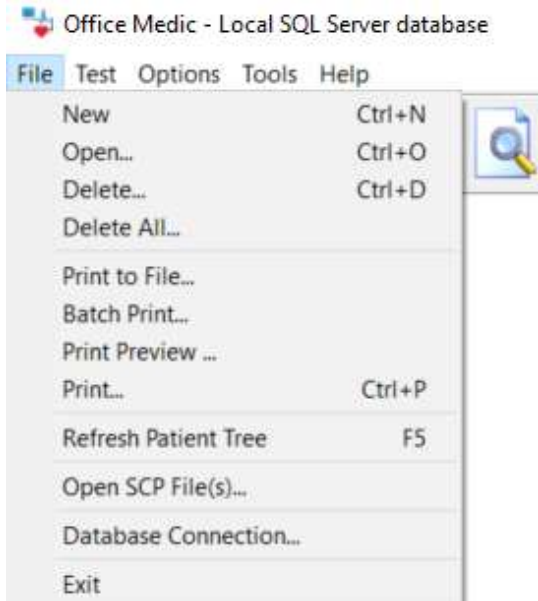

### **New** (Ctrl+N)

Opens the Patient Information window. Required fields are highlighted by an asterisk.

Patient Information

Last Name\* First Name\* Account #\* ID #

Address Phone Number

Height(ft)\* (in)\* Weight(lbs) Gender\* Race\* \* Required

Birth Date \* Age Smoking-Pack Years

OK Cancel

The screenshot shows the 'Patient Information' dialog box. It contains several input fields for patient data. The fields for 'Last Name\*', 'First Name\*', 'Account #\*', 'Gender\*', and 'Race\*' are marked as required with an asterisk. The 'Birth Date' field is also marked as required. The 'Age' field is currently set to '0 m'. The 'Smoking-Pack Years' field is empty. The 'Address' and 'Phone Number' fields are also present. The 'OK' and 'Cancel' buttons are at the bottom right.

---

Note: Smoking-Pack Years is calculated by multiplying the number of cigarette packs smoked per day by the number of years the patient has smoked.

---

**Open** (Ctrl+O)

Select a patient, session or test and then select **Open** to view the selected data.

**Delete** (Ctrl+D)

Select a patient, session or test and then select **Delete** to delete the selected data.

**Delete All**

The Delete All option deletes the entire database.

---

**Warning!** Once deleted, data can only be recovered from the date of your last back-up. Maintain regular back-ups to ensure data is not lost.

---

**Print to File**

Creates an image file (either JPEG, TIFF, or PDF) of an Office Medic report. Highlight the session or test in the patient tree and select this option.

---

Note: The default location for image files is My Documents\Diagnostic Test Data\Image Files.

---

**Batch Print**

The Batch Print option allows for the printing of multiple patient reports.

**Print Preview**

Reports can be previewed by selecting the desired session or test and then select **File | Print Preview**.

**Print** (Ctrl+P)

Select a patient, session or test and select **File | Print** to print a report.

**Refresh Patient Tree** (F5)

Select to refresh the patient database.

**Database Connection...**


Select to switch between local and network databases.

**Exit**

Exits the Office Medic program.

## Test Menu

Select a patient and then select the desired test from the **Test** menu to begin testing.

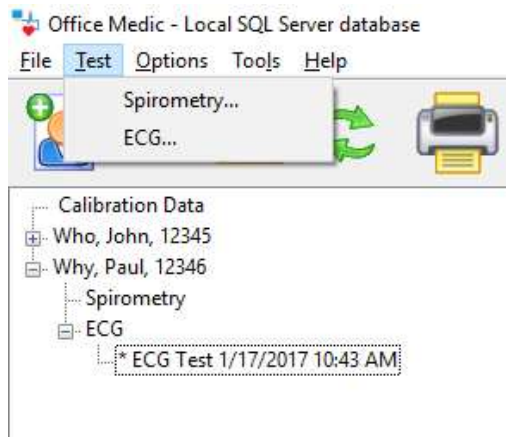


For details on spirometry testing see [Performing a Spirometry Test](#)


For details on ECG testing see [Performing an ECG Test](#)

## Options Menu

Select **Options** to change program settings.

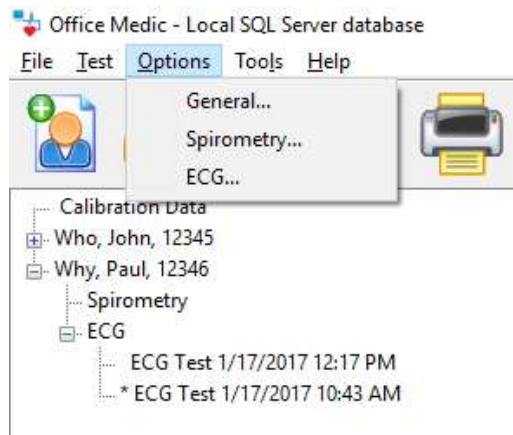

## General Options

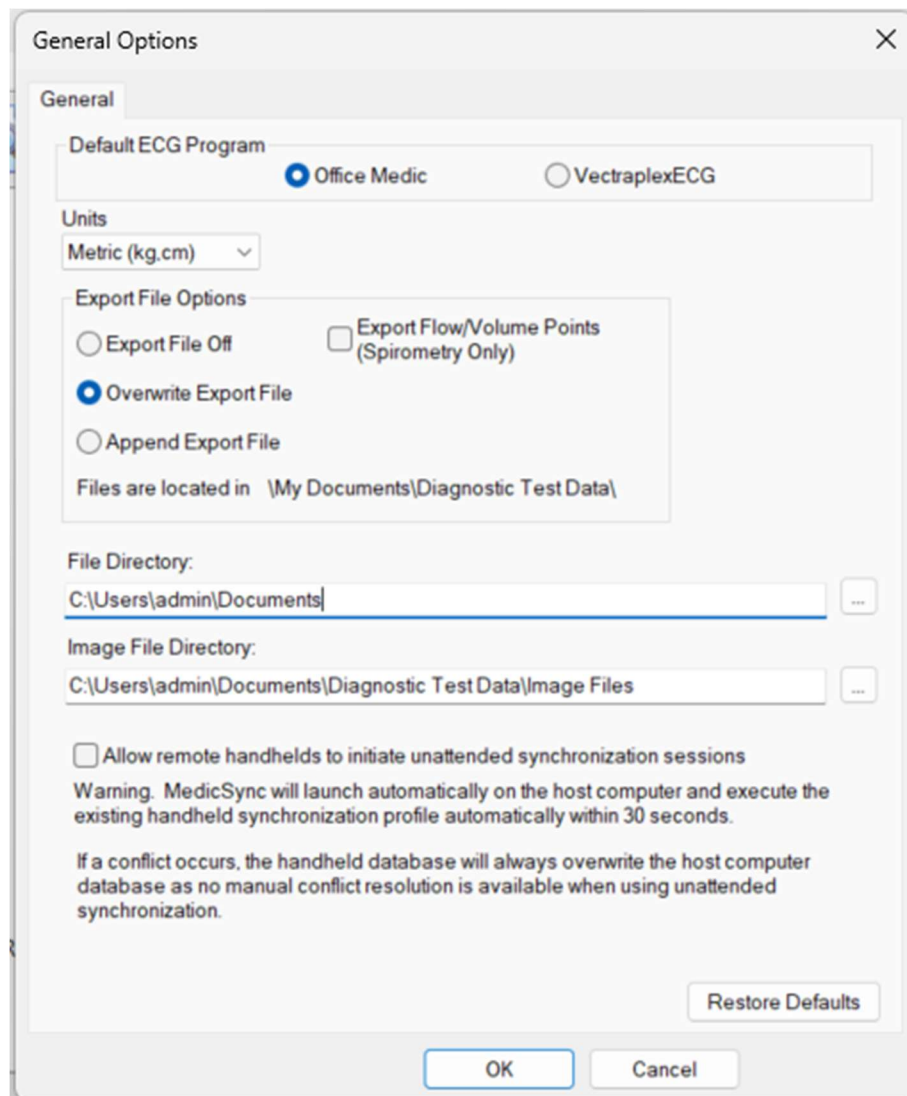

### **Default ECG programs**

Select **Office Medic** or **VectraplexECG**.

### **Units**

Select **Imperial** or **Metric**.

### **Export File**

Creates tab delimited ASCII text files: Session.txt, SpTest.txt, SpCalibr.txt, OxiSess.txt and OxiTest.txt. The Export Flow/Volume Points feature creates two files called SpGraph.txt and SpCalGr.txt.

### **File Directory:**



Select the browse button to change the default path where files are saved.

### **Image File Directory:**




Select the browse button to change the default path where image files are saved.

For details on changing the spirometry options see [Spirometry Options](#).

For details on changing the ECG options see [ECG Options](#).

## Tools Menu

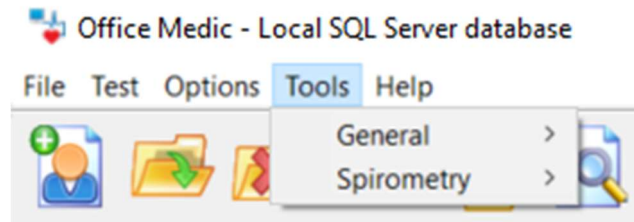


## General Tools



For details on the spirometry tools see [Spirometry Tools](#)

## Help Menu

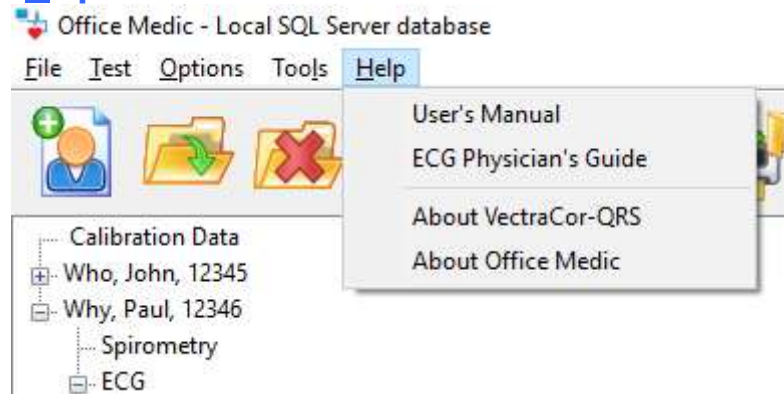

### **User's Manual**

Opens the Office Medic User Manual.

### **ECG Physician's Guide**

Opens the Physician's Guide for the ECG interpretation algorithm.

### **About VectraCor-QRS**

Provides information for contacting VectraCor, Inc.

### **About Office Medic**

Displays the version of Office Medic and statistics about any connected device.


# Spirometry

Note: The information in this chapter applies to spirometry tests acquired using an Orbit Portable Spirometer

## Spirometry Cautions & Warnings

### Warnings



- Use only VectraCor mouthpieces manufactured to meet calibration requirements for the QRS Orbit Portable Spirometer.
-  Mouthpieces are single patient use only and MUST be replaced for each patient.
- Exercise caution when performing spirometry testing on patients with a history of COPD.
- Do not use mouthpieces on a patient with an injured mouth.
- Do not obstruct the opening at the end of the mouthpiece. Obstruction may result in erroneous results.
- FVC and MVV testing can cause fatigue and some patients may be at risk for vertigo, arrhythmia or syncope.
- Patients should open, handle and dispose of his/her own mouthpiece to reduce the risk of cross contamination.
- If condensation forms inside the pressure tube or the pressure tube becomes visibly kinked it must be replaced.

**Warning!** The ATS/ERS Task Force: Standardization of Lung Function Testing recommends daily calibration checks.

### Cautions

- Physicians must properly train individuals, under their care, in the use of this product.
- All tests must be evaluated by a qualified physician.

### Indications for Use: Diagnostic Spirometry

Patient Population: Male/Female, Pediatric to Adult Device Functionality: Diagnostic Spirometry


Spirometric Parameters: FVC, FEV<sub>1</sub>, FEV<sub>1</sub>/FVC, FEV<sub>6</sub>, FEV<sub>1</sub>/FEV<sub>6</sub>, FEV<sub>0.5</sub>, FEV<sub>3</sub>, FEV<sub>3</sub>/FVC, PEFR, PEFT, FEF<sub>25%</sub>, FEF<sub>50%</sub>, FEF<sub>75%</sub>, FEF<sub>25-75%</sub>, PIFR, FIF<sub>50%</sub>, FIF<sub>25-75%</sub>, FIF.2-1.2, FVC/FIVC, FIV, FIVC, FIV<sub>0.5</sub>, FIV<sub>1</sub>, FIV<sub>3</sub>, FIV<sub>1</sub>/FIVC, FIV<sub>3</sub>/FIVC, MVV, MTV, RR, SVC, Extrapolated Volume (Ext. Vol.), FET, and EOTV

Environment of Use: Hospital, Clinical and Home Use

## Spirometry Getting Started

### For the Orbit Portable Spirometer

1. Insert the USB cable into an available USB Port on your PC.
2. Connect the pressure tube to the Luer fitting. Ensure the pressure tube is not kinked or restricted in any way.
3. Connect the other end of the pressure tube to the disposable mouthpiece.

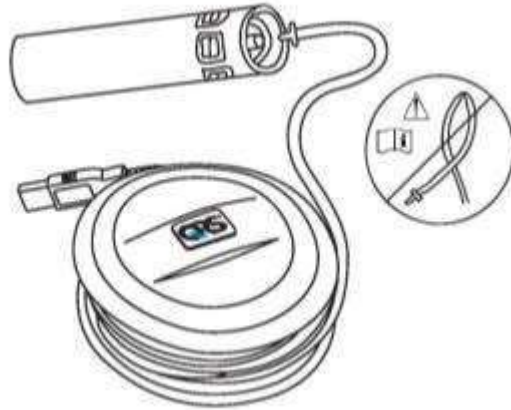

---

**Warning!** Ensure pressure tube is properly connected. If condensation forms inside the pressure tube or the pressure tube becomes visibly kinked it must be replaced.

---

## Proper Patient Preparation

To obtain diagnostically reliable results:

- Loosen tight clothing (ties, belts, bras).
- Remove patient's dentures.
- Explain the procedure thoroughly, including demonstrating it yourself with your own mouthpiece.
- Have the patient sit in an upright position during the test.
- Before beginning the test have the patient take several slow, deep inhalations/exhalations to feel comfortable.


## Proper Testing Procedure

To obtain diagnostically reliable results proper testing procedures must be followed:

- When the equipment is zeroing (two circles flashing) have the patient keep the mouthpiece away from their mouth.
- When testing ensure the patient has a tight seal with their lips around the mouthpiece. The patient should not bite the tube or have pursed lips.
- Place a disposable nose clip securely on the patient's nose or instruct the patient not to exhale through the nose.
- Verbally instruct the patient on properly performing the procedure:
  - FVC – instruct the patient to take the largest possible inhalation, insert the mouthpiece into their mouth and exhale forcefully and completely. If a Flow/Volume Loop is desired, verbally instruct the patient to inhale after completely exhaling.
  - SVC - instruct the patient to take the largest possible inhalation, insert the mouthpiece into their mouth, and exhale slowly and completely.
  - MVV - instruct the patient to breathe as deeply and rapidly for 12 to 15 seconds into the mouthpiece. This test is often difficult to perform for many patients.

---

Important! Ensure the patient has a tight seal around the mouthpiece and is not covering or obstructing the fabric at the end of the mouthpiece with their hand.

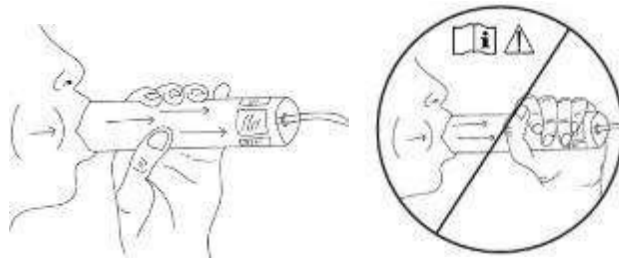

- 
- Encourage the patient to keep exhaling as long as possible. It is helpful to coach the patient with verbal commands and physical gestures. A proper expiration should last at least six seconds.
  - Once finished, have the patient remove the mouthpiece and breathe normally until they have recovered.

---

Important! Using the mouthpiece more than 20 times, or for more than 10 consecutive days, may generate inaccurate results. Use a new mouthpiece after 20 attempts and/or 10 days to get the most accurate results.

---

## Effort Quality Messages for Adult Subjects

Warning Message	Criteria
"Don't hesitate."	BEV (Ext. Vol) > 150 mL or 5% of the FVC
"Blast out faster."	PEFT > 120 msec
"Blow out longer."	FET < 6.0 s for subjects aged 10 years and older or FET < 3 s for subjects aged less than 10 years, and EOTV > 40 mL
"Blast out harder."	PEF values do not match within 1.0 L/s
"Deeper breath."	FEV6 values do not match within 150 mL
Warning message does not appear.	Effort meets above criteria.
"Good test session."	Two acceptable efforts meet the <a href="#">repeatability</a> requirements.

## Test Session Grades

Each test session is given a grade which indicates the degree of confidence in the results. The below test session grade table is for ATS 2019; differing spirometry standards will yield different grades.

Grade	Criteria
A	Three or more acceptable maneuvers with the best two measurements within 150 mL (100 mL or 10% of the highest value, whichever is greater, for children aged ≤6 years).
B	Two acceptable maneuvers with the best two measurements within 150 mL (100 mL or 10% of the highest value, whichever is greater, for children aged ≤6 years).
C	Two or more acceptable maneuvers with the best two measurements within 200 mL (150 mL or 10% of the highest value, whichever is greater, for children aged ≤6 years).
D	Two or more acceptable maneuvers with the best two measurements within than 250 mL (200 mL or 10% of the highest value, whichever is greater, for children aged ≤6 years).
E	One acceptable maneuver <b>OR</b> two or more acceptable maneuvers with the difference of the best two measurements being greater than 250mL (200 mL or 10% of the highest value, whichever is greater, for children aged ≤6 years).
U	One or more usable maneuvers, but no acceptable maneuvers.
F	No acceptable or usable maneuvers.

## Unacceptable Spirometry Tests

A spirometry test is considered unacceptable when:

- Insufficient initial inhalation (lungs not completely filled before the test).
- Slow or hesitant start of expiration.
- Leakage around the mouthpiece or nose clip.
- Mouthpiece obstruction by teeth, tongues, or lips.
- Coughing during the test.
- Large variation of FVC or FEV1 between tests.
- Other problems as indicated by test evaluation messages displayed by the software.
- Mouthpiece was obstructed during test. Obstruction can cause the volume to be unusually high.

## Repeatability

You will be informed when the patient has met the ATS/ERS 2019 repeatability criteria when the difference between the two largest FEV1 and FVC values is within 150 mL for adults and older children, and 100 mL or 10% of the highest value (whichever is greater) for children under 6. Repeatability must be met in three maneuvers, otherwise additional trials (up to eight total maneuvers) should be attempted. An ATS/ERS 2019 warning will be displayed if more than 8 maneuvers are performed on a patient.

You will be informed when the patient has met the ATS 2019 repeatability criteria when:

- Three maneuvers have been accepted and
- The two highest FVC values from any of the maneuvers are within 100ml (or 5%) and the two highest FEV1 values from any of the maneuvers are within 100ml (or 5%).

## Performing a Spirometry Test using ATS/ERS (2019) Interpretation

\*Note: If a different spirometry standard is selected under Options > Spirometry, the below workflow will be different.

1. Prepare the patient as described in the [Proper Patient Preparation](#) section.




2. Select the patient and then select **Test | Spirometry** or the icon  
The Spirometry Test Session screen will appear. Select one of the test buttons to conduct a maneuver.



---

Important! Ensure correct patient is selected.


---

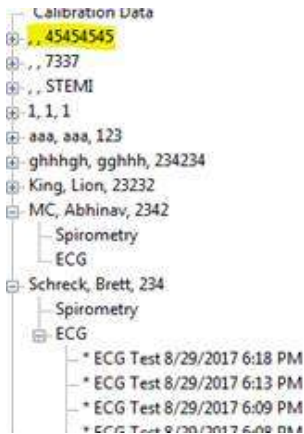

If a patient was created in VectraplexECG and only the account number and gender was entered, the user will receive a pop-up message (as shown above) when a spirometry test is initiated. The user will be required to enter the missing information in VectraplexECG before starting a spirometry test. In order to obtain spirometry predictor values the user will also need to input height, race and birthdate.

---


- Enter the 8-digit number on the mouthpiece label following the # sign.

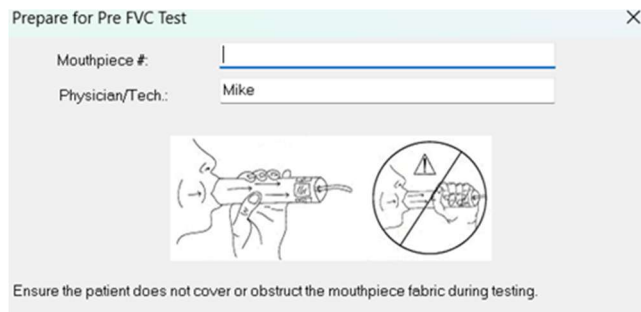

- Perform the Maneuver.


After the mouthpiece number is entered, select OK when ready to test. Two circles will flash red and yellow. When both circles become green instruct the patient to begin the maneuver. Ensure proper testing procedures are being followed as described in the Proper Testing Procedure section.

After the maneuver is performed, the screen below will appear for approval by the technician.

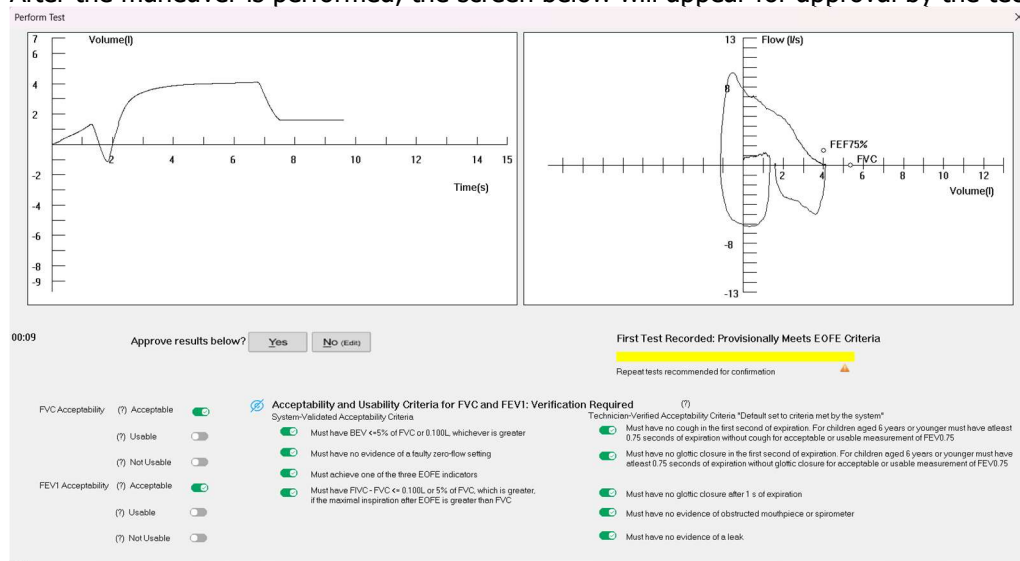

**Important! Ensure the patient does not cover the fabric at the end of the mouthpiece.**

5. Select **YES** to approve results, save the test and display the results. Select **NO (edit)** to make changes

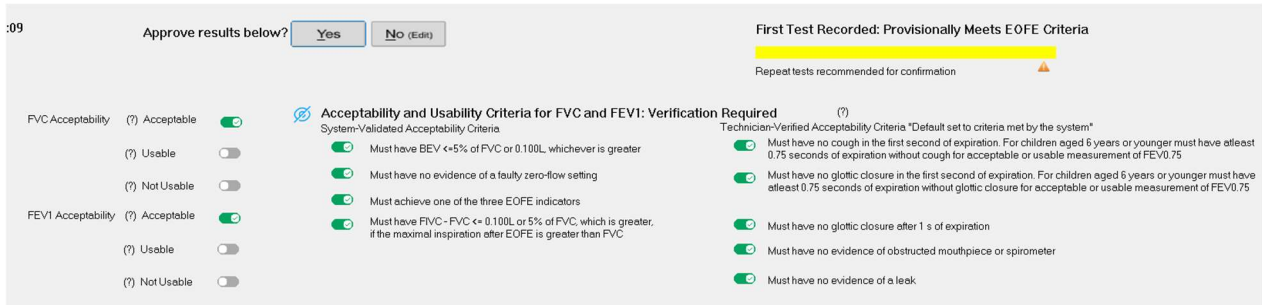


(Zoomed in portion of above screen showing Acceptability and Usability Criteria for FVC and FEV1 per ATS 2019 Recommendations.)

\*Note: There are System-Validated Acceptability Criteria and Technician-Verified Criteria. System-Validated Acceptability Criteria are analyzed and determined by the software. Technician-Verified Criteria are set to be met by default and require the approval of the technician. If the technician wants to change any criterion to not being met, they may do so by clicking the "No (Edit)" option and clicking the respective toggle. This should be done manually by the technician at times when they observe actions or occurrences matching the associated criteria (ex. if the patient coughs).




The user can choose to show or hide the acceptability and usability criteria for each test maneuver by clicking on the eye icon.

To edit the default option, please navigate to Options > Spirometry > Optional Parameters

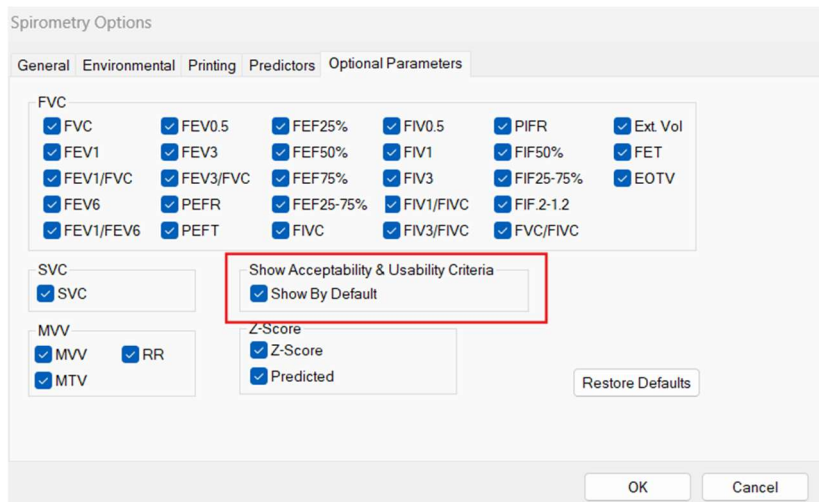

- Review the maneuver, add comments if necessary, perform additional maneuvers as necessary and end the test session.

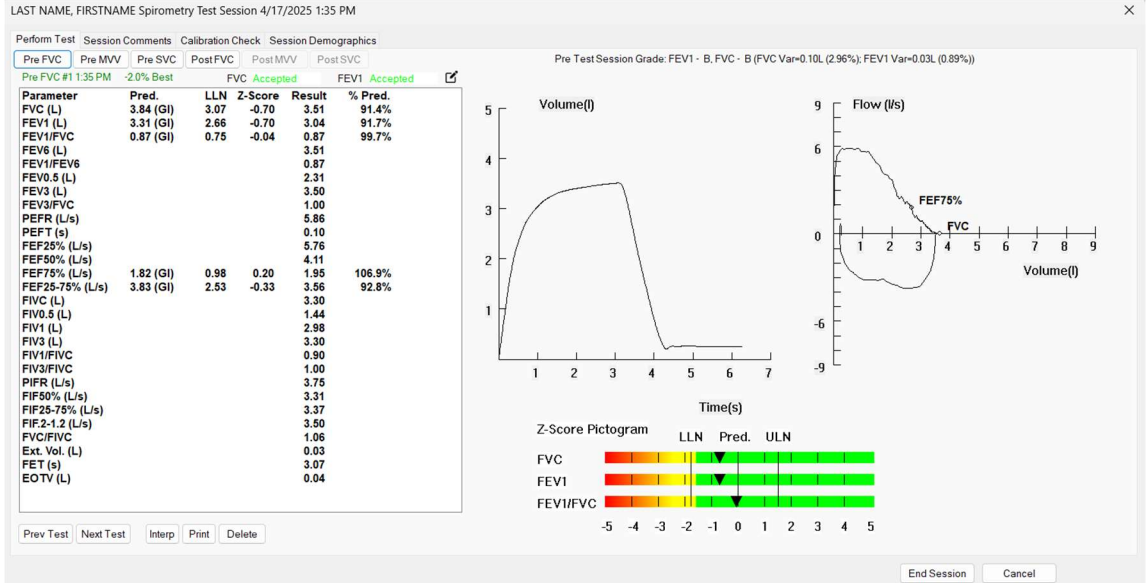

Select **Pre-FVC** to perform additional maneuvers.



Select **Session Comments** to type in freeform text relevant to the session and/or select from a list of standardized operator comments.



# About the Spirometry Test Session Window

LAST NAME, FIRSTNAME Spirometry Test Session 4/17/2025 1:35 PM

Perform Test | Session Comments | Calibration Check | Session Demographics

Pre FVC | Pre MVV | Pre SVC | Post FVC | Post MVV | Post SVC

Pre FVC #1 1:35 PM -2.0% Best FVC Accepted FEV1 Accepted

Pre Test Session Grade: FEV1 - B, FVC - B (FVC Var=0.10L (2.96%); FEV1 Var=0.03L (0.89%))

Parameter	Pred.	LLN	Z-Score	Result	% Pred.
FVC (L)	3.84 (GI)	3.07	-0.70	3.51	91.4%
FEV1 (L)	3.31 (GI)	2.66	-0.70	3.04	91.7%
FEV1/FVC	0.87 (GI)	0.75	-0.04	0.87	99.7%
FEV6 (L)				3.51	
FEV1/FEV6				0.87	
FEV0.5 (L)				2.31	
FEV3 (L)				3.50	
FEV3/FVC				1.00	
PEFR (L/s)				5.86	
PEFT (s)				0.10	
FEF25% (L/s)				5.76	
FEF50% (L/s)				4.11	
FEF75% (L/s)	1.82 (GI)	0.98	0.20	1.95	106.9%
FEF25-75% (L/s)	3.83 (GI)	2.53	-0.33	3.56	92.8%
FIVC (L)				3.30	
FIV0.5 (L)				1.44	
FIV1 (L)				2.98	
FIV3 (L)				3.30	
FIV1/FIVC				0.90	
FIV3/FIVC				1.00	
PIFR (L/s)				3.75	
FIF50% (L/s)				3.31	
FIF25-75% (L/s)				3.37	
FIF2-1.2 (L/s)				3.50	
FIVC/FIVC				1.06	
Ext. Vol. (L)				0.03	
FET (s)				3.07	
EOTV (L)				0.04	

Z-Score Pictogram

LLN Pred. ULN

FVC [Color scale from -5 to 5]

FEV1 [Color scale from -5 to 5]

FEV1/FVC [Color scale from -5 to 5]

Buttons: Prev Test, Next Test, Interp, Print, Delete, End Session, Cancel

## Perform Test Tab

Select any of the spirometry tests to perform a new or an additional pre or post FVC/MVV/SVC test.

Pre FVC | Pre MVV | Pre SVC | Post FVC | Post MVV | Post SVC

## Session Comments Tab

If the user would like to enter in comments about this session, select **Session Comments** to type in freeform text relevant to the session and/or select from a list of standardized operator comments. Freeform comments can be typed in the upper text box while standardized operator comments can be selected from the prepopulated list and appear in the lower text box. There is a character limit of 660 that applies to both freeform text and selected comments combined.

Perform Test | **Session Comments** | Calibration Check | Session Demographics

Pre FVC | Pre MVV | Pre SVC | Post FVC | Post MVV | Post SVC

Pre FVC #1 1:35 PM -2.0% Best FVC Accepted FEV1 Accepted

Parameter	Pred.	LLN	Z-Score	Result	% Pred.
FVC (L)	3.84 (GI)	3.07	-0.70	3.51	91.4%
FEV1 (L)	3.31 (GI)	2.66	-0.70	3.04	91.7%
FEV1/FVC	0.87 (GI)	0.75	-0.04	0.87	99.7%

Perform Test | Session Comments | Calibration Check | Session Demographics

Session Comments Page

Freeform text here

Relating to Patient condition: First attempt at spirometry.

Operator comments 76/660

Relating to Patient condition

- No comments
- First attempt at spirometry
- Reference values are based on ethnicity that may not be suitable for this patient
- Patient used bronchodilator(s) prior to test
- Patient smoked < 1 hr prior to test
- Patient had difficulty understanding directions
- Patient reported consumption of an intoxicant
- Observed symptoms e.g. cough, wheeze, dyspnea or cyanosis

Relating to bronchodilator responsiveness testing

- Facility bronchodilator responsiveness protocol followed for type, dose and delivery method of bronchodilator and wait time before post-BD testing
- Post-BD measurements obtained using other bronchodilator(s), dose(s), delivery method or wait time

Relating to quality of testing session

- No comments
- Acceptability and/or repeatability criteria not met despite patient's best efforts
- Spirometry induced bronchospasm

## Calibration Check Tab

Checks the calibration of the Spirometer and appends the results to the patient's spirometry report. For instructions on performing a calibration check see the [Spirometry Calibration Check](#) section.

## Session Demographics Tab

Select **Session Demographics** to update patient information. This will affect current and future tests only.

Perform Test	Session Comments	Calibration Check	Session Demographics		
Pre FVC	Pre MVV	Pre SVC	Post FVC		
Pre FVC #1 1:35 PM	-2.0% Best	FVC	Accepted		
FEV1	Accepted	FEV1	Accepted		
Parameter	Pred.	LLN	Z-Score	Result	% Pred.
FVC (L)	3.84 (GI)	3.07	-0.70	3.51	91.4%
FEV1 (L)	3.31 (GI)	2.66	-0.70	3.04	91.7%
FEV1/FVC	0.87 (GI)	0.75	-0.04	0.87	99.7%

LAST NAME, FIRSTNAME Spirometry Test Session 4/17/2025 1:35 PM

Perform Test Session Comments Calibration Check Session Demographics

Changes made to session demographics only update the session info. Press the "Update Patient Info" button to update the patient database with these changes.

Height(cm)\*  Weight(kg)  Gender\*  Race\*

Birth Date\*  Age  Smoking-Pack Years

## (Edit) Button

Select the Edit Button to edit the acceptability and usability for the respective test maneuver's FVC and FEV1 values. The user can edit both specific criteria and the overall status of FVC and FEV1. When the user makes a change, this will mark the FVC or FEV1 (whichever is applicable) with an asterisk to designate that it is user-edited.

Perform Test	Session Comments	Calibration Check	Session Demographics		
Pre FVC	Pre MVV	Pre SVC	Post FVC		
Pre FVC #1 1:35 PM	-2.0% Best	FVC	Accepted		
FEV1	Accepted	FEV1	Accepted		
Parameter	Pred.	LLN	Z-Score	Result	% Pred.
FVC (L)	3.84 (GI)	3.07	-0.70	3.51	91.4%
FEV1 (L)	3.31 (GI)	2.66	-0.70	3.04	91.7%
FEV1/FVC	0.87 (GI)	0.75	-0.04	0.87	99.7%

Edit FVC-FEV1 Acceptability Usability

FVC Acceptability (?) Acceptable  (?) Usable  (?) Not Usable

FEV1 Acceptability (?) Acceptable  (?) Usable  (?) Not Usable

(?) Acceptability and Usability Criteria for FVC and FEV1: Verification Required

System-Validated Acceptability Criteria

- Must have BEV <=5% of FVC or 0.100L, whichever is greater
- Must have no evidence of a faulty zero-flow setting
- Must achieve one of the three EOFE indicators
- Must have FVC - FVC <= 0.100L or 5% of FVC, which is greater, if the maximal inspiration after EOFE is greater than FVC

Technician-Verified Acceptability Criteria "Default set to criteria met by the system"

- Must have no cough in the first second of expiration. For children aged 6 years or younger must have atleast 0.75 seconds of expiration without cough for acceptable or usable measurement of FEV0.75
- Must have no glottic closure in the first second of expiration. For children aged 6 years or younger must have atleast 0.75 seconds of expiration without glottic closure for acceptable or usable measurement of FEV0.75
- Must have no glottic closure after 1 s of expiration
- Must have no evidence of obstructed mouthpiece or spirometer
- Must have no evidence of a leak

## Interp Button

Provides an interpretation for the test visible in the test session window. For additional information see the [Spirometry Interpretation](#) section.

## Print Button

Prints the individual test visible in the test session window.


## Delete Button

Deletes the individual test visible in the test session window.

When the session is complete, select **End Session** to save the session and return to the patient database.


## Spirometry Options

Select **Options** | **Spirometry** from the menu bar.

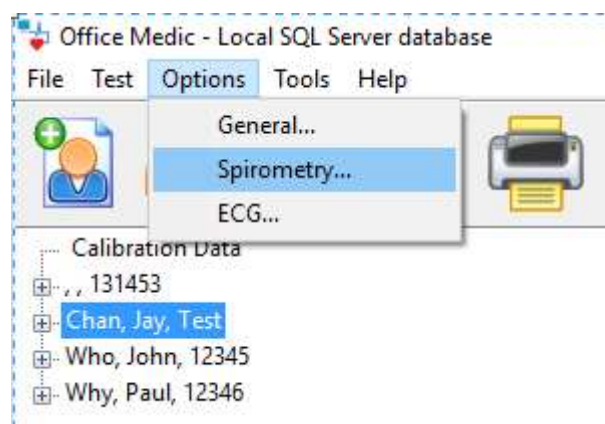

### General Tab

Select **General** to change the graphical incentive displayed.

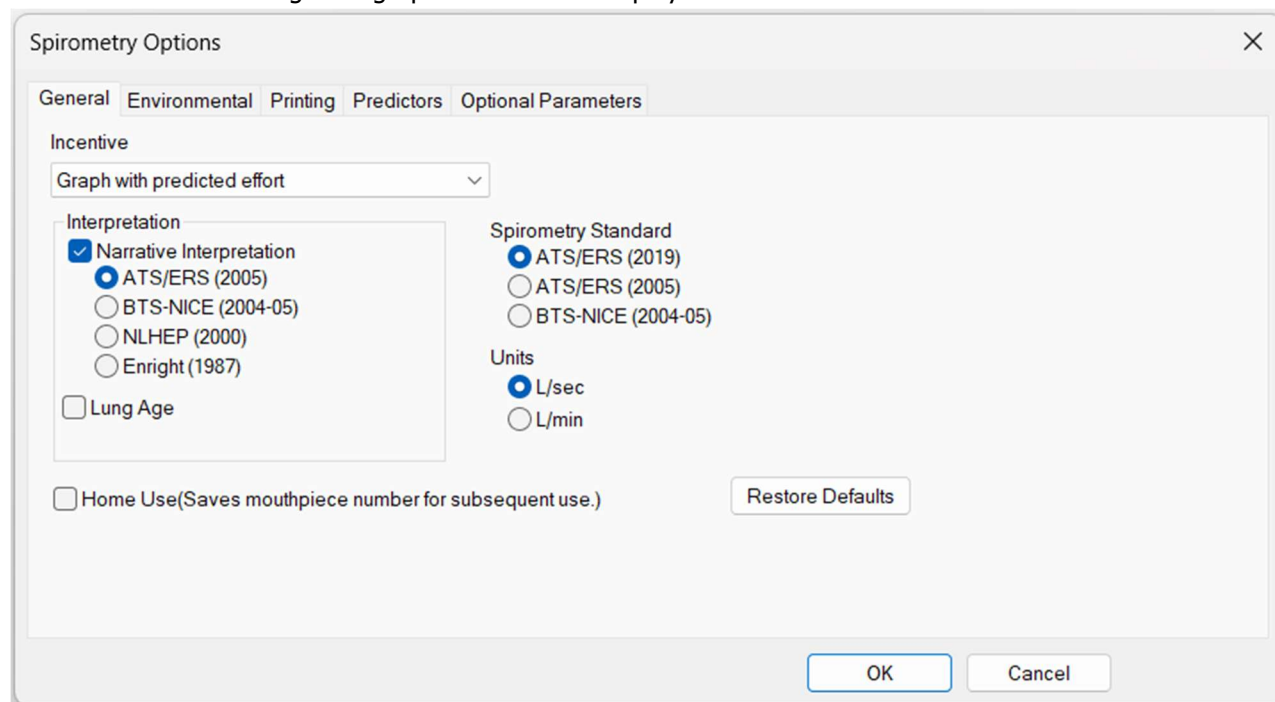

### Interpretation

Turn the **Narrative Interpretation** and **Lung Age** options ON and OFF. For details on the interpretation criteria see the [Spirometry Interpretation](#) section. For details on the Lung Age calculation see the [Lung Age Calculation](#) section.

### Spirometry Standard


Select between the ATS/ERS (2019), (2005) and the BTS-NICE (2004-05) standard. ATS 2019 is recommended.

### Units

Select to have results displayed in Liters per second (L/sec) or Liters per minute (L/min)

## Environmental Tab

Select **Environmental** to adjust environmental conditions such as temperature, elevation and barometric pressure.

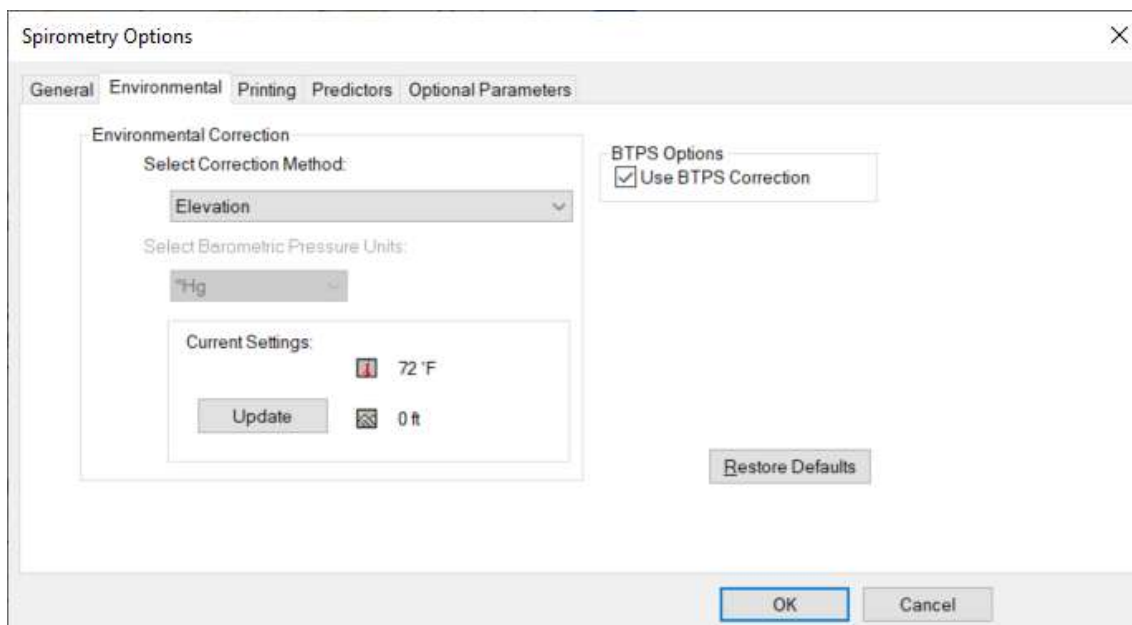

- Elevation: Elevation is your altitude above sea level. Use this option if you do not have a barometer.
- Elevation with Relative Barometric Pressure: The relative barometric pressure is the measured air pressure in your area and varies from day to day.
- Absolute Barometric Pressure: Absolute barometric pressure is the true barometric pressure observed at a specific elevation and not corrected for altitude above mean sea level.

## Select Barometric Pressure Units

Select the units of barometric pressure in either inches of Mercury ("Hg), millimeters of Mercury (mmHg) or millibars hPa (mb).

## Current Settings


Select the **Update** button to change temperature, barometric pressure and elevation data.

## BTPS Options

Use BTPS Correction should be turned on when testing patients. For calibration testing BTPS is automatically turned off and Room Temperature cannot be adjusted.

## Printing Tab

Select **Printing** to change or activate printing options:

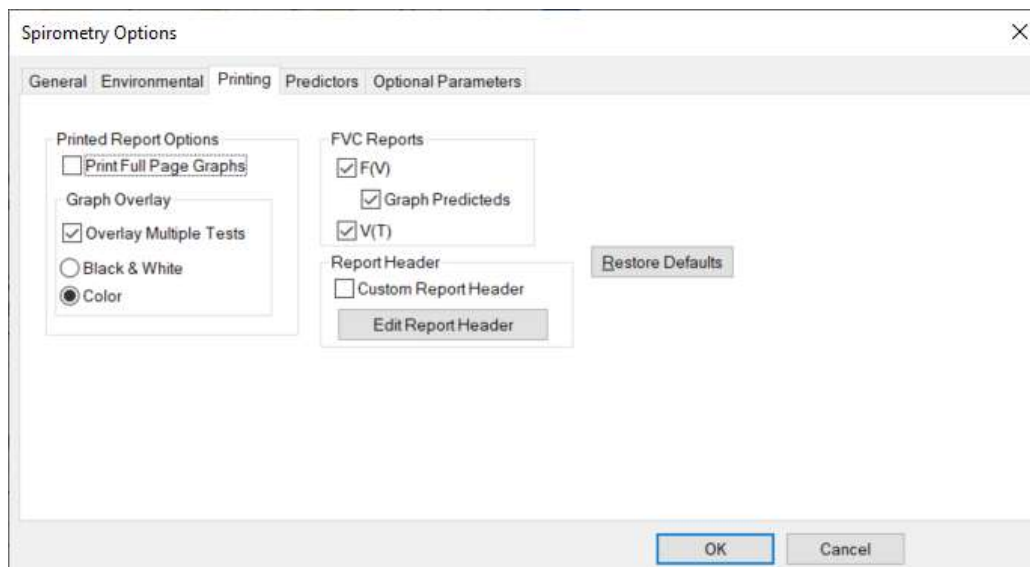

### Print Full Page Graphs

Prints two additional pages, containing full page F(V) and V(T) graphs, in the report.

### Overlay Pre Tests

Overlays the best three Pretests in **Color** or **Black & White**.

---

Note: When a Post test is performed the report will overlay the best Pre and best Post test. Once a Post test is performed, the best three Pretests will not overlay on the report.

---

### Custom Report Header

Select **Edit Report Header** to create or edit a custom header. Select the **Custom Report Header** checkbox to activate the custom report header.

---

Note: Report headers contain patient demographics.

---

### FVC Reports

Prints the F(V) and/or V(T) graphs at the bottom of the report. Select the **Graph Predicteds** options to have the predicted values plot on the F(V) report.

---

Note: Predicteds will not plot on V(T) graphs.

---

## Predictors Tab

Select **Predictors** to change or activate the Predictor options.

The screenshot shows the 'Spirometry Options' dialog box with the 'Predictors' tab selected. The dialog has a title bar with a close button (X) and a tabbed interface with 'General', 'Environmental', 'Printing', 'Predictors', and 'Optional Parameters'. The 'Predictors' tab contains two main sections: 'FVC/SVC Predictors' and 'MVV Predictors'. The 'FVC/SVC Predictors' section has four dropdown menus: 'Adult First Choice' (GLI-G (Race-Neutral)), 'Adult Second Choice' (GLI 2012), 'Pediatric First Choice' (GLI-G (Race-Neutral)), and 'Pediatric Second Choice' (GLI 2012). The 'MVV Predictors' section has two dropdown menus: 'Adult Predictor' (Cherniack '72) and 'Pediatric Predictor' (Polgar '71). To the right of these dropdowns is the text 'Refer to User's Manual for race corrections.' Below these sections is a 'Settings' section with a text input field for 'If the subject is >= 18 years old, use the Adult predictor equation.' and two text input fields for 'If a predicted equation does not include race corrections use: 0 % for Black Subjects' and '0 % for Asian subjects'. At the bottom right of the dialog is a 'Restore Defaults' button, and at the very bottom are 'OK' and 'Cancel' buttons.

### Predictors

A first and second Predictor choice is allowed. Should a patient fall out of the age or height range of the first choice predictor, the second predictor will be used. If the patient falls out of range of both predictors, no predicted data will be shown. See the [Predicted Value Equations](#) section for equation parameters.

### Settings

Sets a race correction for Blacks and Asians. The correction is applied to the predicted value and predicted value LLN. The software default is 12% for Blacks and 6% for Asians. Enter 0% if you do not want to correct for race.

## Optional Parameters Tab

Select **Optional Parameters** to set the parameters displayed on reports.

The image shows a software dialog box titled "Spirometry Options" with a close button (X) in the top right corner. The dialog has five tabs: "General", "Environmental", "Printing", "Predictors", and "Optional Parameters", with the last one being selected. The "Optional Parameters" tab contains several groups of checkboxes for selecting parameters to display on reports. The "FVC" group includes 18 parameters, all of which are checked. The "SVC" group has one checked parameter, "SVC". The "MWV" group has three checked parameters: "MWV", "RR", and "MTV". The "Show Acceptability & Usability Criteria" group has one checked parameter, "Show By Default". The "Z-Score" group has two checked parameters: "Z-Score" and "Predicted". A "Restore Defaults" button is located to the right of the "Z-Score" group. At the bottom of the dialog are "OK" and "Cancel" buttons.

Parameter	Checked
FVC	Yes
FEV0.5	Yes
FEF25%	Yes
FIV0.5	Yes
PIFR	Yes
Ext. Vol	Yes
FEV1	Yes
FEV3	Yes
FEF50%	Yes
FIV1	Yes
FIF50%	Yes
FET	Yes
FEV1/FVC	Yes
FEV3/FVC	Yes
FEF75%	Yes
FIV3	Yes
FIF25-75%	Yes
EOTV	Yes
FEV6	Yes
PEFR	Yes
FEF25-75%	Yes
FIV1/FIVC	Yes
FIF.2-1.2	Yes
FEV1/FEV6	Yes
PEFT	Yes
FIVC	Yes
FIV3/FIVC	Yes
FVC/FIVC	Yes

SVC

SVC

Show Acceptability & Usability Criteria

Show By Default

MWV

MWV    RR    MTV

Z-Score

Z-Score    Predicted

Restore Defaults

OK   Cancel

## Spirometry Tools


### Spirometry Calibration Check

There are two methods for accessing and storing the Calibration test:

1. Select **Tools | Spirometry | Perform Calibration Check**. This method stores the calibration report chronologically under **Calibration Data** in the Patient Directory window.



2. Select **Calibration Check** within a test session window. This method appends the calibration results to the patient's spirometry test report.

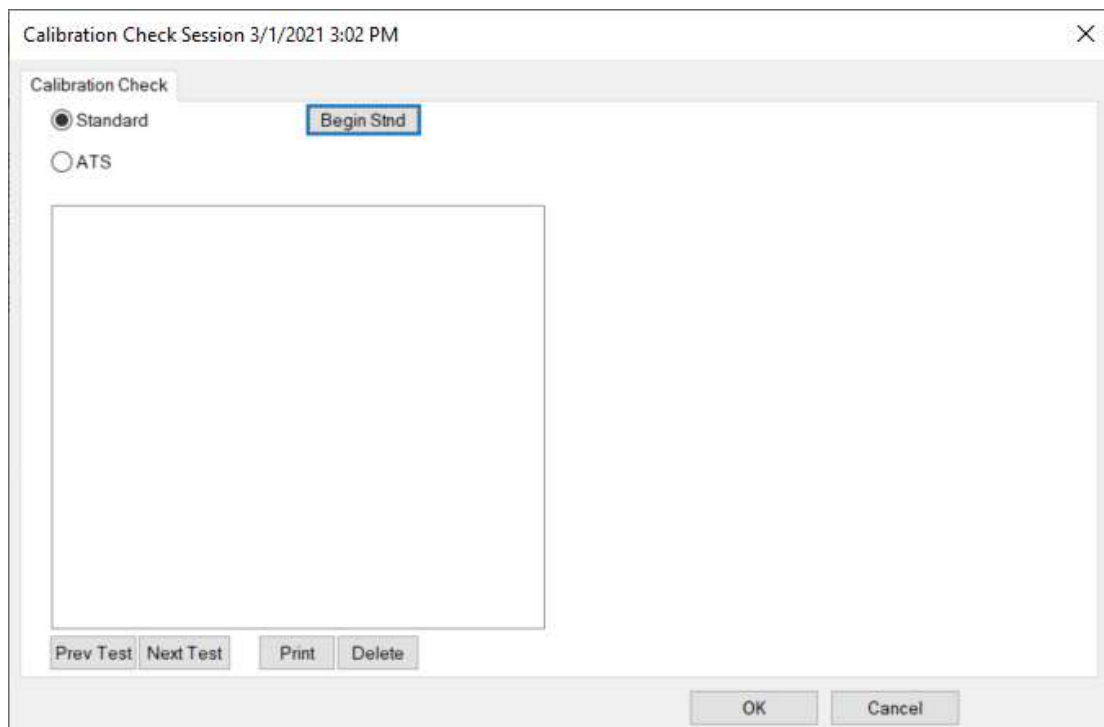

There are two methods of calibration:

- Standard - A single volumetric test.
- ATS - ATS 3-speed flow and volume test.


---

Note: The Spirometer does not require a calibration check in order to operate.

---

### To check calibration: Orbit Portable Spirometer

1. Insert the USB cable into the USB port.
2. Connect the pressure tube to the Luer fitting.
3. Connect the pressure tube to the mouthpiece.

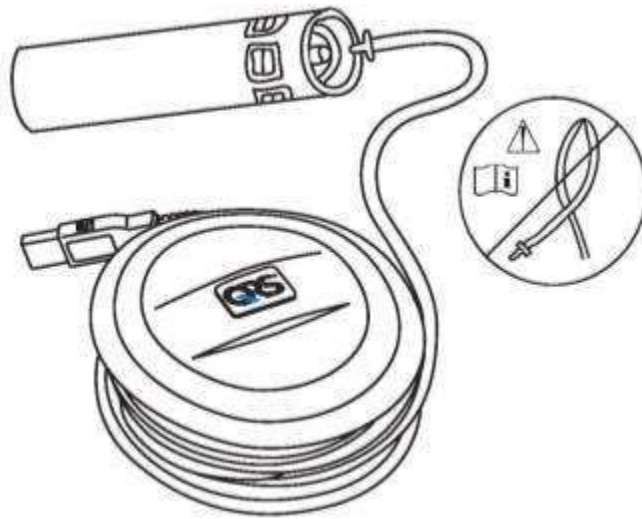

4. Connect a syringe to the mouthpiece (recommended 3-liter syringe).

---

Note: The calibration syringe must form a tight seal around the mouthpiece. If you are unable to get a tight seal contact Technical Support for more information.

---

5. Select the desired calibration check:
  - For standard calibration select **Begin Stnd**, enter the mouthpiece number and the syringe volume (1 to 10 liters) and select **OK**.
  - For ATS/ERS 2019 calibration select **ATS** and enter the mouthpiece number and select **OK**. A 3-liter syringe must be used.
6. When both circles stop flashing and turn green push the syringe in fully.

---

Note: The calibration check is for verification only. If the spirometer is found to be out of calibration, repeat with a different mouthpiece. If the problem persists, see [Service](#).

---

## Predicted Value Equations

### Predicted Study Summary Table

Reference	Abbreviation	Gender	Age Range [yrs]	Height Range	Race				FVC FEV1	FEV1/FVC	FEV6	FEV1/FEV6	FEF25-75%	PEFR	FEF25%	FEF50%	FEF75%	MVV	SVC	FEV0.5	FEV3	FEV3/FVC	FET	FIVC	
					Caucasian	Black	Mexican - American	Asian																	
NHANES III (1999)	Nh	M	8-19	48-75.6 in (122-192 cm)	X				X	X	X	X	X	X	X										
		M	8-19	48-76.4 in (122-194 cm)		X			X	X	X	X	X	X	X										
		M	8-19	47.2-70.9 in (120-180 cm)			X		X	X	X	X	X	X	X										
		M	20-80	62.2-76.4 in (158-194 cm)	X				X	X	X	X	X	X	X										
		M	20-80	62.2-77.2 in (158-196 cm)		X			X	X	X	X	X	X	X										
		M	20-80	61.4-75.6 in (156-192 cm)			X		X	X	X	X	X	X	X										
		F	8-17	46.5-70.1 in (118-178 cm)	X				X	X	X	X	X	X	X										
		F	8-17	46.5-72.4 in (118-184 cm)		X			X	X	X	X	X	X	X										
		F	8-17	44.9-67.7 in (114-172 cm)			X		X	X	X	X	X	X	X										
		F	18-80	57.1-70.9 in (145-180 cm)	X				X	X	X	X	X	X	X										
F	18-80	53.5-70.9 in (136-180 cm)		X			X	X	X	X	X	X	X												
F	18-80	53.5-67.7 in (136-172 cm)			X		X	X	X	X	X	X	X												
ECCS/ERS (Quanjer 1993)	EC	M	18-70	61-76.8 in (155-195 cm)	X				X	X	X			X	X	X	X						X		
		F	18-70	57.1-70.9 in (145-180 cm)	X				X	X	X			X	X	X	X						X		
Wang (1993)	Wg	M	6-18	43.3-74.8 in (110-190 cm)	X				X	X	X			X											
		M	6-18	47.2-74.8 in (120-190 cm)		X			X	X	X			X											
		F	6-18	43.3-70.9 in (110-180 cm)	X				X	X	X			X											
Quanjer (1995)	Qu	F	6-18	47.2-70.9 in (120-180 cm)		X			X	X	X			X											
		M	6-18	43.3-80.7 in (110-205 cm)	X				X	X	X			X											
		F	6-18	43.3-72.8 in (110-185 cm)	X				X	X	X			X											
Zapletal (1987)	Za	M	6-18	42.1-71.7 in (107-182 cm)	X				X	X	X			X	X	X	X	X	X	X	X				
		F	6-18	42.1-71.7 in (107-182 cm)	X				X	X	X			X	X	X	X	X	X	X	X				
Morris (1971/73)	Mo	M	20-90	58-80 in (147.3-203.2 cm)	X				X	X				X											
		M	20-79	58-80 in (147.3-203.2 cm)	X						X														
		F	20-90	56-72 in (142.2-182.9 cm)	X				X	X				X											
Cherniack (1972)	Ch	F	20-79	56-72 in (142.2-182.9 cm)	X						X														
		M	15-79	35-85 in (88.9-215.9 cm)	X				X	X				X	X	X	X	X	X	X					
Roberts (1991)	Ro	M	18-86	63.4-77.2 in (161-196 cm)	X				X	X	X			X											
		F	18-86	57.5-69.7 in (146-177 cm)	X				X	X	X			X											
Knudson (1983)	Kn	M	6-11	44-61 in (111.8-154.9 cm)	X				X	X	X			X											
		M	12-24	55-76 in (139.7-193 cm)	X				X	X	X			X											
		M	25+	62-77 in (157.5-195.6 cm)	X				X	X				X											
		M	25-85	62-77 in (157.5-195.6 cm)	X						X														
		F	6-10	42-58 in (106.7-147.3 cm)	X				X	X	X			X											
		F	11-19	52-72 in (132.1-182.9 cm)	X				X	X	X			X											
		F	20-69	58-71 in (147.3-180.3 cm)	X				X	X				X											
		F	20-88	58-71 in (147.3-180.3 cm)	X						X														
Hsu (1979)	Hs	F	70+	58-66 in (147.3-167.6 cm)	X				X	X			X												
		M	7-20	43.7-74.8 in (111-190 cm)	X				X	X	X			X	X										
		M	7-20	43.7-74.8 in (111-190 cm)		X			X	X	X			X	X										
		F	7-18	43.7-74.8 in (111-190 cm)	X				X	X	X			X	X										
		F	7-18	43.7-74.8 in (111-190 cm)		X			X	X	X			X	X										
		F	7-18	43.7-74.8 in (111-190 cm)			X		X	X	X			X	X										

Crapo (1961)	Cr	M 15–91	61.8–76.4 in (157–194 cm)	X			X	X	X		X					X	X	
		F 17–84	57.5–70.1 in (146–178 cm)	X			X	X	X		X						X	X
Warwick (1977)	Wa	M < 18	35.4–74 in (90–188 cm)	X			X	X	X		X		X	X				X
		F < 18	35.4–70.1 in (90–178 cm)	X			X	X	X		X		X	X				X
Poldar (1971)	Po	M 4–17	43.3–67 in (110–170 cm)	X			X	X	X		X	X				X		
		F 4–17	43.3–67 in (110–170 cm)	X			X	X	X		X	X				X		

Shaded = LLN available

## MORRIS (1971/1973)

Morris, James F., et. Al.: Spirometric Standards for Healthy Non-smoking Adults. American Review of Respiratory Disease 1971; vol 103(1): 57–67.

Morris, James F, et al.: Normal values for the ratio of one-second forced expiratory volume to forced vital capacity. American Review of Respiratory Disease 1973 Vol 108: 1000–1003.

### MALE

20–90 years,  
58–80 in. (147.3–203.2 cm)

$$\text{FVC (L)} = 0.148 * \text{H[in]} - 0.025 * \text{A[yr]} - 4.241$$

$$\text{FEV1 (L)} = 0.092 * \text{H[in]} - 0.032 * \text{A[yr]} - 1.26$$

$$\text{FEF25–75\% (L/sec)} = 0.047 * \text{H[in]} - 0.045 * \text{A[yr]} + 2.513$$

### MALE, 20–79 years

$$\text{FEV1/FVC (L/sec)} = (-0.3118 * \text{H[in]} - 0.2422 * \text{A[yr]} + 107.12)/100$$

### FEMALE

20–90 years,  
56–72 in. (142.2–182.9 cm)

$$\text{FVC} = 0.115 * \text{H[in]} - 0.024 * \text{A[yr]} - 2.852$$

$$\text{FEV1} = 0.089 * \text{H[in]} - 0.025 * \text{A[yr]} - 1.932$$

$$\text{FEF25–75\%} = 0.06 * \text{H[in]} - 0.03 * \text{A[yr]} + 0.551$$

### FEMALE, 20–79 years

$$\text{FEV1/FVC (L/sec)} = (-0.0679 * \text{H[in]} - 0.1815 * \text{A[yr]} + 88.7)/100$$

## CHERNIACK (1972)

Cherniack, RM and Raber, MB: Normal Standards for Ventilatory Function Using an Automatic Wedge Spirometer American Review of Respiratory Disease 1972; Vol 106(1), p38–46.

### MALE

15–79 years,  
35–85 in. (88.9–215.9 cm)

$$\text{FVC (L)} = 0.12102 * \text{H[in]} - 0.01357 * \text{A[yr]} - 3.18373$$

$$\text{FEV1 (L)} = 0.09107 * \text{H[in]} - 0.0232 * \text{A[yr]} - 1.50723$$

$$\text{FEF25\% (L/sec)} = 0.0903 * \text{H[in]} - 0.01987 * \text{A[yr]} + 2.72554$$

$$\text{FEF50\% (L/sec)} = 0.06526 * \text{H[in]} - 0.03049 * \text{A[yr]} + 2.40337$$

$$\text{FEF75\% (L/sec)} = 0.03583 * \text{H[in]} - 0.04142 * \text{A[yr]} + 1.98361$$

$$\text{FEF25–75\% (L/sec)} = 0.05948 * \text{H[in]} - 0.037 * \text{A[yr]} + 2.61187$$

$$\text{PEFR} = 0.14393 * \text{H[in]} - 0.02403 * \text{A[yr]} + 0.22544$$

$$\text{MVV} = 3.02915 * \text{H[in]} - 0.81621 * \text{A[yr]} - 37.94893$$

### FEMALE

15–79 years,  
35–85 in. (88.9–215.9 cm)

$$\text{FVC (L)} = 0.07833 * \text{H[in]} - 0.01539 * \text{A[yr]} - 1.04912$$

$$\text{FEV1 (L)} = 0.06029 * \text{H[in]} - 0.01936 * \text{A[yr]} - 0.18693$$

$$\text{FEF25\% (L/sec)} = 0.06876 * \text{H[in]} - 0.01926 * \text{A[yr]} + 2.14653$$

$$\text{FEF50\% (L/sec)} = 0.0622 * \text{H[in]} - 0.02344 * \text{A[yr]} + 1.4264$$

$$\text{FEF75\% (L/sec)} = 0.02334 * \text{H[in]} - 0.0345 * \text{A[yr]} + 2.21596$$

$$\text{FEF25–75\% (L/sec)} = 0.04931 * \text{H[in]} - 0.0312 * \text{A[yr]} + 2.2561$$

$$\text{PEFR} = 0.0913 * \text{H[in]} - 0.01776 * \text{A[yr]} + 1.1316$$

$$\text{MVV} = 2.13844 * \text{H[in]} - 0.68503 * \text{A[yr]} - 4.86957$$

## ROBERTS (1991)

Roberts, Michael C. et. al: Reference values and prediction equations for normal lung function in non-smoking white

urban population. Thorax 1991; 46: 643–650

**MALE**  
18–86 years,  
63.4–77.2 in. (161–196 cm)  
FVC (L) =  $0.06628 * H[\text{cm}] - 0.028 * A[\text{yrs}] - 5.377$   
FEV1 (L) =  $0.03961 * H[\text{cm}] - 0.033 * A[\text{yrs}] - 1.558$   
FEV1/FVC =  $(-0.21476 * H[\text{cm}] - 0.242 * A[\text{yrs}] + 126.252)/100$   
PEFR =  $0.05317 * H[\text{cm}] - 0.062 * A[\text{yrs}] + 3.884$   
FEF50% (L/sec) =  $-0.044 * A[\text{yrs}] + 6.456$

**FEMALE**  
18–86 years,  
57.5–69.7 in. (146–177 cm)  
FVC (L) =  $0.04321 * H[\text{cm}] - 0.023 * A[\text{yrs}] - 2.379$   
FEV1 (L) =  $0.03321 * H[\text{cm}] - 0.025 * A[\text{yrs}] - 1.394$   
FEV1/FVC =  $(-0.172 * A[\text{yrs}] + 88.134)/100$   
PEFR =  $0.04087 * H[\text{cm}] - 0.05 * A[\text{yrs}] + 2.945$   
FEF50% (L/sec) =  $-0.038 * A[\text{yrs}] + 5.556$

### KNUDSON (1983)

Knudson, Ronald J., et. al: Change in the Normal Maximum Expiratory Flow-Volume Curve with Growth and Aging. American Review of Respiratory Disease 1983; 127(5–6): 725–734.

**MALE**  
6–11 years,  
44–61 in. (111.8–154.9 cm)  
FVC (L) =  $0.0409 * H[\text{cm}] - 3.3756$   
FEV1 (L) =  $0.0348 * H[\text{cm}] - 2.8142$   
FEF50% (L/sec) =  $0.0378 * H[\text{cm}] - 2.5454$   
FEF75% (L/sec) =  $0.0171 * H[\text{cm}] - 1.0149$   
FEF25–75% (L/sec) =  $0.0338 * H[\text{cm}] - 2.3197$   
FEV1/FVC =  $100.4389 - 0.0813 * H[\text{cm}]$

**MALE**  
12–24 years,  
55–76 in. (139.7–193.0 cm)  
FVC (L) =  $0.059 * H[\text{cm}] + 0.0739 * A[\text{yrs}] - 6.8865$   
FEV1 (L) =  $0.0519 * H[\text{cm}] + 0.0636 * A[\text{yrs}] - 6.1181$   
FEF50% (L/sec) =  $0.0543 * H[\text{cm}] + 0.115 * A[\text{yrs}] - 6.3851$   
FEF75% (L/sec) =  $0.0397 * H[\text{cm}] - 0.0057 * A[\text{yrs}] - 4.2421$   
FEF25–75% (L/sec) =  $0.0539 * H[\text{cm}] + 0.0749 * A[\text{yrs}] - 6.199$   
FEV1/FVC =  $100.4389 - 0.0813 * H[\text{cm}]$

**MALE**  
25+ years,  
62–77 in. (157.5–195.6 cm)  
FVC (L) =  $0.0844 * H[\text{cm}] - 0.0298 * A[\text{yrs}] - 8.7818$   
FEV1 (L) =  $0.0665 * H[\text{cm}] - 0.0292 * A[\text{yrs}] - 6.5147$   
FEF50% (L/sec) =  $0.0684 * H[\text{cm}] - 0.0366 * A[\text{yrs}] - 5.5409$   
FEF75% (L/sec) =  $0.031 * H[\text{cm}] - 0.023 * A[\text{yrs}] - 2.4827$   
FEF25–75% (L/sec) =  $0.0579 * H[\text{cm}] - 0.0363 * A[\text{yrs}] - 4.5175$   
**MALE  $\geq$  25 and  $<$  85 years**  
FEV1/FVC =  $86.6862 - 0.105 * A[\text{yrs}]$

**FEMALE**  
6–10 years,  
42–58 in. (106.7–147.3 cm)  
FVC (L) =  $0.043 * H[\text{cm}] - 3.7486$   
FEV1 (L) =  $0.0336 * H[\text{cm}] - 2.7578$   
FEF50% (L/sec) =  $0.1846 * A[\text{yrs}] + 0.7362$   
FEF75% (L/sec) =  $0.0109 * H[\text{cm}] - 0.1657$   
FEF25–75% (L/sec) =  $0.022 * H[\text{cm}] - 0.8119$   
FEV1/FVC =  $109.9739 - 0.1909 * H[\text{cm}] + 0.6655 * A[\text{yrs}]$

**FEMALE**  
11–19 years,  
52–72 in. (132.1–182.9 cm)  
FVC (L) =  $0.0416 * H[\text{cm}] + 0.0699 * A[\text{yrs}] - 4.447$   
FEV1 (L) =  $0.0351 * H[\text{cm}] + 0.0694 * A[\text{yrs}] - 3.7622$   
FEF50% (L/sec) =  $0.0288 * H[\text{cm}] + 0.1111 * A[\text{yrs}] - 2.304$   
FEF75% (L/sec) =  $0.0243 * H[\text{cm}] + 0.2923 * A[\text{yrs}] - 4.4009 - 0.0075 * A[\text{yrs}]^2$   
FEF25–75% (L/sec) =  $0.0279 * H[\text{cm}] + 0.1275 * A[\text{yrs}] - 2.8007$   
FEV1/FVC =  $109.9739 - 0.1909 * H[\text{cm}] + 0.6655 * A[\text{yrs}]$

**FEMALE**  
20–69 years,  
58–71 in. (147.3–180.3 cm)  
FVC (L) =  $0.0444 * H[\text{cm}] - 0.0169 * A[\text{yrs}] - 3.1947$   
FEV1 (L) =  $0.0332 * H[\text{cm}] - 0.019 * A[\text{yrs}] - 1.821$   
FEF50% (L/sec) =  $0.0321 * H[\text{cm}] - 0.024 * A[\text{yrs}] - 0.4371$

$$\begin{aligned} \text{FEF75\% (L/sec)} &= 0.0174 * \text{H[cm]} - 0.0254 * \text{A[yr]} - 0.1822 \\ \text{FEF25-75\% (L/sec)} &= 0.03 * \text{H[cm]} - 0.0309 * \text{A[yr]} - 0.4057 \\ \text{FEMALE } \geq 20 \text{ and } < 88 \text{ years} \\ \text{FEV1/FVC} &= 121.6777 - 0.1852 * \text{H[cm]} - 0.1896 * \text{A[yr]} \end{aligned}$$

**FEMALE**

70+ years,  
58-66 in. (147.3-167.6 cm)

$$\begin{aligned} \text{FVC (L)} &= 0.0313 * \text{H[cm]} - 0.0296 * \text{A[yr]} - 0.1889 \\ \text{FEV1 (L)} &= 0.0143 * \text{H[cm]} - 0.0397 * \text{A[yr]} + 2.6539 \\ \text{FEF50\% (L/sec)} &= 0.0118 * \text{H[cm]} - 0.0755 * \text{A[yr]} + 6.2402 \\ \text{FEF75\% (L/sec)} &= -0.0172 * \text{A[yr]} + 1.8894 \\ \text{FEF25-75\% (L/sec)} &= -0.0615 * \text{A[yr]} + 6.3706 \end{aligned}$$

**HSU (1979)**

Hsu, Katharine, et. al.: Ventilatory Functions of Normal Children and Young Adults – Mexican American, White and Black. J Pediatr 1979; 95: 14-23.

To determine the Predicted FEV1/FVC value for this predicted set QRS software uses:

Pred FEV1/Pred FVC

**MALE, White**

7-20 years,  
43.7-74.8 in. (111-190 cm)

$$\begin{aligned} \text{FVC [L]} &= (0.000358 * \text{H[cm]}^{3.18})/1000 \\ \text{FEV1 [L]} &= (0.000774 * \text{H[cm]}^3)/1000 \\ \text{PEFR [L/min]} &= 0.000335 * \text{H[cm]}^{2.79} \\ \text{FEF25-75\% [L/min]} &= 0.000798 * \text{H[cm]}^{2.46} \end{aligned}$$

**MALE, Black**

7-20 years,  
43.7-74.8 in. (111-190 cm)

$$\begin{aligned} \text{FVC [L]} &= (0.00107 * \text{H[cm]}^{2.93})/1000 \\ \text{FEV1 [L]} &= (0.00103 * \text{H[cm]}^{2.92})/1000 \\ \text{PEFR [L/min]} &= 0.000174 * \text{H[cm]}^{2.92} \\ \text{FEF25-75\% [L/min]} &= 0.000361 * \text{H[cm]}^{2.60} \end{aligned}$$

**MALE, Mexican-American**

7-20 years,  
43.7-74.8 in. (111-190 cm)

$$\begin{aligned} \text{FVC [L]} &= (0.00106 * \text{H[cm]}^{2.97})/1000 \\ \text{FEV1 [L]} &= (0.00173 * \text{H[cm]}^{2.85})/1000 \\ \text{PEFR [L/min]} &= 0.000769 * \text{H[cm]}^{2.63} \\ \text{FEF25-75\% [L/min]} &= 0.000913 * \text{H[cm]}^{2.45} \end{aligned}$$

**FEMALE, White**

7-18 years,  
43.7-74.8 in. (111-190 cm)

$$\begin{aligned} \text{FVC [L]} &= (0.00257 * \text{H[cm]}^{2.78})/1000 \\ \text{FEV1 [L]} &= (0.00379 * \text{H[cm]}^{2.68})/1000 \\ \text{PEFR [L/min]} &= 0.00258 * \text{H[cm]}^{2.37} \\ \text{FEF25-75\% [L/min]} &= 0.00379 * \text{H[cm]}^{2.16} \end{aligned}$$

**FEMALE, Black**

7-18 years,  
43.7-74.8 in. (111-190 cm)

$$\begin{aligned} \text{FVC [L]} &= (0.000834 * \text{H[cm]}^{2.98})/1000 \\ \text{FEV1 [L]} &= (0.00114 * \text{H[cm]}^{2.89})/1000 \\ \text{PEFR [L/min]} &= 0.000551 * \text{H[cm]}^{2.68} \\ \text{FEF25-75\% [L/min]} &= 0.00145 * \text{H[cm]}^{2.34} \end{aligned}$$

**FEMALE, Mexican-American**

7-18 years,  
43.7-74.8 in. (111-190 cm)

$$\begin{aligned} \text{FVC [L]} &= (0.00125 * \text{H[cm]}^{2.92})/1000 \\ \text{FEV1 [L]} &= (0.00161 * \text{H[cm]}^{2.85})/1000 \\ \text{PEFR [L/min]} &= 0.000697 * \text{H[cm]}^{2.64} \\ \text{FEF25-75\% [L/min]} &= 0.00120 * \text{H[cm]}^{2.40} \end{aligned}$$

**CRAPO (1981)**

Crapo, et. al: Reference Spirometric Values using Techniques and Equipment that Meet ATS Recommendations. American Review of Respiratory Disease 1981; 123: 659-664.

**MALE**

15-91 years,  
61.8-76.4 in. (157-194 cm)

$$\begin{aligned} \text{FVC (L)} &= 0.06 * \text{H[cm]} - 0.0214 * \text{A[yr]} - 4.65 \\ \text{FEV05 (L)} &= 0.0327 * \text{H[cm]} - 0.0152 * \text{A[yr]} - 1.914 \\ \text{FEV1 (L)} &= 0.0414 * \text{H[cm]} - 0.0244 * \text{A[yr]} - 2.19 \\ \text{FEV3 (L)} &= 0.0535 * \text{H[cm]} - 0.0271 * \text{A[yr]} - 3.512 \\ \text{FEF25-75\% (L/sec)} &= 0.0204 * \text{H[cm]} - 0.038 * \text{A[yr]} + 2.133 \end{aligned}$$

$$\text{FEV1/FVC} = (-0.13 * \text{H[cm]} - 0.152 * \text{A[yr]} + 110.49)/100$$

$$\text{FEV3/FVC} = (-0.0627 * \text{H[cm]} - 0.145 * \text{A[yr]} + 112.09)/100$$

**FEMALE**

17–84 years,  
57.5–70.1 in. (146–178 cm)

$$\text{FVC (L)} = 0.0491 * \text{H[cm]} - 0.0216 * \text{A[yr]} - 3.59$$

$$\text{FEV05 (L)} = 0.0238 * \text{H[cm]} - 0.0185 * \text{A[yr]} - 0.809$$

$$\text{FEV1 (L)} = 0.0342 * \text{H[cm]} - 0.0255 * \text{A[yr]} - 1.578$$

$$\text{FEV3 (L)} = 0.0442 * \text{H[cm]} - 0.0257 * \text{A[yr]} - 2.745$$

$$\text{FEF25–75\%} = 0.0154 * \text{H[cm]} - 0.046 * \text{A[yr]} + 2.683$$

$$\text{FEV1/FVC} = (-0.202 * \text{H[cm]} - 0.252 * \text{A[yr]} + 126.58)/100$$

$$\text{FEV3/FVC} = (-0.0937 * \text{H[cm]} - 0.163 * \text{A[yr]} + 118.16)/100$$

**WARWICK (1977/80)**

Warwick, WJ: Pulmonary Function in Healthy Minnesota Children. Minnesota Medicine 1977; Supplement 60: 435–440.  
Warwick, WJ: Pulmonary Function in Healthy Minnesota Children. Minnesota Medicine March 1980; 191–195.

**MALE**

< 18 YEARS,  
35.4–74 in. (90–188 cm)

$$\text{LnFVC (L)} = 3.0131 * \ln(\text{H[cm]}) - 14.0535$$

$$\text{LnFEV1 (L)} = 2.7572 * \ln(\text{H[cm]}) - 12.9007$$

$$\text{LnFEV1/FVC} = -0.2679 * \ln(\text{H[cm]}) + 1.2137$$

$$\text{LnFEF50\% (L/sec)} = 2.1326 * \ln(\text{H[cm]}) - 9.3589$$

$$\text{LnFEF75\% (L/sec)} = 2.1534 * \ln(\text{H[cm]}) - 10.2213$$

$$\text{LnPEFR (L/sec)} = 2.4991 * \ln(\text{H[cm]}) - 10.7785$$

$$\text{LnFET (s)} = 1.6208 * \ln(\text{H[cm]}) - 7.2327$$

**FEMALE**

< 18 YEARS,  
35.4–70.1 in. (90–178 cm)

$$\text{LnFVC (L)} = 2.9446 * \ln(\text{H[cm]}) - 13.8007$$

$$\text{LnFEV1 (L)} = 2.7522 * \ln(\text{H[cm]}) - 12.921$$

$$\text{LnFEV1/FVC} = -0.2126 * \ln(\text{H[cm]}) + 0.9719$$

$$\text{LnFEF50\% (L/sec)} = 2.1958 * \ln(\text{H[cm]}) - 9.6458$$

$$\text{LnFEF75\% (L/sec)} = 2.2961 * \ln(\text{H[cm]}) - 10.8666$$

$$\text{LnPEFR (L/sec)} = 2.4369 * \ln(\text{H[cm]}) - 10.535$$

$$\text{LnFET (s)} = 1.2423 * \ln(\text{H[cm]}) - 5.3288$$

**POLGAR (1971)**

Polgar and Promadhat: Pulmonary Function Testing in Children: Techniques and Standards 1971.

To determine the Predicted FEV1/FVC value for this predicted set QRS software uses:

Pred FEV1/Pred FVC

**MALE**

4–17 years,  
43.3–67 in. (110–170 cm)

$$\text{FVC (L)} = 0.0000044 * \text{H[cm]}^{2.67}$$

$$\text{FEV1 (L)} = 0.0000021 * \text{H[cm]}^{2.8}$$

$$\text{FEF25–75\% (L/min)} = -207.70 + 2.621 * \text{H[cm]}$$

$$\text{PEFR (L/min)} = -425.5714 + 5.2428 * \text{H[cm]}$$

$$\text{MVV} = 1.276 * \text{H[cm]} - 99.507$$

**FEMALE**

4–17 years,  
43.3–67 in. (110–170 cm)

$$\text{FVC (L)} = 0.0000033 * \text{H[cm]}^{2.72}$$

$$\text{FEV1 (L)} = 0.0000021 * \text{H[cm]}^{2.8}$$

$$\text{FEF25–75\% (L/min)} = -207.70 + 2.621 * \text{H[cm]}$$

$$\text{PEFR (L/min)} = -425.5714 + 5.2428 * \text{H[cm]}$$

$$\text{MVV} = 1.276 * \text{H[cm]} - 99.507$$

**ECCS/ERS (Quanjer 1993)**

Quanjer, Ph.H, et. al: Lung Volumes and Ventilatory Flows: Official Statement of the European Respiratory Society. European Respiratory Journal 1992–1993; Supplement 15–16: 5–40.

**MALE**

18–70 years,  
61–76.8 in. (155–195 cm)

$$\text{FVC (L)} = 0.0576 * \text{H[cm]} - 0.026 * \text{A[yr]} - 4.34$$

$$\text{FEV1 (L)} = 0.0430 * \text{H[cm]} - 0.029 * \text{A[yr]} - 2.49$$

$$\text{FEV1/FVC} = (-0.180 * \text{A[yr]} + 87.21)/100$$

$$\text{FEF25\% (L/sec)} = 0.0546 * \text{H[cm]} - 0.029 * \text{A[yr]} - 0.47$$

$$\text{FEF50\% (L/sec)} = 0.0379 * \text{H[cm]} - 0.031 * \text{A[yr]} - 0.35$$

For subjects aged 18–25 years the predicted mean is the same as for subjects 25 year.

$$\begin{aligned} \text{FEF}_{75\%} \text{ (L/sec)} &= 0.0261 * \text{H[cm]} - 0.026 * \text{A[yr]} - 1.34 \\ \text{FEF}_{25-75\%} \text{ (L/sec)} &= 0.0194 * \text{H[cm]} - 0.043 * \text{A[yr]} + 2.7 \\ \text{PEFR (L/sec)} &= .0614 * \text{H[cm]} - 0.043 * \text{A[yr]} + 0.15 \\ \text{FIVC} &= 0.0610 * \text{H[cm]} - 0.028 * \text{A[yr]} - 4.65 \end{aligned}$$

**FEMALE**

18–70 years,  
57.1–70.9 in. (145–180 cm)

For subjects aged 18–25 years the predicted mean is the same as for subjects 25 year.

$$\begin{aligned} \text{FVC (L)} &= 0.0443 * \text{H[cm]} - 0.026 * \text{A[yr]} - 2.89 \\ \text{FEV1 (L)} &= 0.0395 * \text{H[cm]} - 0.025 * \text{A[yr]} - 2.6 \\ \text{FEV1/FVC} &= (-0.190 * \text{A[yr]} + 89.1) / 100 \\ \text{FEF}_{25\%} \text{ (L/sec)} &= 0.0322 * \text{H[cm]} - 0.025 * \text{A[yr]} + 1.6 \\ \text{FEF}_{50\%} \text{ (L/sec)} &= 0.0245 * \text{H[cm]} - 0.025 * \text{A[yr]} + 1.16 \\ \text{FEF}_{75\%} \text{ (L/sec)} &= 0.0105 * \text{H[cm]} - 0.025 * \text{A[yr]} + 1.11 \\ \text{FEF}_{25-75\%} \text{ (L/sec)} &= 0.0125 * \text{H[cm]} - 0.034 * \text{A[yr]} + 2.92 \\ \text{PEFR (L/sec)} &= .0550 * \text{H[cm]} - 0.030 * \text{A[yr]} - 1.11 \\ \text{FIVC} &= 0.0466 * \text{H[cm]} - 0.026 * \text{A[yr]} - 3.28 \end{aligned}$$

**NHANES III (1999)**

Hankinson, John L., Odencrantz, John R., Fedan, Kathleen B.. Spirometric Reference Values from a Sample of the General U.S. Population. Am J Respir Crit Care Med 1999; Vol 159: 179–187.

**MALE Caucasian**

8–19 years, 48.0–75.6 in. (122–192 cm)

$$\begin{aligned} \text{FVC (L)} &= -0.2584 - 0.20415 * \text{A[yr]} + 0.010133 * \text{A[yr]}^2 + 0.00018642 * \text{H[cm]}^2 \\ \text{FEV1 (L)} &= -0.7453 - 0.04106 * \text{A[yr]} + 0.004477 * \text{A[yr]}^2 + 0.00014098 * \text{H[cm]}^2 \\ \text{FEV1/FVC} &= (88.066 - 0.2066 * \text{A[yr]}) / 100 \\ \text{FEV6 (L)} &= -0.3119 - 0.18612 * \text{A[yr]} + 0.009717 * \text{A[yr]}^2 + 0.00018188 * \text{H[cm]}^2 \\ \text{FEV1/FEV6} &= (87.34 - 0.1382 * \text{A[yr]}) / 100 \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= -1.0863 + 0.13939 * \text{A[yr]} + 0.00010345 * \text{H[cm]}^2 \\ \text{PEF (L/Sec)} &= -0.5962 - 0.12357 * \text{A[yr]} + 0.013135 * \text{A[yr]}^2 + 0.00024962 * \text{H[cm]}^2 \end{aligned}$$

**MALE Caucasian**

20–80 years, 62.2–76.4 in. (158–194 cm)

$$\begin{aligned} \text{FVC (L)} &= -0.1933 + 0.00064 * \text{A[yr]} - 0.000269 * \text{A[yr]}^2 + 0.00018642 * \text{H[cm]}^2 \\ \text{FEV1 (L)} &= 0.5536 - 0.01303 * \text{A[yr]} - 0.000172 * \text{A[yr]}^2 + 0.00014098 * \text{H[cm]}^2 \\ \text{FEV1/FVC} &= (88.066 - 0.2066 * \text{A[yr]}) / 100 \\ \text{FEV6 (L)} &= 0.1102 - 0.00842 * \text{A[yr]} - 0.000223 * \text{A[yr]}^2 + 0.00018188 * \text{H[cm]}^2 \\ \text{FEV1/FEV6} &= (87.34 - 0.1382 * \text{A[yr]}) / 100 \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= 2.7006 - 0.04995 * \text{A[yr]} + 0.00010345 * \text{H[cm]}^2 \\ \text{PEF (L/Sec)} &= 1.0523 + 0.08272 * \text{A[yr]} - 0.001301 * \text{A[yr]}^2 + 0.00024962 * \text{H[cm]}^2 \end{aligned}$$

**FEMALE Caucasian**

8–17 years, 46.5–70.1 in. (118–178 cm)

$$\begin{aligned} \text{FVC (L)} &= -1.2082 + 0.05916 * \text{A[yr]} + 0.00014815 * \text{H[cm]}^2 \\ \text{FEV1 (L)} &= -0.8710 + 0.06537 * \text{A[yr]} + 0.00011496 * \text{H[cm]}^2 \\ \text{FEV1/FVC} &= (90.809 - 0.2125 * \text{A[yr]}) / 100 \\ \text{FEV6 (L)} &= -1.1925 + 0.06544 * \text{A[yr]} + 0.00014395 * \text{H[cm]}^2 \\ \text{FEV1/FEV6} &= (90.107 - 0.1563 * \text{A[yr]}) / 100 \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= -2.5284 + 0.5249 * \text{A[yr]} - 0.015309 * \text{A[yr]}^2 + 0.00006982 * \text{H[cm]}^2 \\ \text{PEF (L/Sec)} &= -3.6181 + 0.60644 * \text{A[yr]} - 0.016846 * \text{A[yr]}^2 + 0.00018623 * \text{H[cm]}^2 \end{aligned}$$

**FEMALE Caucasian**

18–80 years, 57.1–70.9 in. (145–180 cm)

$$\begin{aligned} \text{FVC (L)} &= -0.356 + 0.0187 * \text{A[yr]} - 0.000382 * \text{A[yr]}^2 + 0.00014815 * \text{H[cm]}^2 \\ \text{FEV1 (L)} &= 0.4333 - 0.00361 * \text{A[yr]} - 0.000194 * \text{A[yr]}^2 + 0.00011496 * \text{H[cm]}^2 \\ \text{FEV1/FVC} &= (90.809 - 0.2125 * \text{A[yr]}) / 100 \\ \text{FEV6 (L)} &= -0.1373 + 0.01317 * \text{A[yr]} - 0.000352 * \text{A[yr]}^2 + 0.00014395 * \text{H[cm]}^2 \\ \text{FEV1/FEV6} &= (90.107 - 0.1563 * \text{A[yr]}) / 100 \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= 2.367 - 0.01904 * \text{A[yr]} - 0.0002 * \text{A[yr]}^2 + 0.00006982 * \end{aligned}$$

	$H[\text{cm}]^2$ $\text{PEF (L/Sec)} = 0.9267 + 0.06929 * A[\text{yrs}] - 0.001031 * A[\text{yrs}]^2 + 0.00018623 * H[\text{cm}]^2$
<b>MALE, Black (African-American)</b> 8–19 years, 48.0–76.4 in. (122–194 cm)	$\text{FVC (L)} = -0.4971 - 0.15497 * A[\text{yrs}] + 0.007701 * A[\text{yrs}]^2 + 0.00016643 * H[\text{cm}]^2$ $\text{FEV1 (L)} = -0.7048 - 0.05711 * A[\text{yrs}] + 0.004316 * A[\text{yrs}]^2 + 0.00013194 * H[\text{cm}]^2$ $\text{FEV1/FVC} = (89.239 - 0.1828 * A[\text{yrs}])/100$ $\text{FEV6 (L)} = -0.5525 - 0.14107 * A[\text{yrs}] + 0.007241 * A[\text{yrs}]^2 + 0.00016429 * H[\text{cm}]^2$ $\text{FEV1/FEV6} = (88.841 - 0.1305 * A[\text{yrs}])/100$ $\text{FEF25–75% (L/Sec)} = -1.1627 + 0.12314 * A[\text{yrs}] + 0.00010461 * H[\text{cm}]^2$ $\text{PEF (L/Sec)} = -0.2684 - 0.28016 * A[\text{yrs}] + 0.018202 * A[\text{yrs}]^2 + 0.00027333 * H[\text{cm}]^2$
<b>MALE, Black (African-American)</b> 20–80 years, 62.2–77.2 in. (158–196 cm)	$\text{FVC (L)} = -0.1517 - 0.01821 * A[\text{yrs}] + 0.00016643 * H[\text{cm}]^2$ $\text{FEV1 (L)} = 0.3411 - 0.02309 * A[\text{yrs}] + 0.00013194 * H[\text{cm}]^2$ $\text{FEV1/FVC} = (89.239 - 0.1828 * A[\text{yrs}])/100$ $\text{FEV6 (L)} = -0.0547 - 0.02114 * A[\text{yrs}] + 0.00016429 * H[\text{cm}]^2$ $\text{FEV1/FEV6} = (88.841 - 0.1305 * A[\text{yrs}])/100$ $\text{FEF25–75% (L/Sec)} = 2.1477 - 0.04238 * A[\text{yrs}] + 0.00010461 * H[\text{cm}]^2$ $\text{PEF (L/Sec)} = 2.2257 - 0.04082 * A[\text{yrs}] + 0.00027333 * H[\text{cm}]^2$
<b>FEMALE, Black (African-American)</b> 8–17 years, 46.5–72.4 in. (118–184 cm)	$\text{FVC (L)} = -0.6166 - 0.04687 * A[\text{yrs}] + 0.003602 * A[\text{yrs}]^2 + 0.00013606 * H[\text{cm}]^2$ $\text{FEV1 (L)} = -0.963 + 0.05799 * A[\text{yrs}] + 0.00010846 * H[\text{cm}]^2$ $\text{FEV1/FVC} = (91.655 - 0.2039 * A[\text{yrs}])/100$ $\text{FEV6 (L)} = -0.637 - 0.04243 * A[\text{yrs}] + 0.003508 * A[\text{yrs}]^2 + 0.00013497 * H[\text{cm}]^2$ $\text{FEV1/FEV6} = (91.229 - 0.1558 * A[\text{yrs}])/100$ $\text{FEF25–75% (L/Sec)} = -2.5379 + 0.43755 * A[\text{yrs}] - 0.012154 * A[\text{yrs}]^2 + 0.00008572 * H[\text{cm}]^2$ $\text{PEF (L/Sec)} = -1.2398 + 0.16375 * A[\text{yrs}] + 0.00019746 * H[\text{cm}]^2$
<b>FEMALE, Black (African-American)</b> 18–80 years, 53.5–70.9 in. (136–180 cm)	$\text{FVC (L)} = -0.3039 + 0.00536 * A[\text{yrs}] - 0.000265 * A[\text{yrs}]^2 + 0.00013606 * H[\text{cm}]^2$ $\text{FEV1 (L)} = 0.3433 - 0.01283 * A[\text{yrs}] - 0.000097 * A[\text{yrs}]^2 + 0.00010846 * H[\text{cm}]^2$ $\text{FEV1/FVC} = (91.655 - 0.2039 * A[\text{yrs}])/100$ $\text{FEV6 (L)} = -0.1981 + 0.00047 * A[\text{yrs}] - 0.00023 * A[\text{yrs}]^2 + 0.00013497 * H[\text{cm}]^2$ $\text{FEV1/FEV6} = (91.229 - 0.1558 * A[\text{yrs}])/100$ $\text{FEF25–75% (L/Sec)} = 2.0828 - 0.03793 * A[\text{yrs}] + 0.00008572 * H[\text{cm}]^2$ $\text{PEF (L/Sec)} = 1.3597 + 0.03458 * A[\text{yrs}] - 0.000847 * A[\text{yrs}]^2 + 0.00019746 * H[\text{cm}]^2$
<b>MALE, Hispanic (Mexican-American)</b> 8–19 years, 47.2–70.9 in. (120–180 cm)	$\text{FVC (L)} = -0.7571 - 0.0952 * A[\text{yrs}] + 0.006619 * A[\text{yrs}]^2 + 0.00017823 * H[\text{cm}]^2$ $\text{FEV1 (L)} = -0.8218 - 0.04248 * A[\text{yrs}] + 0.004291 * A[\text{yrs}]^2 + 0.00015104 * H[\text{cm}]^2$ $\text{FEV1/FVC} = (90.024 - 0.2186 * A[\text{yrs}])/100$ $\text{FEV6 (L)} = -0.6646 - 0.1127 * A[\text{yrs}] + 0.007306 * A[\text{yrs}]^2 + 0.0001784 * H[\text{cm}]^2$ $\text{FEV1/FEV6} = (89.388 - 0.1534 * A[\text{yrs}])/100$ $\text{FEF25–75% (L/Sec)} = -1.3592 + 0.10529 * A[\text{yrs}] + 0.00014473 * H[\text{cm}]^2$ $\text{PEF (L/Sec)} = -0.9537 - 0.19602 * A[\text{yrs}] + 0.014497 * A[\text{yrs}]^2 + 0.00030243 * H[\text{cm}]^2$
<b>MALE, Hispanic (Mexican-American)</b> 20–80 years, 61.4–75.6 in. (156–192 cm)	$\text{FVC (L)} = 0.2376 - 0.00891 * A[\text{yrs}] - 0.000182 * A[\text{yrs}]^2 + 0.00017823 * H[\text{cm}]^2$ $\text{FEV1 (L)} = 0.6306 - 0.02928 * A[\text{yrs}] + 0.00015104 * H[\text{cm}]^2$ $\text{FEV1/FVC} = (90.024 - 0.2186 * A[\text{yrs}])/100$ $\text{FEV6 (L)} = 0.5757 - 0.0286 * A[\text{yrs}] + 0.0001784 * H[\text{cm}]^2$  $\text{FEV1/FEV6} = (89.388 - 0.1534 * A[\text{yrs}])/100$

$$\begin{aligned} \text{FEF}_{25-75\%} \text{ (L/Sec)} &= 1.7503 - 0.05018 * A[\text{yrs}] + 0.00014473 * H[\text{cm}]^2 \\ \text{PEF (L/Sec)} &= 0.087 + 0.0658 * A[\text{yrs}] - 0.001195 * A[\text{yrs}]^2 + 0.00030243 * H[\text{cm}]^2 \end{aligned}$$

**FEMALE, Hispanic (Mexican-American)**

8–17 years, 44.9–67.7 in. (114–172 cm)

$$\begin{aligned} \text{FVC (L)} &= -1.2507 + 0.07501 * A[\text{yrs}] + 0.00014246 * H[\text{cm}]^2 \\ \text{FEV1 (L)} &= -0.9641 + 0.0649 * A[\text{yrs}] + 0.00012154 * H[\text{cm}]^2 \\ \text{FEV1/FVC} &= (92.360 - 0.2248 * A[\text{yrs}])/100 \\ \text{FEV6 (L)} &= -1.241 + 0.07625 * A[\text{yrs}] + 0.00014106 * H[\text{cm}]^2 \\ \text{FEV1/FEV6} &= (91.644 - 0.1670 * A[\text{yrs}])/100 \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= -2.1825 + 0.42451 * A[\text{yrs}] - 0.012415 * A[\text{yrs}]^2 + 0.0000961 * H[\text{cm}]^2 \\ \text{PEF (L/Sec)} &= -3.2549 + 0.47495 * A[\text{yrs}] - 0.013193 * A[\text{yrs}]^2 + 0.00022203 * H[\text{cm}]^2 \end{aligned}$$

**FEMALE, Hispanic (Mexican-American)**

18–80 years, 53.5–67.7 in. (136–172 cm)

$$\begin{aligned} \text{FVC (L)} &= 0.121 + 0.00307 * A[\text{yrs}] - 0.000237 * A[\text{yrs}]^2 + 0.00014246 * H[\text{cm}]^2 \\ \text{FEV1 (L)} &= 0.4529 - 0.01178 * A[\text{yrs}] - 0.000113 * A[\text{yrs}]^2 + 0.00012154 * H[\text{cm}]^2 \\ \text{FEV1/FVC} &= (92.36 - 0.2248 * A[\text{yrs}])/100 \\ \text{FEV6 (L)} &= 0.2033 + 0.0002 * A[\text{yrs}] - 0.000232 * A[\text{yrs}]^2 + 0.00014106 * H[\text{cm}]^2 \\ \text{FEV1/FEV6} &= (91.664 - 0.167 * A[\text{yrs}])/100 \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= 1.7456 - 0.01195 * A[\text{yrs}] - 0.000291 * A[\text{yrs}]^2 + 0.0000961 * H[\text{cm}]^2 \\ \text{PEF (L/Sec)} &= 0.2401 + 0.06174 * A[\text{yrs}] - 0.001023 * A[\text{yrs}]^2 + 0.00022203 * H[\text{cm}]^2 \end{aligned}$$

**ZAPLETAL (1987)**

Zapletal, A.: Lung Function in Children and Adolescents. Methods, Reference Values. Progress in Respiration Research Vol 22 (1987)

**MALE**

6–18 years,  
42.1–71.7 in. (107–182 cm)

$$\begin{aligned} \text{FVC (L)} &= 10^{(-2.9236 + 2.936 * \log(H[\text{cm}]))} / 1000 \\ \text{FEV1 (L)} &= 10^{(-2.8652 + 2.8729 * \log(H[\text{cm}]))} / 1000 \\ \text{FEV1/FVC} &= (90.6043 - 0.04104 * H[\text{cm}])/100 \\ \text{FEF}_{25\%} \text{ (L/Sec)} &= 10^{(-4.0164 + 2.1541 * \log(H[\text{cm}]))} \\ \text{FEF}_{50\%} \text{ (L/Sec)} &= 10^{(-4.2168 + 2.1771 * \log(H[\text{cm}]))} \\ \text{FEF}_{75\%} \text{ (L/Sec)} &= 10^{(-4.5808 + 2.2116 * \log(H[\text{cm}]))} \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= 10^{(-4.6651 + 2.3588 * \log(H[\text{cm}]))} \\ \text{PEFR (L/Sec)} &= 10^{(-4.3722 + 2.3422 * \log(H[\text{cm}]))} \\ \text{SVC (L)} &= 10^{(-2.5768 + 2.7799 * \log(H[\text{cm}]))} / 1000 \\ \text{MVV (L/Min)} &= 10^{(-1.9178 + 3.0388 * \log(H[\text{cm}]))} / 1000 \end{aligned}$$

**FEMALE**

6–18 years,  
42.1–71.7 in. (107–182 cm)

$$\begin{aligned} \text{FVC (L)} &= 10^{(-2.704 + 2.8181 * \log(H[\text{cm}]))} / 1000 \\ \text{FEV1 (L)} &= 10^{(-2.6056 + 2.7413 * \log(H[\text{cm}]))} / 1000 \\ \text{FEV1/FVC} &= (90.6043 - 0.04104 * H[\text{cm}])/100 \\ \text{FEF}_{25\%} \text{ (L/Sec)} &= 10^{(-4.0164 + 2.1541 * \log(H[\text{cm}]))} \\ \text{FEF}_{50\%} \text{ (L/Sec)} &= 10^{(-4.2168 + 2.1771 * \log(H[\text{cm}]))} \\ \text{FEF}_{75\%} \text{ (L/Sec)} &= 10^{(-4.5808 + 2.2116 * \log(H[\text{cm}]))} \\ \text{FEF}_{25-75\%} \text{ (L/Sec)} &= 10^{(-4.6651 + 2.3588 * \log(H[\text{cm}]))} \\ \text{PEFR (L/Sec)} &= 10^{(-4.3722 + 2.3422 * \log(H[\text{cm}]))} \\ \text{SVC (L)} &= 10^{(-2.297 + 2.6361 * \log(H[\text{cm}]))} / 1000 \\ \text{MVV (L/Min)} &= 10^{(-1.9178 + 3.0388 * \log(H[\text{cm}]))} / 1000 \end{aligned}$$

**QUANJER (1995)**

Quanjer, PhH, et. al.: Spirometric Values for White European Children and Adolescents: Polgar Revisited, Pediatric Pulmonology 1995, 19: 135–142.

**MALE**

6–18 years,

$$\begin{aligned} \text{LnFVC [I]} &= -1.2782 + [1.3731 + 0.0164 * A[\text{yrs}]] * H[\text{m}] \\ \text{LnFEV1 [I]} &= -1.2933 + [1.2669 + 0.0174 * A[\text{yrs}]] * H[\text{m}] \end{aligned}$$

43.3–80.7 in. (110–205 cm)

FEV1/FVC = 86.2

**FEMALE**

6–18 years,  
43.3–72.8 in. (110–185 cm)

$$\begin{aligned} \text{LnFVC [l]} &= -1.4507 + [1.4800 + 0.0127 * A[\text{yrs}]] * H[\text{m}] \\ \text{LnFEV1 [l]} &= -1.5974 + [1.5016 + 0.0119 * A[\text{yrs}]] * H[\text{m}] \\ \text{FEV1/FVC} &= 88.9 \end{aligned}$$

**WANG (1993)**

Wang, Xiaobin, et.al.; Pulmonary Function Between 6 and 18 Years of Age. Pediatric Pulmonology 1993; 15: 75–88.

**MALE, White**

6–18 years,  
43.3–74.8 in. (110–190 cm)

$$\begin{aligned} \text{LnFVC(L)} &= \alpha + \beta * \text{LnHt[m]} \\ \text{LnFEV1(L)} &= \alpha + \beta * \text{LnHt[m]} \\ \text{LnFEV1/FVC(L)} &= \alpha + \beta * \text{LnHt[m]} \\ \text{LnFEF25–75\%(L/s)} &= \alpha + \beta * \text{LnHt[m]} \end{aligned}$$

**MALE, Black**

6–18 years,  
47.2–74.8 in. (120–190 cm)

Refer to the Wang look-up tables for  $\alpha$  and  $\beta$ .

**FEMALE, White**

6–18 years,  
43.3–70.9 in. (110–180 cm)

**FEMALE, Black**

6–18 years,  
47.2–70.9 in. (120–180 cm)

**Wang look-up tables: MALE, White, 6–18 years**

Age [years]	FVC		FEV1		FEV1/FVC		FEF25–75%	
	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$
6	-0.024	2.470	-0.109	2.252	-0.078	-0.248	-	-
7	-0.018	2.489	-0.104	2.270	-0.086	-0.220	-	-
8	0.005	2.443	-0.089	2.257	-0.091	-0.199	0.264	1.505
9	0.017	2.426	-0.063	2.197	-0.086	-0.206	0.308	1.443
10	0.030	2.407	-0.057	2.212	-0.081	-0.209	0.290	1.557
11	0.009	2.468	-0.093	2.324	-0.101	-0.147	0.242	1.738
12	-0.061	2.649	-0.161	2.512	-0.101	-0.133	0.165	1.982
13	-0.175	2.924	-0.292	2.843	-0.116	-0.085	0.007	2.396
14	-0.219	3.060	-0.329	2.983	-0.106	-0.087	0.014	2.483
15	-0.079	2.859	-0.141	2.709	-0.060	-0.155	0.241	2.163
16	0.104	2.591	0.062	2.409	-0.045	-0.178	0.503	1.764
17	0.253	2.374	0.262	2.099	0.008	-0.272	0.762	1.368
18	0.296	2.316	0.251	2.129	-0.054	-0.170	0.678	1.528

**MALE, Black, 6–18 years**

Age [years]	FVC		FEV1		FEV1/FVC		FEF25–75%	
	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$
6	-0.088	1.961	-0.166	1.723	-0.091	-0.152	-	-
7	-0.040	2.040	-0.122	1.846	-0.091	-0.153	-	-
8	-0.094	2.323	-0.225	2.271	-0.118	-0.104	0.097	1.544
9	-0.074	2.308	-0.142	2.059	-0.079	-0.218	0.255	1.248
10	-0.110	2.417	-0.157	2.117	-0.047	-0.303	0.230	1.428
11	-0.138	2.453	-0.176	2.166	-0.048	-0.263	0.256	1.438
12	-0.224	2.710	-0.307	2.548	-0.084	-0.162	0.085	1.936
13	-0.342	2.975	-0.486	2.962	-0.141	-0.018	-0.121	2.476
14	-0.337	3.035	-0.472	3.010	-0.123	-0.050	-0.115	2.536
15	-0.226	2.889	-0.318	2.789	-0.070	-0.140	0.170	2.120
16	0.058	2.425	0.074	2.140	0.018	-0.289	0.663	1.299
17	0.148	2.310	0.053	2.223	-0.095	-0.087	0.505	1.618
18	0.152	2.341	0.130	2.121	-0.041	-0.190	0.859	1.053

**FEMALE, White, 6–18 years**

Age [years]	FVC		FEV1		FEV1/FVC		FEF25–75%	
	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$
6	-0.013	2.007	-0.109	1.949	-0.097	-0.055	-	-
7	0.062	2.385	-0.144	2.243	-0.084	-0.132	-	-
8	-0.055	2.381	-0.137	2.239	-0.079	-0.152	0.247	1.668
9	-0.039	2.351	-0.123	2.222	-0.084	-0.128	0.254	1.710
10	-0.068	2.458	-0.161	2.364	-0.092	-0.097	0.195	1.933
11	-0.120	2.617	-0.223	2.558	-0.102	-0.061	0.161	2.091
12	-0.174	2.776	-0.264	2.709	-0.090	-0.067	0.185	2.120
13	-0.061	2.576	-0.153	2.535	-0.093	-0.040	0.294	1.976
14	0.139	2.208	0.046	2.178	-0.096	-0.026	0.450	1.711
15	0.210	2.099	0.148	2.008	-0.062	-0.093	0.581	1.486
16	0.226	2.097	0.181	1.972	-0.048	-0.120	0.654	1.366
17	0.214	2.146	0.176	1.992	-0.038	-0.154	0.688	1.290
18	0.195	2.179	0.152	2.031	-0.069	-0.096	0.520	1.622

### FEMALE, Black, 6–18 years

Age [years]	FVC		FEV1		FEV1/FVC		FEF25–75%	
	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$	$\alpha$	$\beta$
6	-0.172	2.117	-0.288	2.182	-0.109	0.059	-	-
7	-0.135	2.132	-0.250	2.158	-0.104	-0.030	-	-
8	-0.176	2.362	-0.276	2.295	-0.103	-0.066	-0.283	2.990
9	-0.200	2.452	-0.294	2.330	-0.097	-0.104	0.025	2.062
10	-0.230	2.571	-0.344	2.507	-0.120	-0.043	0.051	2.028
11	-0.204	2.526	-0.308	2.460	-0.089	-0.105	0.078	2.006
12	-0.107	2.342	-0.219	2.312	-0.115	-0.021	0.225	1.804
13	-0.042	2.294	-0.117	2.196	-0.051	-0.148	0.418	1.504
14	0.105	2.021	0.041	1.920	-0.063	-0.103	0.574	1.257
15	0.253	1.787	0.203	1.662	-0.043	-0.139	0.599	1.281
16	0.111	2.098	0.129	1.824	-0.022	-0.188	0.653	1.175
17	0.205	1.930	0.273	1.547	0.048	-0.342	0.713	1.067
18	-0.042	2.423	-0.084	2.259	-0.197	0.145	-0.209	2.896

### GLI (2012)

Multi-ethnic reference values for spirometry for the 3-95 year age range: the global lung function 2012 equations.

*Quanjer, P.H., Stanojevic, S., Cole, T.J., Baur, X., Hall, G.L., Enright, P.L., Hankinson, J.L., Ip, M.S., Zheng, J., Stocks, J. on behalf of the ERS Global Lung Function Initiative. (2012), European Respiratory Journal, 2012; 40: 1324–1343.*

The Global Lung Function Initiative (GLI) equations are the first global multi-ethnic reference equations for spirometry that span all-ages. These are the result of unprecedented, unselfish and professional international cooperation endorsed by five international societies. Briefly, data from 74,187 healthy non-smokers (57.1% females) aged 3-95 years were used to derive multi-ethnic reference equations using modern statistical methods, including development of age dependent lower limits of normal.

\*Note: When Asian is selected under the Race demographic dropdown in Office Medic, the reference values will be based on the SE Asian GLI dataset, as this set provided a larger patient population.

For more information on the Global Lung Initiative and the specific reference equations, please visit <http://www.ers-education.org/guidelines/global-lung-function-initiative.aspx>

### GLI Global (2022)

Bowerman, Cole, et al. "A Race-neutral Approach to the Interpretation of Lung Function Measurements." *American Journal of Respiratory and Critical Care Medicine*, vol. 207, no. 6, 15 Mar. 2023, pp. 768–774, <https://doi.org/10.1164/rccm.202205-0963oc>.

*Bhakta, Nirav R., et al. "Race and ethnicity in pulmonary function test interpretation: An official American Thoracic Society statement." American Journal of Respiratory and Critical Care Medicine, vol. 207, no. 8, 15 Apr. 2023, pp. 978–995, <https://doi.org/10.1164/rccm.202302-0310st>.*

As of 2023, the American Thoracic Society (ATS) recommends the use of race-neutral reference equations for pulmonary function testing. The Global Lung Function Initiative has released global race-neutral reference equations for spirometry interpretation in patients between the ages of 3 to 95. The original GLI dataset was recalculated with inverse-probability weights to create the race-neutral GLI Global (2022) reference equations. For more information on the Global Lung Function Initiative and their race neutral reference equation, please visit the cited sources.

## Lung Age Calculation

Lung age is calculated for patients 20-84 years old. \*Lung age is equal to the predicted FEV1 that matches the patient's actual FEV1.

### For example:

Predicted equation: Crapo  
Patient demographics: Height: 5ft 10in  
Age: 46 years  
Gender: Male  
Race: Caucasian  
Actual FEV1: 4.49L  
Predicted FEV1: 4.05L  
Patient's Lung Age: 28 years

Based on Crapo's predicted equation, the patient's actual FEV1 (4.49L) is equal to the predicted FEV1 of a 28 year old. Therefore, the patient's lung age is 28 years old.

---

Note: Lung age may differ based on the predicted equation selected.

---

\* Morris JF, Temple W.; Spirometric "lung age" estimation for motivating smoking cessation. Prev Med. 1985 Sep; 14(5):655-62.

---

Note: "Lung age not available" dialog box may appear when certain predictors and ages are selected because they are not supported for this function.


---

## Spirometry Interpretation

Note: A disclaimer is provided on all spirometry reports: "All test results should be evaluated by a qualified physician."

### Enright (1997)


Office Spirometry: A Practical Guide to the Selection and Use of Spirometers by Paul L. Enright, M.D.  
Robert E. Hyatt M.D. 1987

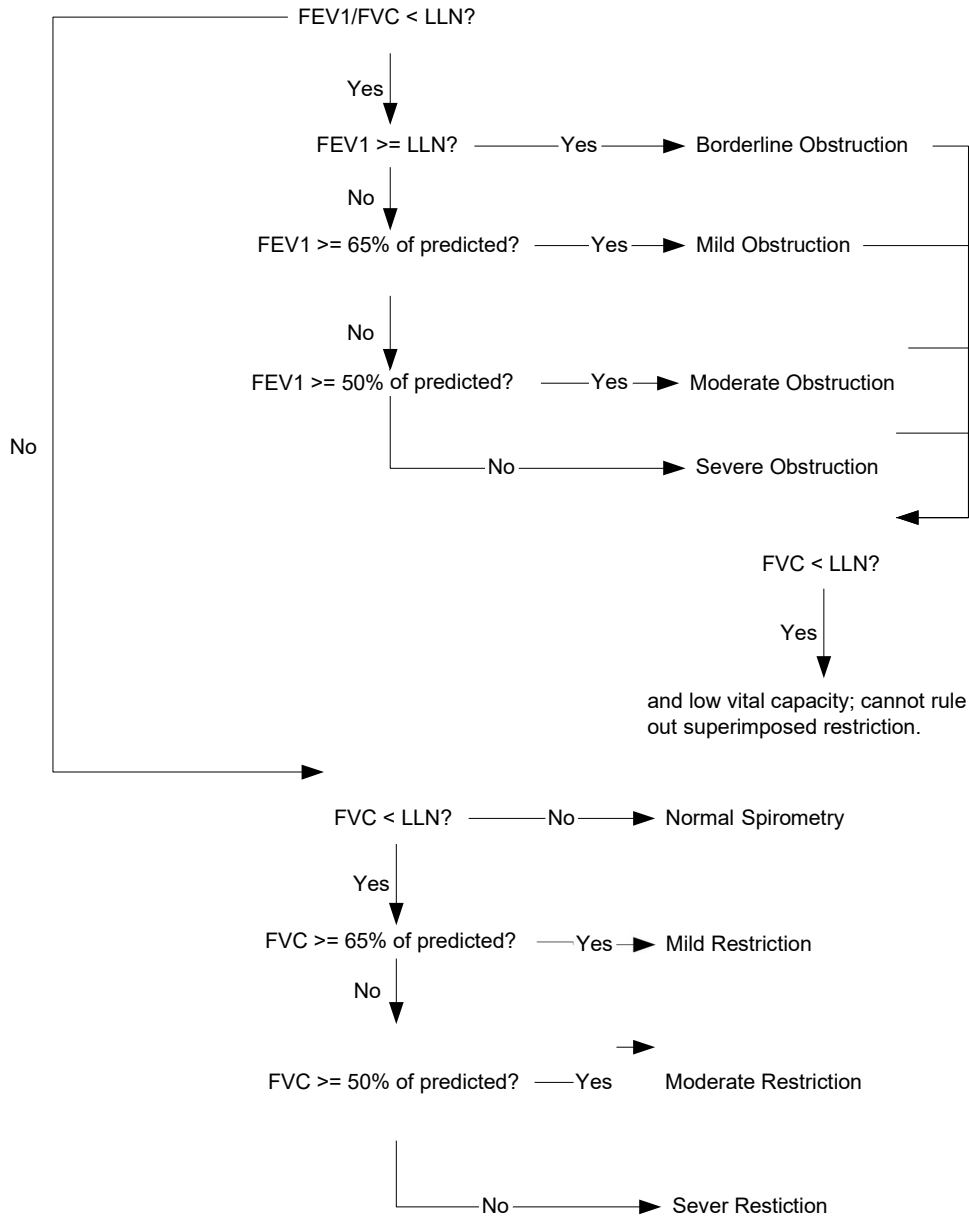

### BTS-NICE (2004-05)

The British Thoracic Society (BTS) COPD Consortium: Spirometry in Practice: A Practical Guide to Using Spirometry in Primary Care. Second Edition. April 2005.


National Institute for Clinical Excellence (NICE): Chronic obstructive pulmonary disease: Management of chronic obstructive pulmonary disease in adults in primary and secondary care. Clinical Guideline 12. February 2004. Developed by the National Collaborating Centre for Chronic Conditions.

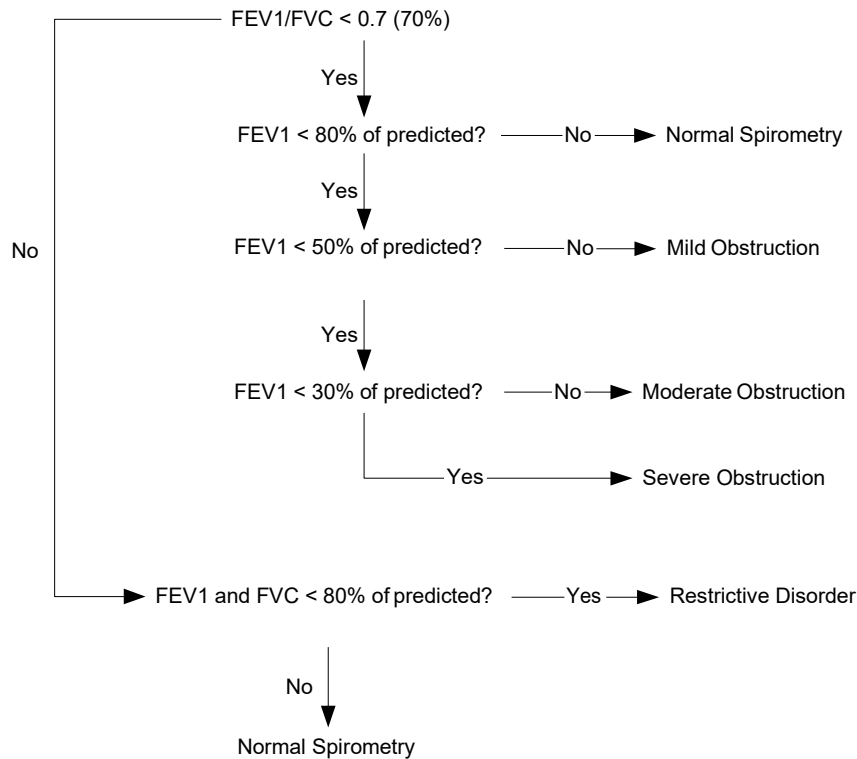

## NLHEP (2000)


Ferguson GT, et. al.: Office Spirometry for Lung Health Assessment in Adults. A Consensus Statement from the National Lung Health Education Program (NLHEP). Chest April 2000; Volume 117: 1146–1161.

FVC is used in place of FEV6 when the predicted study does not provide an FEV6 predicted value/LLN.

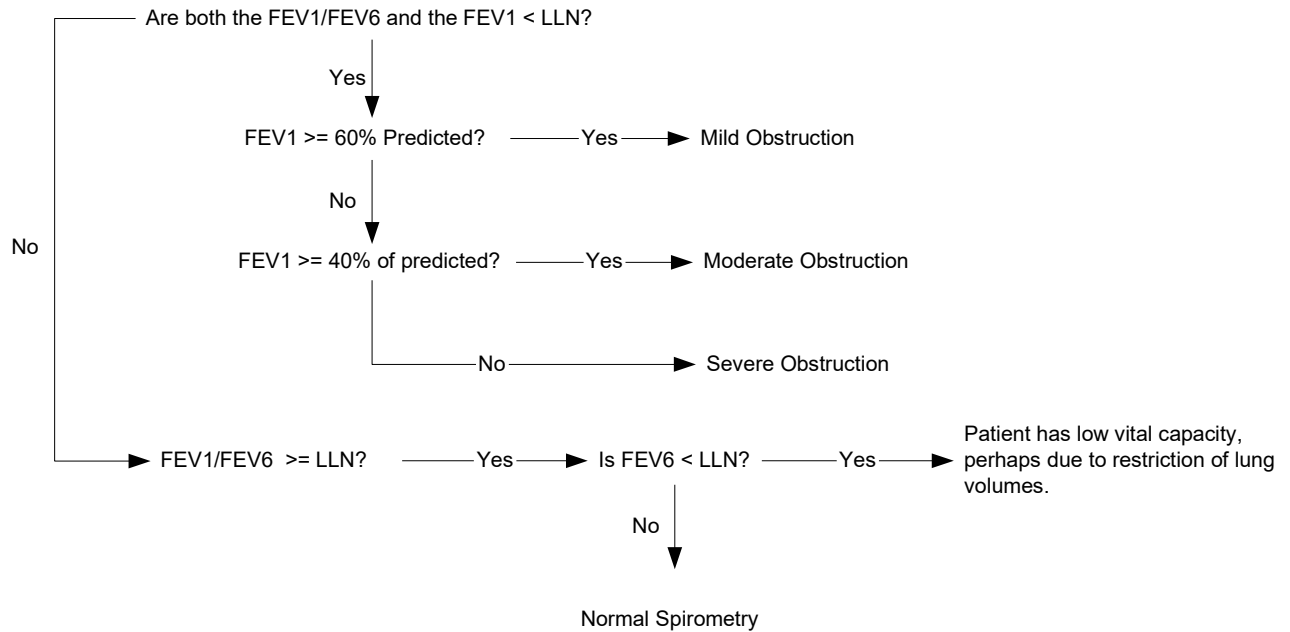

**ATS/ERS (2005)**

ATS/ERS Task Force: Interpretive strategies for lung function tests. Standardisation of spirometry. Eur. Respir. J., Nov 2005; 26: 948-968.

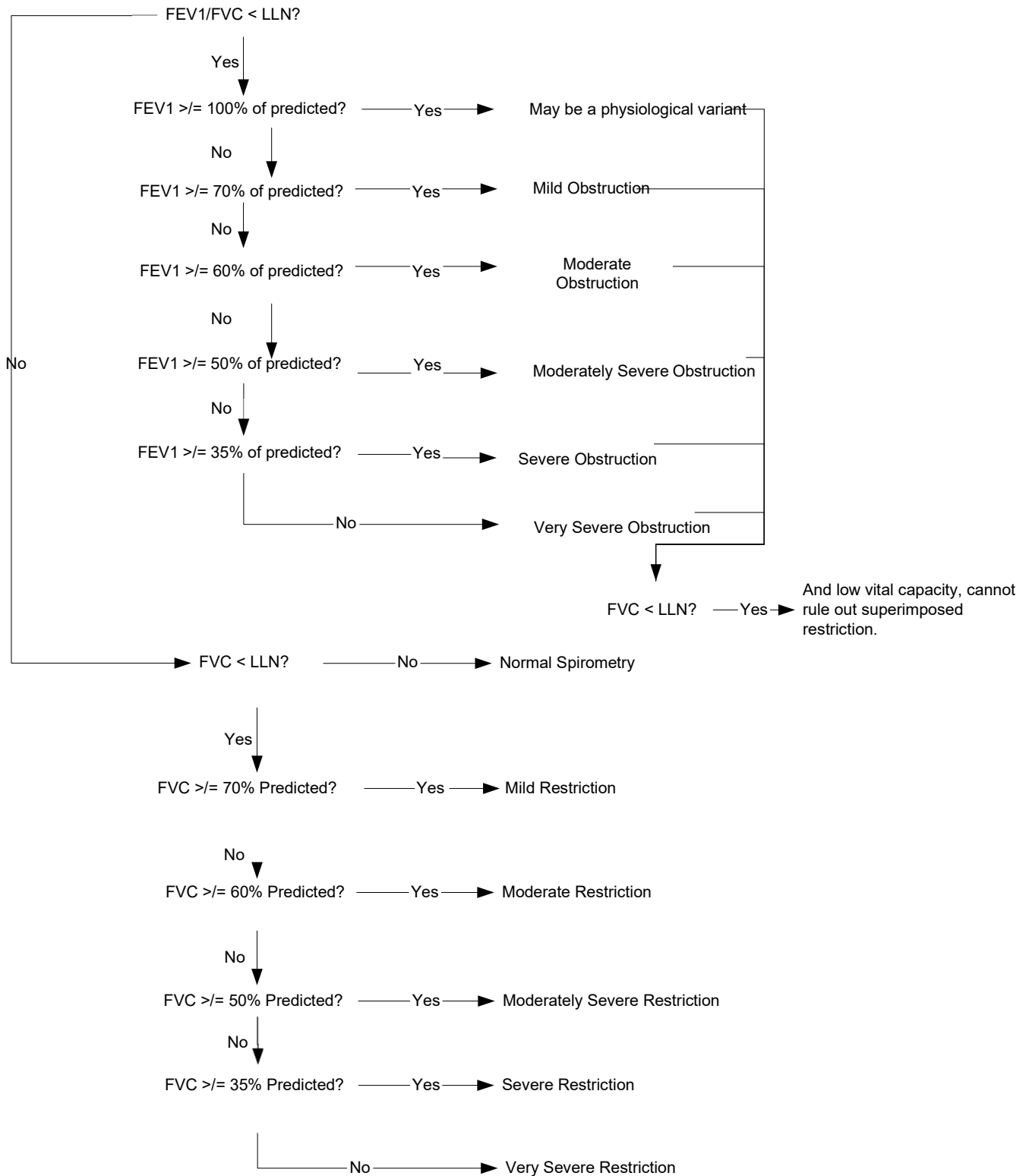

## ATS/ERS (2019)

Graham, Brian L., et al. "Standardization of spirometry 2019 update. an official American Thoracic Society and European Respiratory Society Technical Statement." *American Journal of Respiratory and Critical Care Medicine*, vol. 200, no. 8, 15 Oct. 2019, <https://doi.org/10.1164/rccm.201908-1590st>.

- Each test maneuver performed will yield a quality grade for the overall session for both FEV1 and FVC. An example of a session quality grading is shown below.


Pre Test Session Grade: FEV1 - B, FVC - B (FVC Var=0.02L (0.61%); FEV1 Var=0.02L (0.68%))  
 Post Test Session Grade: FEV1 - E, FVC - D (FVC Var=0.23L (7.22%); FEV1 Var=0.27L (9.83%))

Each session yields a grade in accordance with the table below.

Grade	Number of Tests	Repeatability for Patients Aged >6 yr	Repeatability for Patients Aged ≤ 6 yr
<b>A</b>	≥3 acceptable	Within 0.150L	Within 0.100L*
<b>B</b>	2 acceptable	Within 0.150L	Within 0.100L*
<b>C</b>	≥2 acceptable	Within 0.200L	Within 0.150L*
<b>D</b>	≥2 acceptable	Within 0.250L	Within 0.200L*
<b>E</b>	≥2 acceptable OR 1 acceptable	>0.250L OR N/A	>0.200L* OR N/A
<b>U</b>	0 acceptable AND ≥1 usable	N/A	N/A
<b>F</b>	0 acceptable and 0 usable	N/A	N/A

*\*Or 10% of the highest value, whichever is greater; applies for age 6 years or younger only.*

- Z-Score and Z-Score Pictogram  
 When using either "GLI 2012" or "GLI-G (Race-Neutral)" as the predictor, several spirometry parameters will have affiliated z-scores. In spirometry, z-scores are utilized to measure how far a patient deviates from the predicted normal value, adjusted for gender, age, height, and ethnicity (if not using GLI-G).

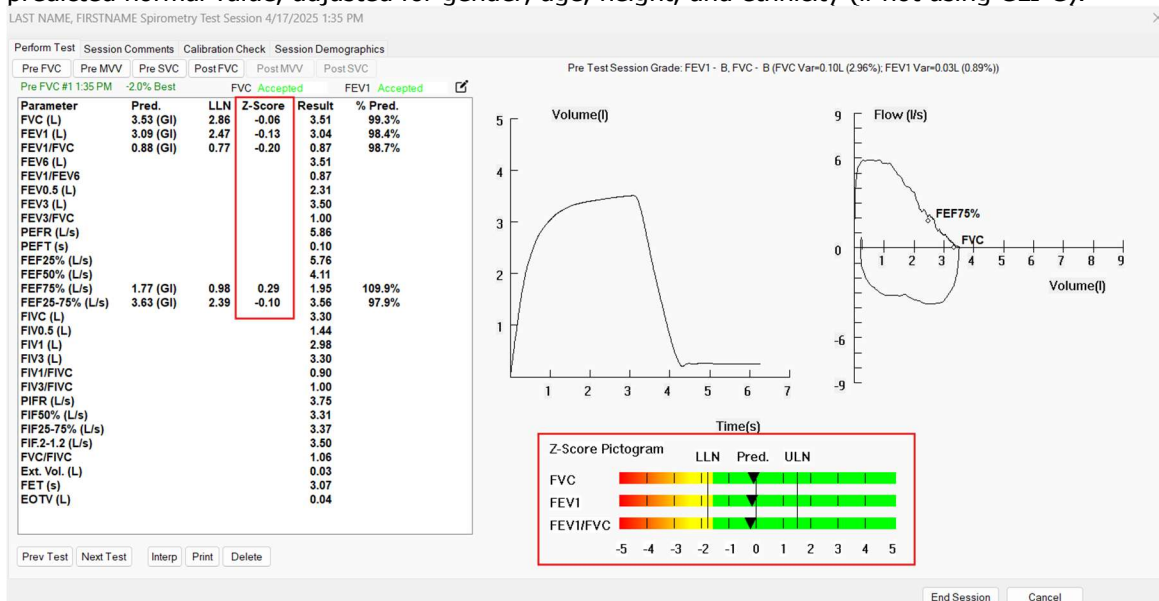

The Lower Limit of Normal (LLN) in Spirometry is defined as 1.645 Standard Deviations below the predicted value. The level of lung function impairment can be generalized using the following table below:

Mild	-1.65 to -2.5
Moderate	-2.51 to -4.0
Severe	Greater than -4.1


- FVC and FEV1 Repeatability

FVC repeatability is achieved when the difference between the two best (largest) FVC values is <0.150 L for patients older than 6 years of age and <0.100 L or 10% of the largest FVC, whichever is greater, for those aged 6 years or younger. FEV1 repeatability is achieved when the difference between the two best (largest) FEV1 values is <0.150 L for patients older than 6 years of age and <0.100 L or 10% of the largest FVC, whichever is greater, for those aged 6 years or younger. Repeatability must be met in three maneuvers, otherwise additional trials (up to eight total maneuvers) should be attempted. When repeatability is met, the below pop-up message will prompt.



If FEV1 drops below 80% of the start value, the test procedure should be terminated for patient safety. All failed tests will be kept, not deleted.

- Each test maneuver will be evaluated for acceptability and usability. These criteria ensure that maximal patient effort is being achieved and that acceptable spirometry measurements are obtained. While it is recommended that the patient strives for acceptable measurements, in some cases, the patient’s maximal effort will not meet all criteria. Although their measurements may not be technically acceptable, they can be clinically useful. The overall evaluation and the criteria can be found on the “Perform Test” window and all parameters can be manually adjusted by the user, simply click “No (Edit)” and select the respective toggles. The user can edit individual criteria within the system-validated and technician-verified criteria, as well as whether the maneuver’s FVC/FEV1 is acceptable, usable or not usable.

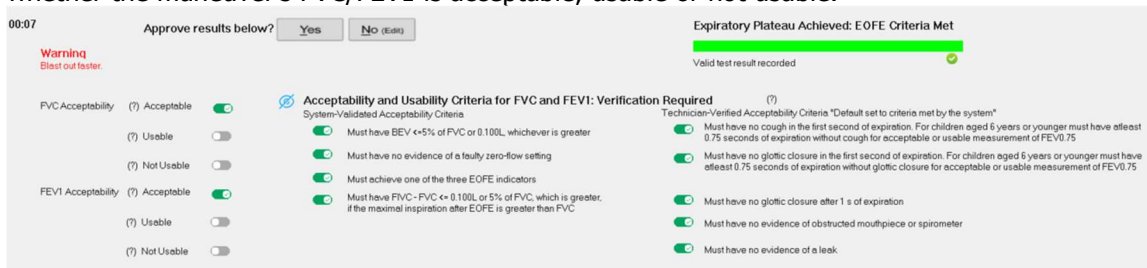


The table below explains what parameters must be met for FVC or FEV1 to be marked as usable, acceptable, or not usable:

Acceptability and Usability Criterion	Required for Acceptability?		Required for Usability?	
	FEV <sub>1</sub>	FVC	FEV <sub>1</sub>	FVC
Must have BEV ≤ 5% of FVC or 0.100L, whichever is greater	✓	✓	✓	✓

Must have no evidence of a faulty zero-flow setting	✓	✓	✓	✓
Must have no cough in the first second of expiration	✓	✗	✓	✗
Must have no glottic closure in the first second of expiration	✓	✓	✓	✓
Must have no glottic closure after 1s of expiration	✗	✓	✗	✗
Must achieve one of these three EOFE indicators: Expiratory plateau ( $\leq 0.0025L$ in the last 1s of expiration) Expiratory time $\geq 15s$ FVC is within the repeatability tolerance of or is greater than the largest prior observed FVC	✗	✓	✗	✗
Must have no evidence of obstructed mouthpiece or spirometer	✓	✓	✗	✗
Must have no evidence of a leak	✓	✓	✗	✗
If the maximal inspiration after EOFE is greater than FVC, then FIVC - FVC must be $< 0.100 L$ or 5% of FVC, whichever is greater	✓	✓	✗	✗


- It is now recommended to add a maximal inhalation after the maximal exhalation. Additionally, one of three criteria must be met for an acceptable EOFE. The first test recorded will always prompt the following message: "First Test Recorded: Provisionally Meets EOFE Criteria"

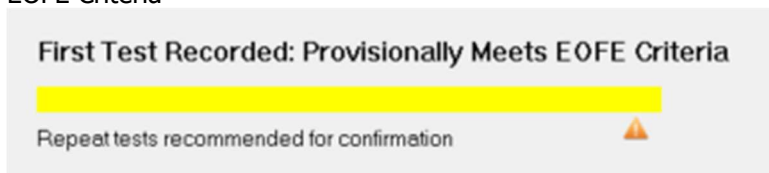


Any subsequent tests will prompt the following four messages. See below flow chart for the decision matrix.




**Repeatability Achieved: EOFE Acceptability Met**


Result validated against prior observed values 

**EOFE Acceptability Not Met: Data Usable for Review**

Consider retesting if clinically necessary 

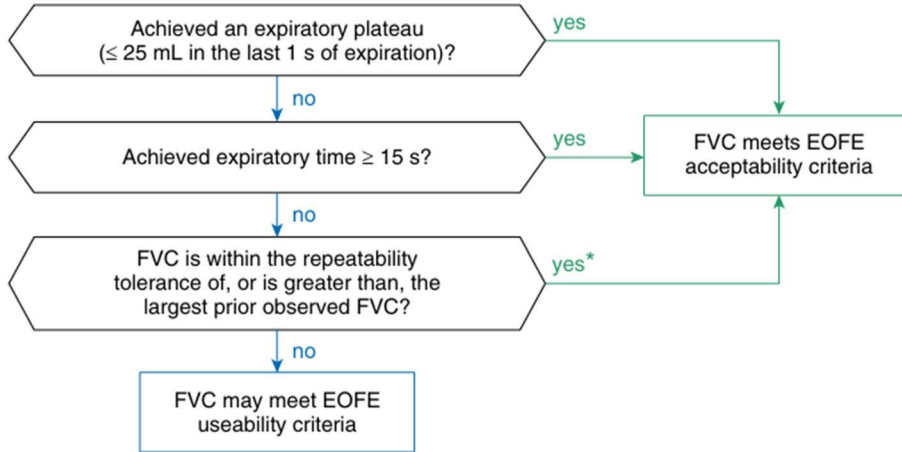

- The addition of a flow vs. volume graph

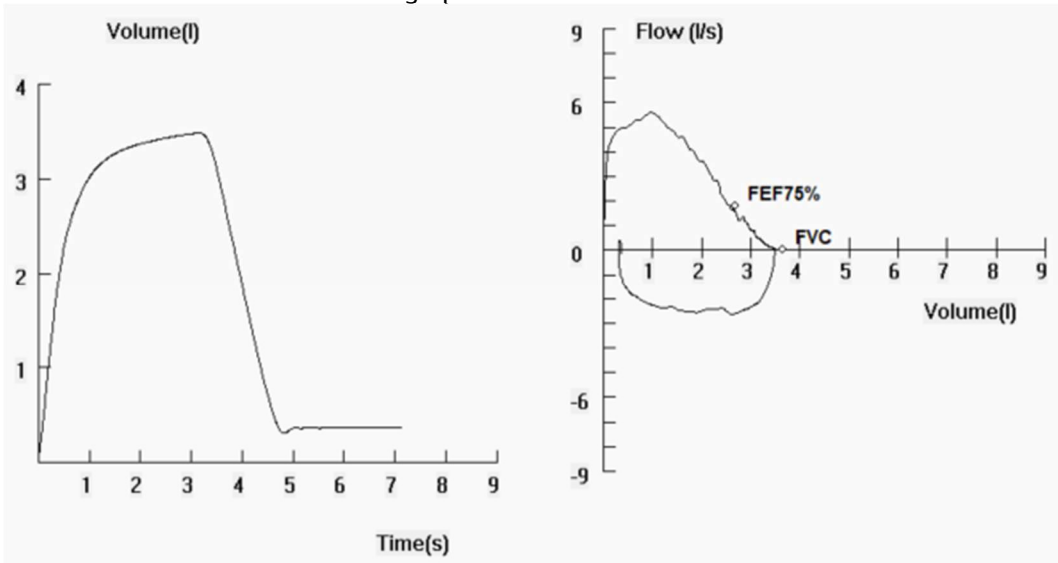


- The "Prepare for [insert spirometry test]" window has been updated with a bronchodilator withholding chart as seen in the below screenshot.

Prepare for Pre FVC Test

Mouthpiece #:

Physician/Tech.:

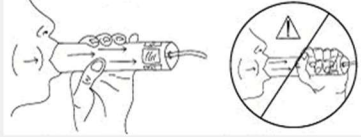

Ensure the patient does not cover or obstruct the mouthpiece fabric during testing.

Environmental Settings

22 °C


0 m

Bronchodilator	Withholding Time
SABA (e.g., albuterol or salbutamol)	4-6 h
SAMA (e.g., ipratropium bromide)	12 h
LABA (e.g., formoterol or salmeterol)	24 h
Ultra-LABA (e.g., indacaterol, vilanterol, or olodaterol)	36 h
LAMA (e.g., tiotropium, umeclidinium, aclidinium, or glycopyrronium)	36-48 h
Other	Allow to perform pre spirometry test

Definition of abbreviations: LABA = long-acting  $\beta$ 2-agonist; LAMA = long-acting muscarinic antagonist; SABA = short-acting  $\beta$ 2-agonist; SAMA = short-acting muscarinic antagonist. Note: withholding times for post-bronchodilator testing are shorter than those for methacholine challenge testing(147) because the bronchoprotection provided by these agents lasts longer than their bronchodilation effects. in the case of dual bronchodilators, the withholding time for the longer-acting bronchodilator is used.

Select OK or press enter to begin test

- Bronchodilator responsiveness equation has been added to post-test sessions.

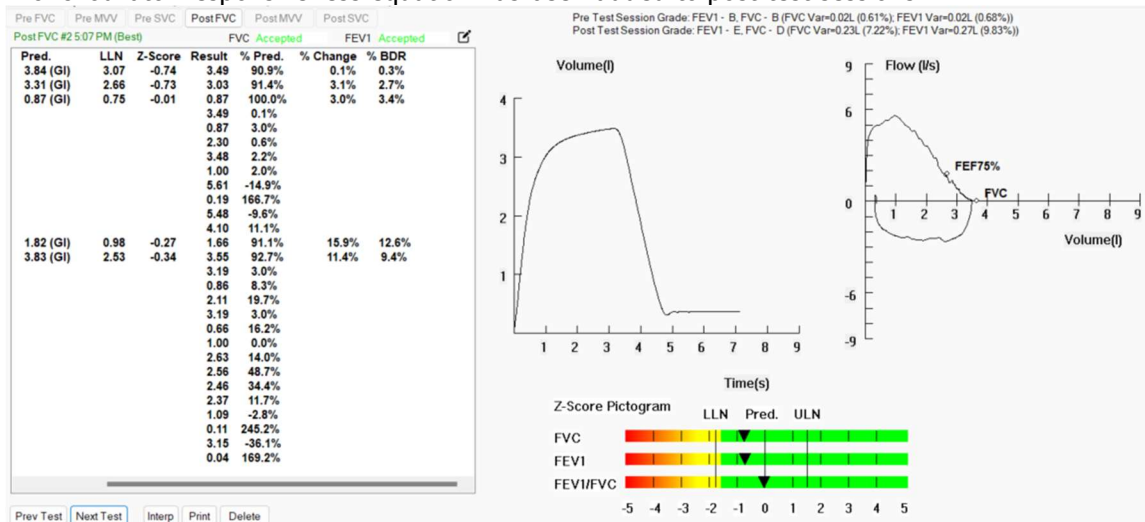

# Electrocardiography

## ECG Cautions, Warnings, and Other Information

### Warnings



- The computerized interpretation is only valid when used in conjunction with clinical findings. All computer generated tracings and interpretations must be confirmed by a qualified physician. Test interpretations are intended for the physician's use only. All ECG numerical and graphical data should be evaluated with respect to the patient's clinical and historical picture.
- No specific filter settings were used to pass the distortion test.
- Isoelectric segments within the VectraCor are excluded from the VectraCor waves.
- Use printout for final diagnosis.
- The ECG Device is not intended for use in a sterile environment.
- Do not use for direct cardiac application.
- The ECG device is reusable.
- The ECG is protected against malfunction caused by electrosurgery.
- The ECG does not incorporate means to protect the patient against burns when used during electrosurgery
- To prevent burns from ECG electrodes during monopolar electrosurgery:
  - Make sure that the patient return pad or large-surface neutral electrode is properly applied.
  - Position ECG electrodes away from the electrosurgery site and current pathway through the body.
- Loss of ECG signal may occur during high frequency electrosurgery. Signal should re-establish within 10 seconds of removal of the high-frequency source. No other conditions of use must be met to operate the device in a high – frequency electrosurgery environment. Indications for use are the same as stated above.
- If signal quality is lost due to electromagnetic disturbances, the ECG signal may be affected which can lead to incorrect diagnosis.
- The use of cardiac pacemakers or other electrical stimulators may lead to incorrect interpretations and diagnosis.
- Do not attempt to insert the ECG device (including patient cables) into an electrical outlet.



- Avoid patient movement to reduce artifact. The ECG Device is for acquiring resting ECGs only. The device should not be used for stress testing.
- Though false positive errors will intentionally outnumber false negative errors, both will occur, thus the necessity for over reading by a qualified physician of any computer-interpreted ECG. The computer interpretation does not produce a definitive diagnosis.
- Ensure electrodes are connected only to patient.
- Conductive parts of electrodes and connectors, including neutral electrode, should not contact other conductive parts including earth.
- Select a three lead view during defibrillation to ensure signals are clearly separated following electrode polarization.
- Defibrillator warnings:
  - Do not touch the patient during defibrillation.

- Do not touch the defibrillator’s paddle-electrode surface when discharging the defibrillator.
- Keep defibrillation electrodes well clear of other electrodes or metal parts in contact with the patient.
- Do not touch the patient, bed, or any conductive material in contact with the patient during defibrillation.
- When defibrillation discharge is applied, the ECG may be overloaded with voltage, but will recover normal function within 5s.
- Defibrillation use requires use of manufacturer specified electrodes

**Cautions**



- For diagnostic ECG according to the requirements of the AAMI EC11:1991 standard, use factory default settings. ECG diagnosis should be based on a printed 3x4 report with software filters off, and using a 1:1 scale 300dpi printer.
- The Universal SmartECG is designed for use with electrodes that comply with AAMI EC12:2000.
- Reseal electrode pouch after opening to prevent dehydrating.
- Suggested maximum electrode duration is 8 hours.
- Leakage current of device is increased when several items of the device are interconnected. However, leakage current is tested to be within acceptable levels.
- Do not clean the case with alcohol.
- Do not saturate or immerse the case with liquid during cleaning.
- Do not sterilize ECG device.

**ECG Indications for Use: Receipt, Storage, Viewing, Printing and Interpretive Analysis of 12 channel simultaneous ECGs**

- The system can be used within electro-surgical environment.

**Application Specification:**

Medical Indication:	Diagnosis of cardiovascular conditions/diseases
User Profile:	Trained professional
Patient Population:	Adult Male/Female (18+)
Environment of Use:	Hospital and Clinical Use and HF Surgical environments
Tissue Type of Device Interaction:	Patient Skin
Conditions of Use:	Single patient, non-sterile, single-use electrodes, non-invasive, resting ECGs, continuous use, portable
Operating Principle:	[Physical Method] Electrodes are attached to the patient to acquire the ECG (electrocardiogram) with software application. [Mechanism] The attached electrodes records the ECG signal. The software application starts and stops the recording of the ECG. Then, the software can display or print a hard copy of the ECG report.

## ECG Getting Started

Connecting the 12 Channel ECG device to your PC:



- Connect the USB port of the ECG device to the USB port on your PC.
- The LED will light up to indicate when powered.


---


Note: The LED on the ECG will not only indicate when the device is powered (solid green) and ready for acquisition, but also when it is acquiring data (flashing green). If the LED is turned off, the device is inoperable.

---

### Performing an ECG Test: Quick Start Guide

1. Connect the ECG device to the PC.
2. Select a patient from the Patient Directory.
  - If patient is not in the directory, create the patient.
3. Prep Patient & Connect the ECG device
  - Shave electrode sites if necessary. Thoroughly clean the area and let dry.
  - Prep skin by briskly rubbing with gauze, being careful not to break or damage the skin.
  - Remove electrodes from backing.
  - Apply each electrode, adhesive side down to desired site.
  - For positive electrode contact, start from outer edge and run your finger around the electrode several times, working toward the center.
  - Connect the lead wires to the patient ensuring correct lead placement. Excess movement can cause artifact. Patient should be stable.

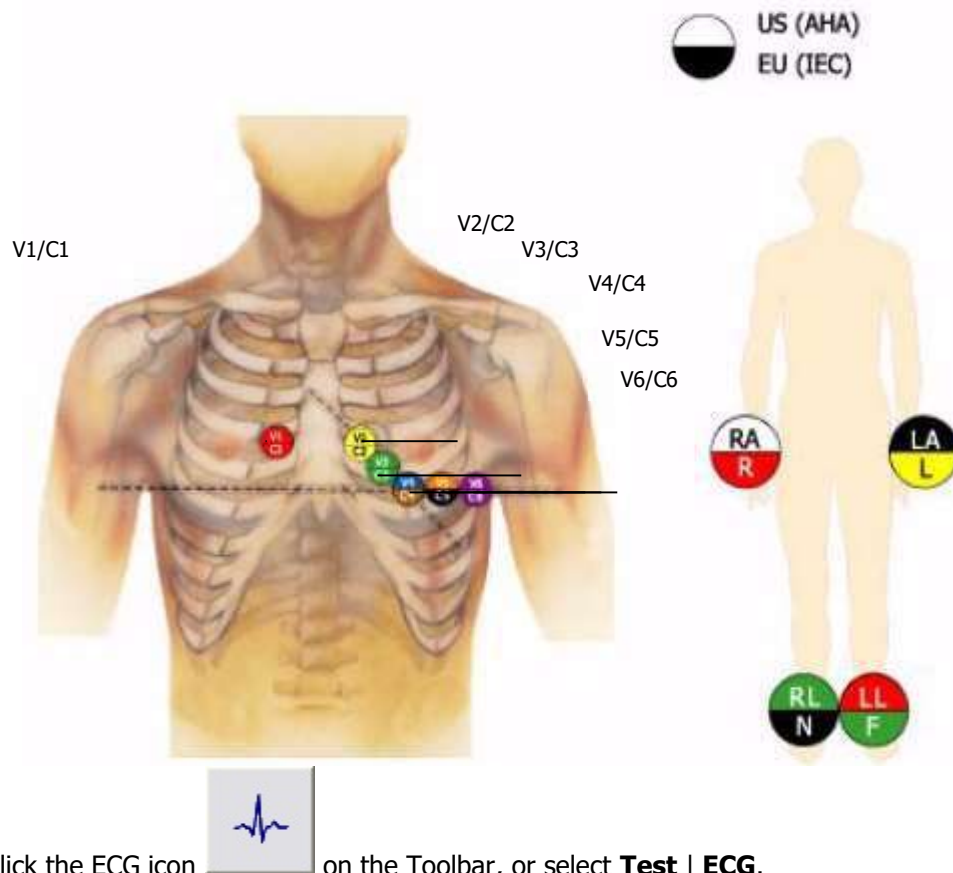


4. Click the ECG icon  on the Toolbar, or select **Test | ECG**.  
The acquisition window will appear on the screen with the patient's real-time ECG displayed.

**Warning:** Avoid patient movement to reduce artifact. The ECG Device is for acquiring resting ECGs only. The device should not be used for stress testing.

5. After the ECG data is captured by Office Medic, select **Save and Review**.  
6. After review, select **Print** or **Print to File**.

## About the Acquisition Window

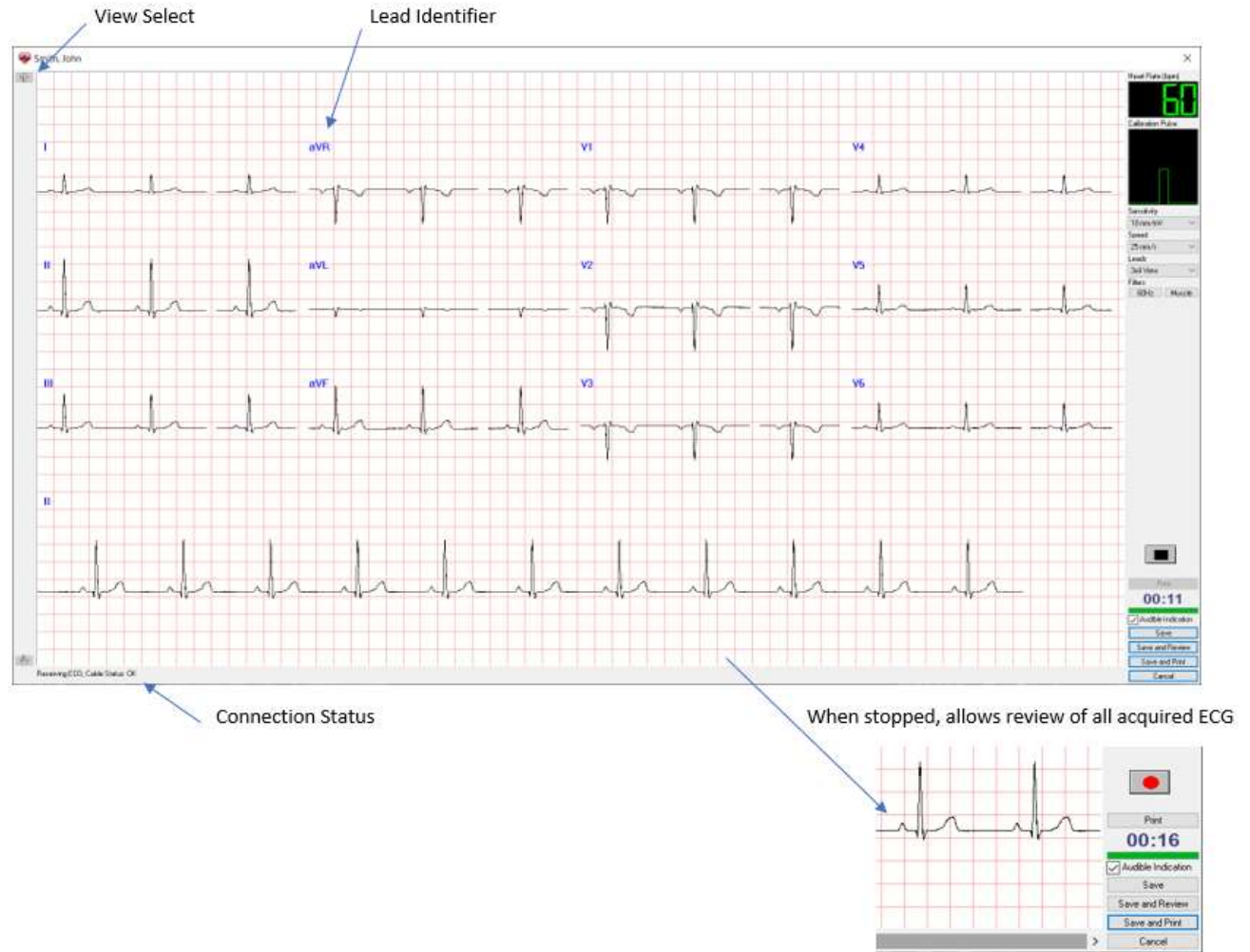

### View Select

Scrolls through the different leads when you view the 3 or 6 lead sets.


### Lead Identifiers

Identifies each of the 12 leads. If a lead is disconnected (leads off) then a red circle with a diagonal line is placed over the lead identifier.










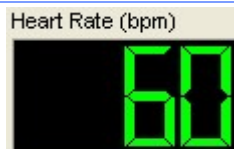





### Connection Status

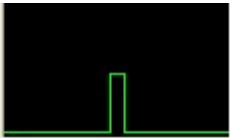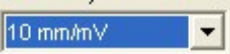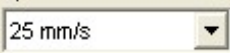Displays the status of the connected ECG cable.

### Heart Rate (bpm)




Displays the active Heart Rate of the patient.
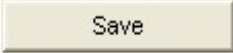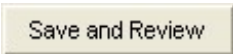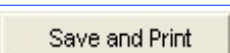
<b>Calibration Pulse</b>		Provides a visual indication of the combined sensitivity (1mV vertical height) and speed (100ms horizontal width).
<b>Sensitivity</b>		Changes the number of millimeters that represent one millivolt. The available options are <b>5mm/mV</b> , <b>10mm/mV</b> , and <b>20mm/mV</b> .
<b>Speed</b>		Changes the number of millimeters that are passed in one second. The available options are <b>12.5mm/s</b> , <b>25mm/s</b> and <b>50mm/s</b> .
<b>Leads</b>		View 3, 6 or 12 leads, or 3x4 view. The ability to select between the Limb leads and the Chest leads is also available when viewing 3 or 6 leads at a time. A Custom Lead group can be defined in the <a href="#">ECG Options</a> .
<b>Power Filter</b>		Turns the Main filter on and off. Note: the default frequency of the Main filter is set in the <a href="#">ECG Options</a> .
<b>Muscle Filter</b>		Turns the Muscle filter on and off.
<b>Stop</b>		Stops the real time recording to view the previous 15 minutes of ECG. The user can select the desired 10 seconds of ECG and select <b>Save</b> to save and exit the test.
<b>Record</b>		Resumes recording data. Once selected, all paused data will no longer be available.
<b>Print</b>		Allows you to print all or a selected portion of the stopped ECG. This printed report is not intended for diagnostic use, or as a patient record - for that purpose print from the review window, or use "Save and Print."
<b>Elapsed Time</b>		Minutes and seconds of current ECG acquisition.
<b>Status Bar</b>		Represents whether 10 seconds of valid ECG data has been received. When the status bar is full, the <b>Save</b> button is activated and data can then be saved.
<b>Audible Indication</b>	<input checked="" type="checkbox"/> Audible Indication	Audibly indicates a "leads off" status or QRS detection as selected in "options."
<b>Save</b>		Saves the test and closes the window.
<b>Save and Review</b>		Saves the test and launches a review of the results.
<b>Save and Print</b>		Saves the test and automatically prints a report or PDF (optional).
<b>Cancel</b>		Closes the test without saving.

Note: Users can acquire ECG either from Office Medic or VectraplexECG. Please see VectraplexECG user manual for additional information. To acquire ECG with VectraplexECG, set the default ECG program to VectraplexECG by going to **Options > General**. Please call VectraCor or visit [www.VectraCor.com](http://www.VectraCor.com) for more information about VectraplexECG.


## ECG Options

Select **Options** | **ECG**.

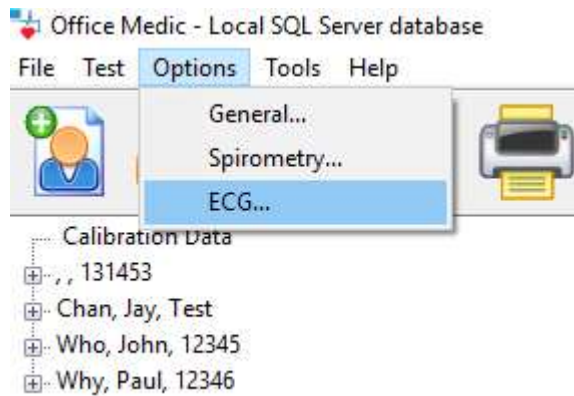

### General Tab

Select **General** to set or change general ECG Options.

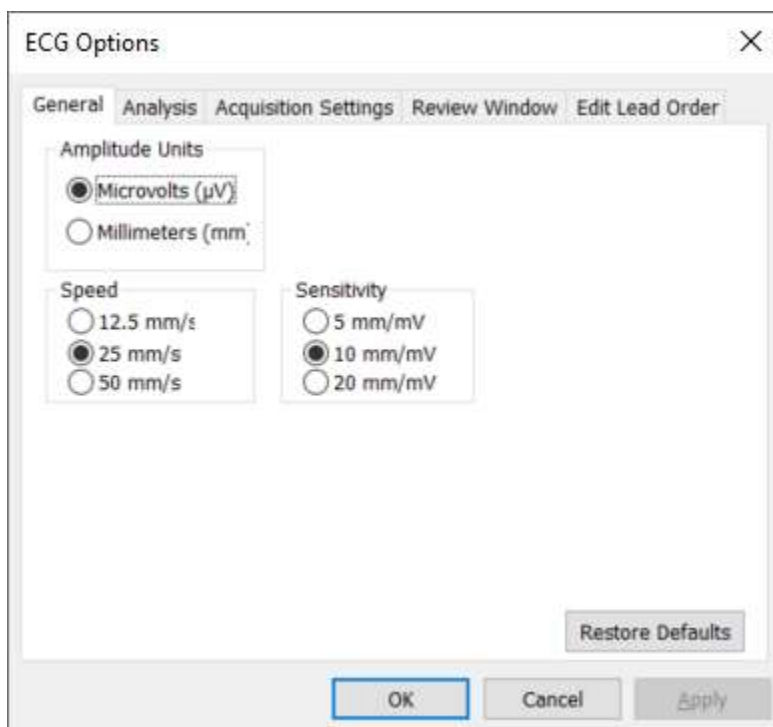

## Analysis Tab

Select **Analysis** to set or change the ECG analysis options.

The screenshot shows the 'ECG Options' dialog box with the 'Analysis' tab selected. The dialog has a title bar with a close button (X) and a tabbed interface with five tabs: 'General', 'Analysis', 'Acquisition Settings', 'Review Window', and 'Edit Lead Order'. The 'Analysis' tab is active, showing several checked options: 'Automatically include narrative interpretation', 'Suppress incomplete patient details warning', 'Include interpretation codes within narrative text', and 'Allow user to re-analyze'. Below these is a 'QTc Calculation Method' section with two radio buttons: 'Bazett' (selected) and 'Fridericia'. At the bottom right is a 'Restore Defaults' button, and at the bottom are 'OK', 'Cancel', and 'Apply' buttons.


## Acquisition Settings Tab

Select **Acquisition Settings** to set or change options available for the acquisition window.

The screenshot shows the 'ECG Options' dialog box with the 'Acquisition Settings' tab selected. The dialog has a title bar with a close button (X) and a tabbed interface with five tabs: 'General', 'Analysis', 'Acquisition Settings', 'Review Window', and 'Edit Lead Order'. The 'Acquisition Settings' tab is active, showing several configuration sections: 'Acquisition Colors' with radio buttons for 'Black on Red' (selected), 'Green on Black', and 'Blue on Grey'; 'Acquisition Grid' with radio buttons for 'No Grid', 'Dotted Lines', and 'Solid Lines' (selected); 'Acquisition Sounds' with checkboxes for 'Beep on QRS', 'Beep on Leads Off' (checked), and 'Use MIDI Sounds'; 'Custom Lead Group' with three dropdown menus for 'Lead 1' (I), 'Lead 2' (II), and 'Lead 3' (III); 'Filter' with checkboxes for 'Muscle Filter' and 'Power Filter', and radio buttons for '50 Hz' and '60 Hz' (selected); 'Rhythm Strip Lead' with a dropdown menu set to 'II'; 'Leads' with a dropdown menu set to '3x4 View'; and a checkbox for 'Create PDF when Save and Print is selected.' At the bottom right is a 'Restore Defaults' button, and at the bottom are 'OK', 'Cancel', and 'Apply' buttons.


### Review Window Tab

Select **Review Window** to set or change options available for the ECG Review Window.

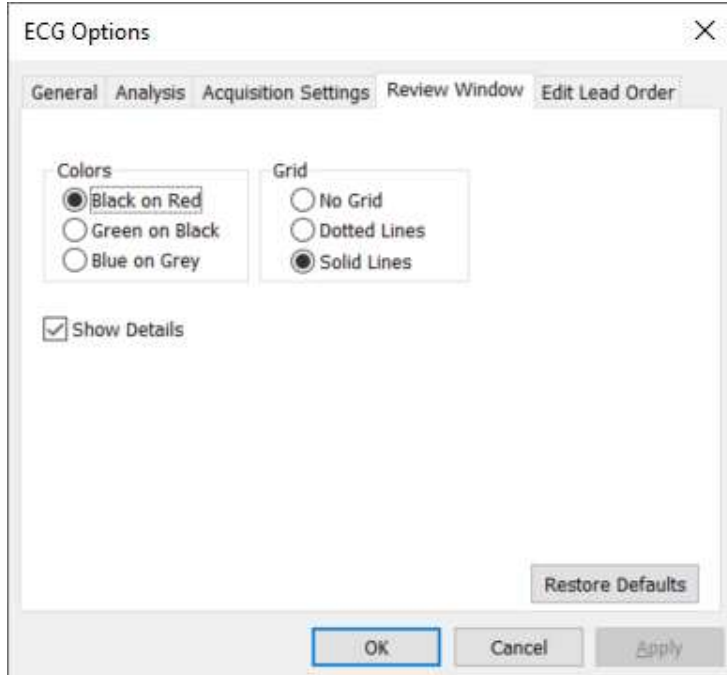


### Edit Lead Order Tab

Select **Edit Lead Order** to change the lead order. Note, the setting applies to both the acquisition and review windows.

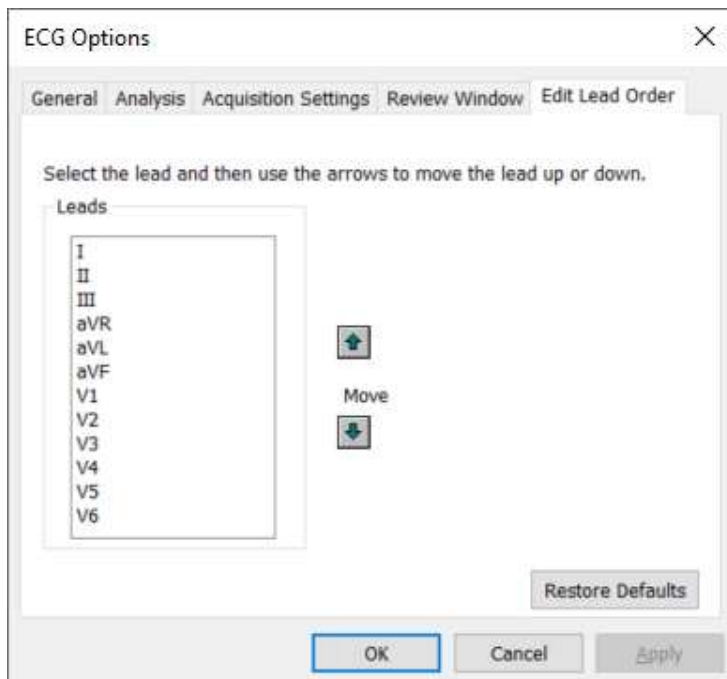


## Reviewing an ECG


### Reviewing an ECG within the ECG Review Window

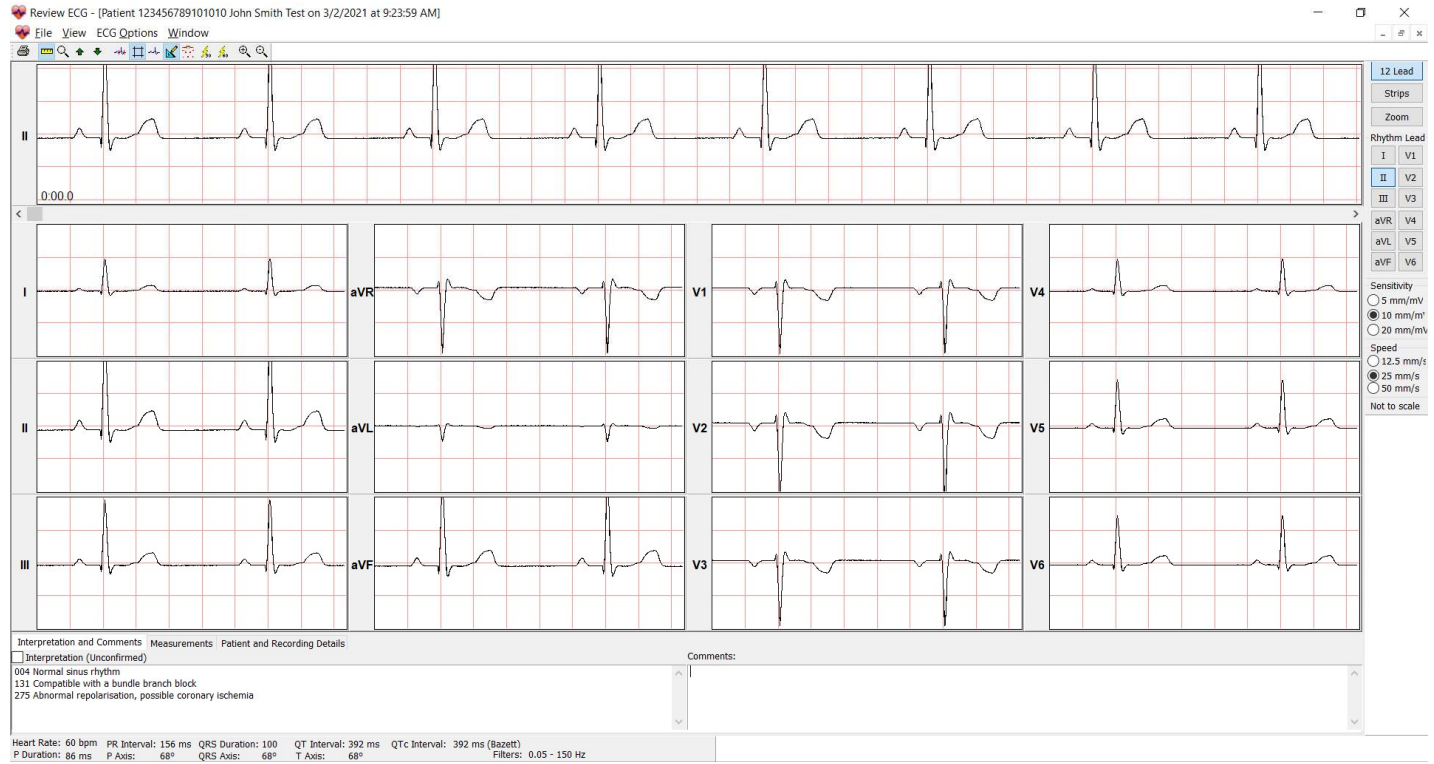


Note: ECG tests with an asterisk symbol are acquired by VectraplexECG. VectraplexECG acquired ECGs must be viewed in VectraplexECG.






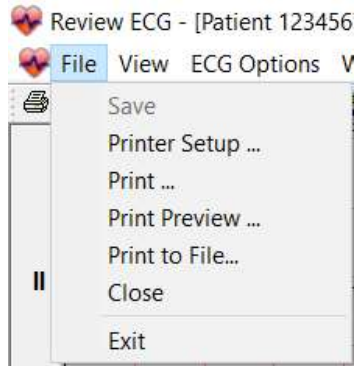


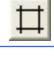

## File Menu

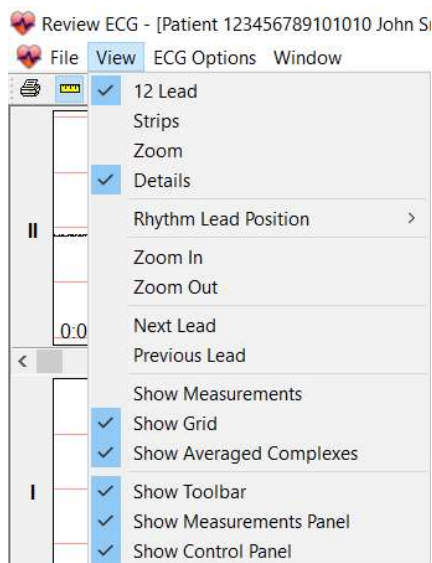


Menu Item	Icon	Function
Save	NA	Saves changes.
Printer Setup...	NA	Opens the print setup window for the default printer.
Print..		Prints the ECG test.
Print Preview...	NA	Previews the hardcopy report.
Print to File...	NA	Creates an image file (JPEG, TIFF, or PDF) of the hardcopy report.
Close	NA	Closes open tests without closing the review window.
Exit	NA	Closes open tests and exits the review window.
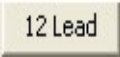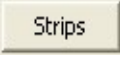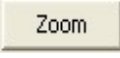
## View Menu




Menu Item	Icon	Function
12 Lead		Selects the 12 lead view of the ECG. See <a href="#">12 Lead View</a> for an example.
Strips		Selects the three lead strip view of the ECG. See <a href="#">ECG Strips View</a> for an example.
Zoom		Selects the zoom view of the ECG. See <a href="#">Zoom View</a> for an example.
Details		Displays the interpretation, comments, and detailed measurements. See <a href="#">Details View</a> for an example.
Rhythm Lead Position	NA	Toggles the Rhythm Lead to the top or the bottom of the screen (12 Lead view only).
Zoom In		Enlarges the ECG.
Zoom Out		Reduces the ECG.
Next Lead/ Previous Lead		Scrolls through recorded leads.
Show Measurements		Turns the averaged complex's measurements On and OFF.
Show Grid		Turns the grid lines on and off.
Show Averaged Complexes		Switches between averaged complexes and 10 second lead strip displays.
Show Toolbar	NA	Displays or removes the toolbar.
Show Measurements Panel	NA	Displays or removes the summary measurements panel.
Show Control Panel	NA	Displays or removes the Control Panel.

## ECG Options Menu

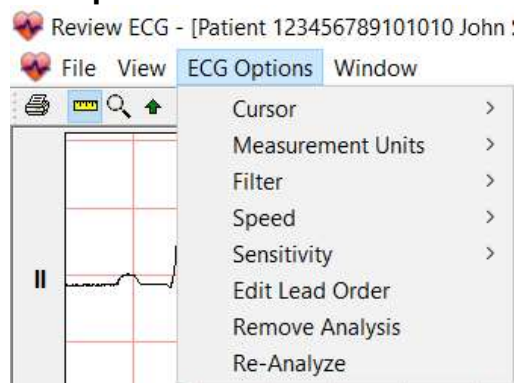


Menu Item	Icon	Function
Cursor	or	Toggles the cursor between the Zoom tool used to increase or decrease the ECG display or the Measure tool used for on-screen calipers.
Measurement Units	NA	Selects Millimeters or MicroVolts
Filter		<b>Muscle Filter</b> Activates the Muscle Filter.
		<b>Power Filter</b> Activates the Mains Filter. The Hz are set in the <a href="#">ECG Options</a> .
Speed	<input type="radio"/> 12.5 mm/s <input checked="" type="radio"/> 25 mm/s <input type="radio"/> 50 mm/s	Changes the number of millimeters that are passed in one second. The available options are 12.5mm/s, 25mm/s and 50mm/s.
Sensitivity	<input type="radio"/> 5 mm/mV <input checked="" type="radio"/> 10 mm/mV <input type="radio"/> 20 mm/mV	Changes the number of millimeters that represent one millivolt. The available options are 5mm/mV, 10mm/mV, and 20mm/mV.
Edit Lead Order...	NA	Select <b>Edit Lead Order</b> to change the lead order.
Remove Analysis	NA	Removes the detailed measurements. The narrative interpretation and comments remain unchanged.
Re-analyze	NA	Resets the interpretation to its original state and removes all changes made by the user to the narrative interpretation.

## Views

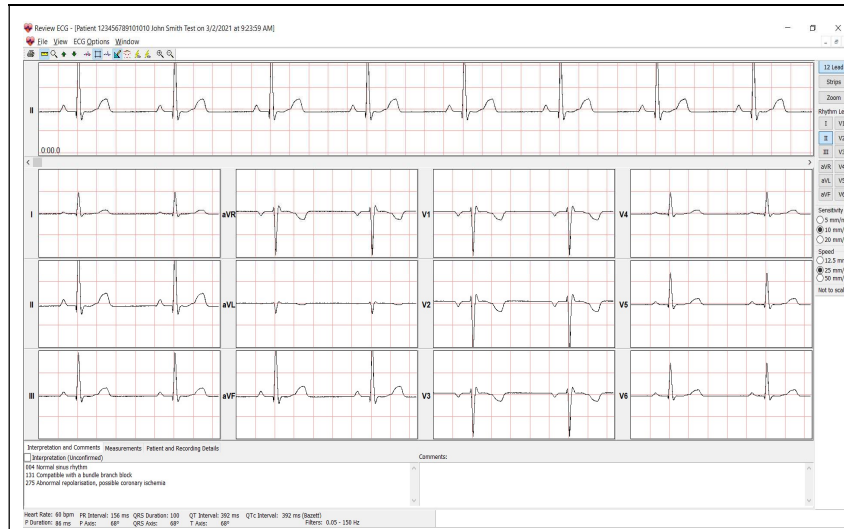


### 12 Lead

12 channels of averaged complexes and a rhythm strip.

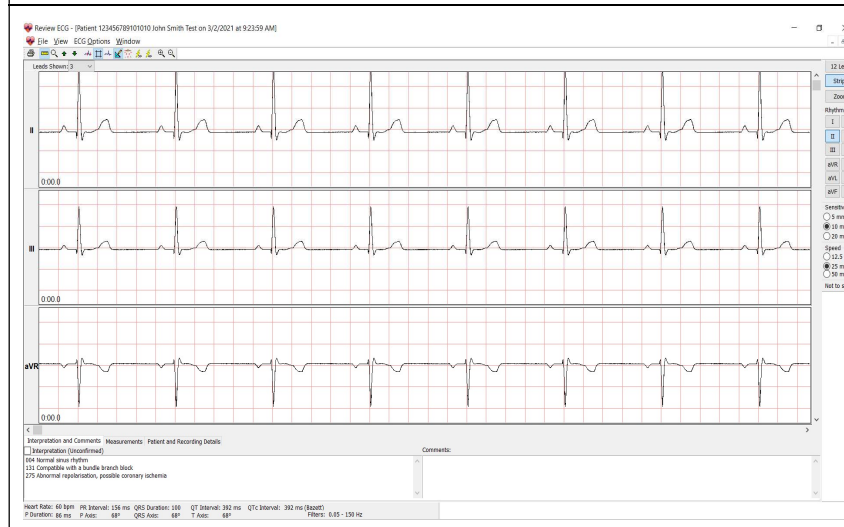

### Strips

Displays ECG strips. Use the Leads Shown option to change the number of visible leads.

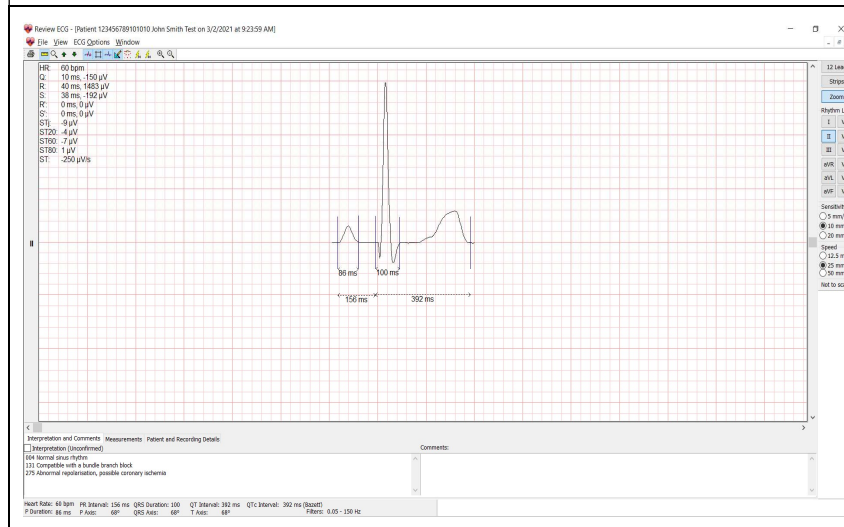

### Zoom

Displays a single lead strip or an averaged complex.

## Details View



The details view shows the interpretation, comments, detailed measurements and patient details of the ECG.

## Interpretation and Comments

Interpretation and Comments	Measurements	Patient and Recording Details
<input type="checkbox"/> Interpretation (Unconfirmed) 004 Normal sinus rhythm 131 Compatible with a bundle branch block 275 Abnormal repolarisation, possible coronary ischemia		Comments:
Heart Rate: 60 bpm PR Interval: 156 ms QRS Duration: 100 ms QT Interval: 392 ms QTc Interval: 392 ms (Bazett) P Duration: 86 ms P Axis: 68° QRS Axis: 68° T Axis: 68° Filters: 0.05 - 150 Hz		

## Measurements

Interpretation and Comments	Measurements	Patient and Recording Details																																																																																																																														
	<table border="1"> <thead> <tr> <th>Lead</th> <th>Type</th> <th>P+</th> <th>P-</th> <th>Q</th> <th>R</th> <th>S</th> <th>R'</th> <th>S'</th> <th>J</th> <th>ST20</th> <th>ST60</th> <th>ST80</th> <th>T+</th> <th>T-</th> <th>ST</th> <th>Q1</th> <th>R1</th> <th>S1</th> <th>R'1</th> <th>S'1</th> </tr> </thead> <tbody> <tr> <td>I</td> <td>qRs</td> <td>50</td> <td>0</td> <td>-32</td> <td>497</td> <td>-64</td> <td>0</td> <td>0</td> <td>-4</td> <td>0</td> <td>-1</td> <td>1</td> <td>97</td> <td>0</td> <td>0</td> <td>8</td> <td>40</td> <td>40</td> <td>-</td> <td>-</td> </tr> <tr> <td>II</td> <td>qRs</td> <td>152</td> <td>0</td> <td>-150</td> <td>1483</td> <td>-192</td> <td>0</td> <td>0</td> <td>-9</td> <td>-4</td> <td>-7</td> <td>1</td> <td>290</td> <td>0</td> <td>-250</td> <td>10</td> <td>40</td> <td>38</td> <td>-</td> <td>-</td> </tr> <tr> <td>III</td> <td>qRs</td> <td>103</td> <td>0</td> <td>-98</td> <td>987</td> <td>-128</td> <td>0</td> <td>0</td> <td>-5</td> <td>-4</td> <td>-6</td> <td>0</td> <td>197</td> <td>0</td> <td>-250</td> <td>10</td> <td>40</td> <td>38</td> <td>-</td> <td>-</td> </tr> <tr> <td>aVR</td> <td>rS+</td> <td>0</td> <td>-101</td> <td>0</td> <td>100</td> <td>-990</td> <td>127</td> <td>0</td> <td>6</td> <td>1</td> <td>3</td> <td>-2</td> <td>0</td> <td>-195</td> <td>250</td> <td>-</td> <td>10</td> <td>40</td> <td>38</td> <td>-</td> </tr> <tr> <td>aVL</td> <td>r+</td> <td>0</td> <td>-27</td> <td>-245</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>0</td> <td>&gt;0</td> <td>42</td> <td>40</td> <td>-</td> <td>-</td> <td>-</td> </tr> </tbody> </table>	Lead	Type	P+	P-	Q	R	S	R'	S'	J	ST20	ST60	ST80	T+	T-	ST	Q1	R1	S1	R'1	S'1	I	qRs	50	0	-32	497	-64	0	0	-4	0	-1	1	97	0	0	8	40	40	-	-	II	qRs	152	0	-150	1483	-192	0	0	-9	-4	-7	1	290	0	-250	10	40	38	-	-	III	qRs	103	0	-98	987	-128	0	0	-5	-4	-6	0	197	0	-250	10	40	38	-	-	aVR	rS+	0	-101	0	100	-990	127	0	6	1	3	-2	0	-195	250	-	10	40	38	-	aVL	r+	0	-27	-245	0	0	0	0	0	0	0	0	0	0	>0	42	40	-	-	-	
Lead	Type	P+	P-	Q	R	S	R'	S'	J	ST20	ST60	ST80	T+	T-	ST	Q1	R1	S1	R'1	S'1																																																																																																												
I	qRs	50	0	-32	497	-64	0	0	-4	0	-1	1	97	0	0	8	40	40	-	-																																																																																																												
II	qRs	152	0	-150	1483	-192	0	0	-9	-4	-7	1	290	0	-250	10	40	38	-	-																																																																																																												
III	qRs	103	0	-98	987	-128	0	0	-5	-4	-6	0	197	0	-250	10	40	38	-	-																																																																																																												
aVR	rS+	0	-101	0	100	-990	127	0	6	1	3	-2	0	-195	250	-	10	40	38	-																																																																																																												
aVL	r+	0	-27	-245	0	0	0	0	0	0	0	0	0	0	>0	42	40	-	-	-																																																																																																												
Heart Rate: 60 bpm PR Interval: 156 ms QRS Duration: 100 ms QT Interval: 392 ms QTc Interval: 392 ms (Bazett) P Duration: 86 ms P Axis: 68° QRS Axis: 68° T Axis: 68° Filters: 0.05 - 150 Hz																																																																																																																																

## Patient and Recording Details

Interpretation and Comments	Measurements	Patient and Recording Details																																				
		<table border="1"> <thead> <tr> <th colspan="2">Patient Details</th> <th colspan="2">Recording Details</th> </tr> </thead> <tbody> <tr> <td>Date of Birth:</td> <td>2/10/2000</td> <td>Age:</td> <td>21 years</td> </tr> <tr> <td>First Name:</td> <td>John</td> <td>Height:</td> <td>5' 9"</td> </tr> <tr> <td>Last Name:</td> <td>Smith</td> <td>Weight:</td> <td>0 lbs</td> </tr> <tr> <td>Patient ID:</td> <td>123456789101010</td> <td>Gender:</td> <td>Male</td> </tr> <tr> <td>Acquired:</td> <td colspan="2">3/2/2021 9:23:59 AM</td> <td>Revision:</td> </tr> <tr> <td>Device Type:</td> <td colspan="2">CL</td> <td></td> </tr> <tr> <td>Recorded With:</td> <td colspan="2">0</td> <td></td> </tr> <tr> <td>Serial Number:</td> <td colspan="2">Z-70-00-0500-</td> <td></td> </tr> </tbody> </table>	Patient Details		Recording Details		Date of Birth:	2/10/2000	Age:	21 years	First Name:	John	Height:	5' 9"	Last Name:	Smith	Weight:	0 lbs	Patient ID:	123456789101010	Gender:	Male	Acquired:	3/2/2021 9:23:59 AM		Revision:	Device Type:	CL			Recorded With:	0			Serial Number:	Z-70-00-0500-		
Patient Details		Recording Details																																				
Date of Birth:	2/10/2000	Age:	21 years																																			
First Name:	John	Height:	5' 9"																																			
Last Name:	Smith	Weight:	0 lbs																																			
Patient ID:	123456789101010	Gender:	Male																																			
Acquired:	3/2/2021 9:23:59 AM		Revision:																																			
Device Type:	CL																																					
Recorded With:	0																																					
Serial Number:	Z-70-00-0500-																																					
Heart Rate: 60 bpm PR Interval: 156 ms QRS Duration: 100 ms QT Interval: 392 ms QTc Interval: 392 ms (Bazett) P Duration: 86 ms P Axis: 68° QRS Axis: 68° T Axis: 68° Filters: 0.05 - 150 Hz																																						

**Warning!** The computerized interpretation provided by the Office Medic software is only valid when used in conjunction with clinical findings. All computer generated tracings and interpretations must be confirmed by a qualified physician.

## Printing an ECG

Select **File** | **Print** or the print icon.

Print Options	Description
Single Lead Report	Prints a single strip or averaged complex with scale options: 1x, 2x, 4x, 8, 16x).
3X4 <u>S</u> imultaneous	Prints 2.5 second segments of all twelve channels displayed at the same point in time along with a 10 second single channel rhythm strip.
3X4 Sequential Report	Prints 2.5 second segments of all twelve channels displayed at the same point in time progressing in four sequential columns along with a 10 second single channel rhythm strip.
<u>A</u> verage Complexes	Print an average QRS complex for all 12 channels along with a 10 second single channel rhythm strip.
6X1 Format, <u>2</u> Page	Prints a ten second trace of each channel (2 page report).
6X2 Format, <u>1</u> Page	Prints a five second trace of each channel (1 page report).
Measurements <u>t</u> able	Prints a chart with amplitude, slope and duration data for all twelve channels.
Include	Allows you to select whether to include Measurements, Interpretation, and/or Comments in the report(s).
Speed and Sensitivity	Allows you to select the Speed (12.5, 25, or 50mm/s) and Sensitivity (5, 10, or 20 mm/mV) of the ECG reports.
Minor grid	Allows you to select the minor grid: Lines, Dots, or None.

Note: when printing to a low resolution printer select Dots or None for the minor grid.

## **ECG Device Verification**

A periodic check of the ECG system with an ECG simulator is recommended. Intervals for these checks can be set at the discretion of your Medical Director. There are commercially available ECG simulators which may be used for this purpose, refer to the accompanying information for instructions on the use of these.

For further information on device verification, contact VectraCor at [www.VectraCor.com](http://www.VectraCor.com).

## **ECG Analysis Program**

Office Medic provides analysis and interpretation of 12 channel ECGs. This is based on algorithms developed by Cardionics S.A. For further information consult the ECG Physician's Guide.

### **What to expect from the analysis program**

The ECG Analysis Program provides an analysis of the amplitudes, duration, and morphologies of the ECG waveform. The analysis is based upon standards of interpretation of these parameters and calculations of the electrical axis and relationship between leads.

The interpreted ECG is a tool to assist the physician in making a clinical diagnosis, and is not a substitute for the physician's knowledge, the patient's history, results of the physical exam, the ECG tracing or other findings.

## Troubleshooting and Tips



### Office Medic shows "Error connecting to Database" message

**Error Description: Invalid connection string attribute, Error number: 2147467259**

If Office Medic had been working before the error, perform the following steps to correct the issue:

1. Open command prompt as administrator.
2. Enter following command in command prompt.  
`SC QUERY "MSSQL$OFFICEMEDIC"`  
If the STATE is RUNNING, skip to step 5.
3. Enter following command to start SQL Server database service.  
`SC START "MSSQL$OFFICEMEDIC"`
4. Enter the following command to delay the SQL service at boot time.  
`SC CONFIG "MSSQL$OFFICEMEDIC" start=delayed-auto`
5. Enter the following command to restart service if SQL service fails to boot.  
`SC FAILURE "MSSQL$OFFICEMEDIC" actions=restart/180000/restart/180000/restart/180000 reset=86400`
6. Right Click on Office Medic and choose "Run as administrator" to reset the user permission to access the database.

If the error occurs after installation and during first use, perform the following steps to correct the issue:

1. Reboot the computer.
2. After reboot, click on Start button. Search for and/or run "Windows update".
3. Apply all Windows updates marked as important.
4. Reboot the computer and re-run "Windows update".
5. Repeat Steps 1 to 4 until all updates are run.
6. Re-install Office Medic.




### How to hide patient names from the patient tree in Office Medic?

**User intention: To hide the patient from the Patient tree or to search for patient rather than utilizing patient tree in Office Medic.**

It is possible to customize Office Medic in search mode where only certain patients will appear on Patient tree. The following steps should be performed to achieve this functionality:

1. Right click on the Office Medic shortcut icon.
2. Place your cursor at the end of the Target field. Insert a space and enter the term `'-query'`.

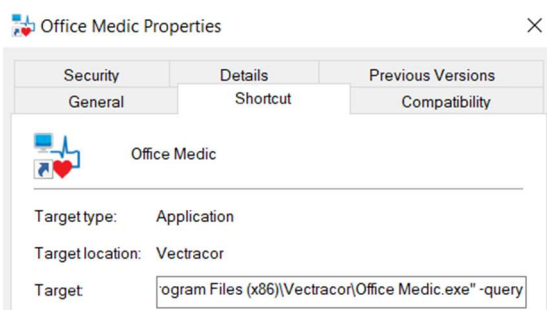

3. Select OK.
4. Open Office Medic. A patient query screen will appear. Enter as much patient information as possible and select OK.

**Instructions for using Patient Query:**

- More specific patient information input results in loading fewer patients.
- In the Patient Query window, clicking OK after leaving blank fields will lead to loading of entire database.
- If Cancel is selected, Office Medic loads with no patients. Click F5 or Refresh Patient Tree to search for a patient.



**How to reduce artifacts in an ECG tracing?**

**Artifacts can occur for different reasons:**

1. Incorrect lead placement.
2. Failure to properly prep the patient like not using enough conductive materials (alcohol, gel etc.).
3. Artifact created by excessive muscle movement or electrical interference.
4. Faulty lead wires.

By promptly following the instructions for Performing an ECG as mentioned in Quick start guide, the user should be able to eliminate or reduce the artifact. If not, contact Technical support to receive an RMA#. Device should be inspected to determine if the lead(s) is faulty.



**How to troubleshoot Leads Off Indication?**

**Error Description: A leads off condition is characterized by a drop in signal on one or more ECG leads. The Software notifies the user that a lead is off by**

1. **Placing a red circle with a slash on the affected marker (visible on left side of the screen).**
2. **Sounding an alarm (optional).**



To determine which lead(s) is down, refer to the table below:

Electrode down AHA (IEC)	Affected lead(s) as displayed by Office Medic
RL (N)	All Leads
RA (R)	All Leads
LA (L)	All but Lead II (L)
LL (F)	All Leads
V1 (C1)	V1 (C1)
V2 (C2)	V2 (C2)
V3 (C3)	V3 (C3)
V4 (C4)	V4 (C4)
V5 (C5)	V5 (C5)
V6 (C6)	V6 (C6)

The most common causes of leads off conditions occur due to

1. Bad or expired electrodes
2. Improper patient preparation.


Perform the following steps to troubleshoot a Lead(s) Off condition:

1. Inspect the electrode and electrode adapters
  - a. Check the expiration date located on the electrode package. Patient electrodes have a conductive gel that breaks down over time and will cause a lead(s) off condition and/or artifacts.
  - b. Replace electrodes on the patient with known good package of non-expired electrodes.
  - c. Re-run ECG test.
2. Ensure the adapter ("alligator" clip or snap adapter) is not damaged by connecting the adapter to a different electrode. If Lead V6 is OFF and V5 appears on the screen, swap the adapters between V5 and V6. If Lead V5 is OFF, replace the adapter with a new one.
3. Ensure the patient is properly prepared as per the Quick start guide for performing an ECG.

If the steps above cannot solve the problem, contact Technical Support.



### Office Medic shows "Cannot find ECG cable" message

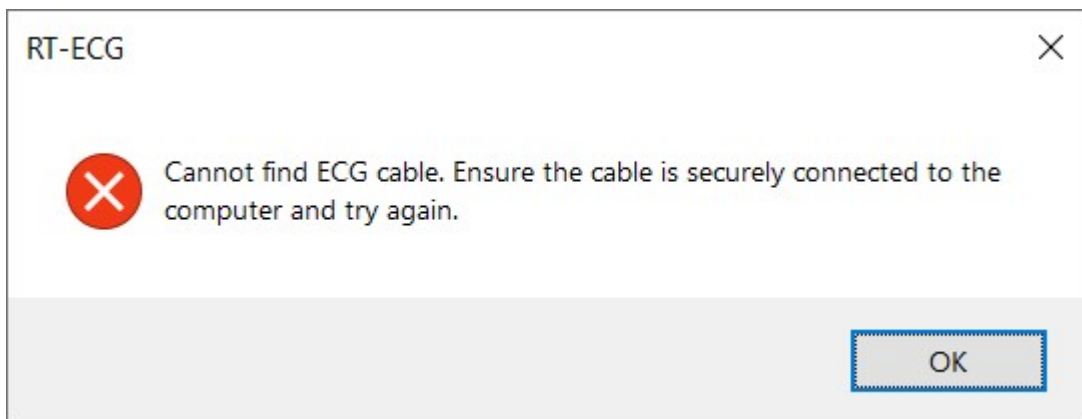

**This error message occurs when the ECG device is not plugged in or not recognized by the computer.**






Perform the following steps to troubleshoot cable connection failed condition:

1. Ensure the ECG device is connected to a USB port.
2. If the error appears when the ECG device is connected, connect device to a different USB port or computer.
3. Contact VectraCor Support and send in device for servicing if the error persists.



## Other Office Medic messages to users

Office Medic will output messages to the user in response to different user actions usually to help the user complete an action.

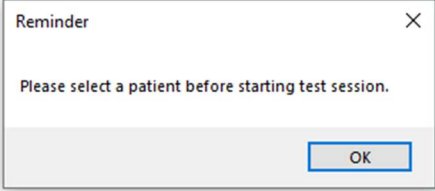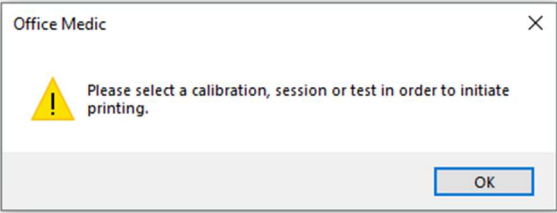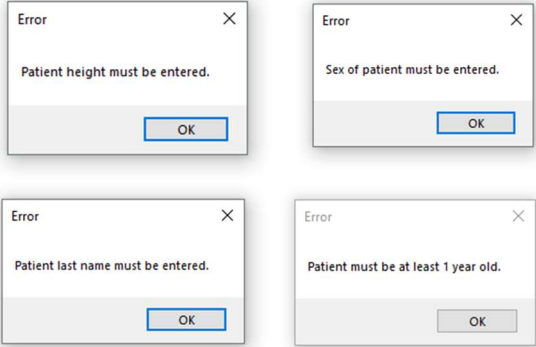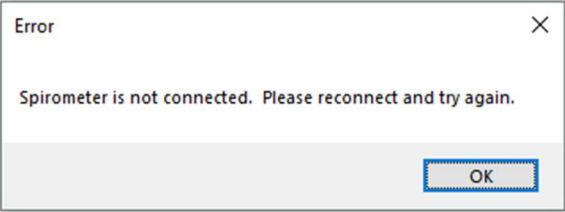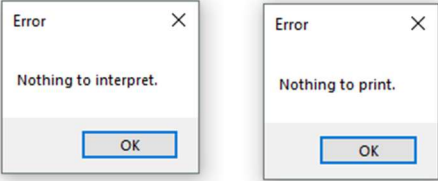Office Medic Message:	Reason for Message:	Correction:
	Office Medic will show this message if a patient has not been selected for a test	Highlight a patient before initializing the test
	A calibration, session, or test has not been selected when user tries to print	Select a calibration, session, or test prior to printing
	User tries to create a patient without filling the required fields	Fill in the required fields before creating the patient
	Spirometer is not connected when trying to do a spirometry test	Connect spirometer to PC
	User hit Interp or Print buttons when no test is performed for the test session	Perform a spirometer test for the session

## Service Information

### Device Care & Maintenance

#### **Cleaning**

To clean the spirometer, clean surfaces with a damp cloth using water only. DRY THOROUGHLY. AVOID CLEANING AROUND CONNECTORS. Excess moisture in or on the case, cables or air fittings could affect operation.

To clean the ECG device, wipe the surfaces of the case, the leadwires, and connectors with a clean cloth moistened in water only. To disinfect the ECG device wipe the case, the leadwires, and connectors with an alcohol-free hospital grade disinfectant. DRY THOROUGHLY.

Cautions:

- Do not clean the case with alcohol.
- Do not saturate or immerse the case with liquid during cleaning.
- Do not sterilize ECG device.

#### **Calibration**

Spirometer:

Follow directions in the Spirometry Tools section

ECG Device:

User calibration of device is not required. However, a periodic check of the ECG system with an ECG simulator is recommended. Intervals for these checks can be set at the discretion of your Medical Director. There are commercially available ECG simulators which may be used for this purpose; refer to its accompanying information for instruction on the use of these. If calibration of the device is necessary (e.g. because of physical shock) and an ECG simulator is not available, contact the VectraCor Customer Care/Service Department.

#### **Handling**

Do not insert a dirty USB cable into the USB port.

Spirometer: Avoid contaminating the luer (connector to the pressure tube).

#### **Storage**

Store the device in a dry place. Avoid sudden changes in temperature.

#### **Physical Shock**

Avoid physical shock; a device that has been dropped should have the calibration verified before use on a patient.

#### **Inspection**

Inspect device for damage initially and before each use. Do not use devices that show visual signs of damage. Contact the VectraCor Service department with questions related to device damage and repair.

#### **Training**

Contact VectraCor for training on Office Medic and its devices; VectraCor will provide materials and training for the system.

## Service

Contact the VectraCor service department:

VectraCor, Inc.  
785 Totowa Road Suite 100  
Totowa, NJ 07512 USA

Monday through Friday 8am to 6pm EST  
Phone: 973-904-0444  
www.vectracor.com

A Return Merchandise Authorization (RMA) number will be issued for repairs

**THE INSTRUMENT MUST BE RETURNED FOR REPAIRS AT THE EXPENSE OF THE PURCHASER. IN-WARRANTY REPAIRED UNITS ARE RETURNED AT THE EXPENSE OF QRS OR ITS AUTHORIZED AGENT. FOR OUT OF WARRANTY WORK THE CUSTOMER IS RESPONSIBLE FOR ALL FREIGHT CHARGES.**

## Limited Warranty

- All instruments sold and supplied by VectraCor are guaranteed to be free from defects in material and workmanship for a period of 3 year from date of purchase. All supplies and accessories carry a 90-day limited warranty. This includes oximetry sensors. If in the judgment of VectraCor the instrument is proven to be defective during the warranty period it will be repaired or replaced with no charge for parts or labor.
- This warranty does not cover any instrument that has been damaged by accident, misuse, abuse or has been altered or repaired by anyone other than an authorized VectraCor agent. This warranty also does not cover any unit that has had the serial number removed, defaced or rendered illegible.
- **THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS AND IS HEREBY LIMITED TO REPAIR OR REPLACEMENT OF INSTRUMENTS FOUND DEFECTIVE DURING THE WARRANTY PERIOD. AN AUTHORIZED VECTRACOR AGENT, MUST MAKE ALL REPAIRS. INSTRUMENTS SENT BY MAIL OR COMMON CARRIER SHOULD BE INSURED AGAINST LOSS OR DAMAGES, AS THEY ARE NOT COVERED BY THIS WARRANTY.**
- Technical support on software is under warranty for 1-year. This includes ECG lead wires. A software support package is available after 1-year at an additional cost.

## Glossary of Terms

<b>%PRED</b>	Ratio of patient's actual results compared to predicted normal values, expressed as a percentage. Abnormality is defined by using one standard deviation for each variable rather than any specific percentage below the predicted value. Results above 100% are above average.
<b>ATS</b>	American Thoracic Society, a scientific medical organization active in pulmonary research and care of patients with lung diseases. The ATS has recommended standards for spirometers.
<b>BF Equipment</b>	Degree of protection against electrical shock.
<b>Bronchodilator</b>	A type of drug (i.e. Albuterol), usually administered in an aerosol spray, that is used to dilate air passages to reduce any restrictions to airflow.
<b>BTPS</b>	Body Temperature and Pressure, Saturated: A number, which uniformly expresses all Spirometry results at body temperature and pressure, fully saturated with water.
<b>Calibration Syringe</b>	A large syringe which injects a measured amount of air into the mouthpiece. Many syringes have a stop ring on the plunger, which allows injecting various calibrated amounts of air.
<b>Class II Equipment</b>	Double insulated equipment.
<b>COPD</b>	Chronic Obstructive Pulmonary Disease.
<b>EOTV</b>	End-of-test volume.
<b>ERS</b>	European Respiratory Society.
<b>EX TIME</b>	Expiratory Time, expressed in seconds - time elapsed between the beginning and completion of expiration.
<b>FEF 25-75%</b>	Forced expiratory flow during the middle half (25-75%) of the FVC (formerly called the maximum middle expiratory flow rate), expressed in liters per second. This is the most sensitive measure of small airways obstruction (typically seen in smokers).
<b>FEF<sub>xx</sub>%</b>	Forced Expiratory Flow at xx% point of the FVC, expressed in liters per second.
<b>FET</b>	Forced Expiratory Time.
<b>FEV<sub>1</sub>/FEV<sub>6</sub> FEV<sub>1</sub>/FVC.</b>	Ratio of FEV <sub>6</sub> exhaled in one second. May be used as a surrogate for
<b>FEV<sub>6</sub> (L) forced expiratory volume</b>	Measured six seconds after commencement of expiration. May be used as a surrogate for FVC.
<b>FEV<sub>x</sub>/FVC%</b>	The percentage ratio of Forced Expiratory Volume (timed) to Forced Expiratory Vital Capacity, expressed as a percentage.
<b>FIF.2-1.2</b>	Forced Inspiratory Flow between 200ml. and 1200ml. Flow of inspired air measured after the first 200ml. And during the next 1000ml.

<b>FIF 25-75%</b>	Forced Inspiratory flow during the middle half (25-75%) of the FIVC expressed in liters per second.
<b>FIFxx%</b>	Forced Inspiratory Flow at xx% point of the FIVC, expressed in liters per second.
<b>FIVx/FIC%</b>	The percentage ratio of Forced Inspiratory Volume (timed) to Forced Inspiratory Vital capacity, expressed as a percentage.
<b>Flow vs. Volume Curve</b>	Graph obtained by forced exhalation test, Flow is plotted on the vertical axis and volume on the horizontal axis.
<b>Forced Expiratory Flow</b>	It is the rate of flow, expressed in liters per second, at various points in the volumetric flow, i.e. FEF25%, FEF50%, FEF75%.
<b>Forced Expiratory Volume (timed), (FEV(t))</b>	Maximal volume of air, expressed in liters, which can be expelled in specific time in a forced capacity test.
<b>Forced Inspiratory Vital Capacity, (FIVC)</b>	Total volume of air, expressed in liters, which can be inhaled during a rapid forced inhalation after a maximal expiration.
<b>Forced Inspiratory Flow</b>	Inspiratory rate of flow, expressed in liters per second, at various points in the volumetric flow, i.e. FIF25%, FIF50%, FIF75%.
<b>Forced Vital Capacity (FVC)</b>	Total volume of air, expressed in liters, which can be exhaled during a rapid forced exhalation after a maximal inspiration.
<b>LLN</b>	Lower limit of normal.
<b>Maximum Voluntary Ventilation (MVV)</b>	The maximum volume of air that can be inhaled and exhaled repeatedly through the lungs over a period of time (usually 12 seconds) and extrapolated to one minute.
<b>Obstruction</b>	Limitation of airflow. It is shown by the FVC test. Low FEV1/FVC% ratio is the main indication of airways obstruction. Reductions in FEV3/FVC% and FEF25–75% best demonstrate obstruction of small airways.
<b>PC Card</b>	Also known as a PCMCIA card. A standard 68-pin computer card designed to add modular hardware to computers.
<b>Perfusion</b>	Display indicating if the pulse waveform signal is of good quality and the SpO2 data is accurate.
<b>Peak Expiratory Flow Rate (PEFR)</b>	Maximum instantaneous flow in the FVC test.
<b>PFT</b>	Pulmonary Function Test.
<b>PEFT</b>	Peak Expiratory Flow Time.
<b>PIFR</b>	Peak Inspiratory Flow Rate, expressed in liters per second.
<b>Predictor Pulmonary Functions Tests</b>	Predicted value according to the "normal" equations used. see PFT

**Pulse Rate**

Heart rate measured in beats per minute (bpm).

**RR**

Respiratory Rate: the average number of inhalations/exhalations per minute performed during a test.

**Signal Intensity**

Indication displaying the patient's pulse.

**Slow Vital Capacity (SVC)**

Total volume of air, expressed in liters, which can be exhaled during a slow exhalation after a maximal inspiration. Amount may be decreased because of disorders that cause volume restriction in the lung.

**SpO2**

Approximate percentage of oxygen saturation in hemoglobin.

# Device Specifications

## Universal SmartECG Specifications

<b>Hub Weight</b>	280 - 335 grams (0.62 – 0.66 lb)
<b>Hub Dimensions</b>	85mm x 91mm x 20mm (3.3" x 3.6" x 0.8")
<b>Patient Leads Length</b>	1.0 meter (3.3 ft) for limb leads 0.6 meter (2.0 ft) for chest leads
<b>PC Connection Length</b>	1.8 meter (5.9 ft) USB cable
<b>Patient Leads</b>	12 Lead Cable (10 patient leads)
<b>Case Material</b>	ABS Plastic
<b>Electrode Connections</b>	4 mm Banana plug with "tab" or "snap" connectors
<b>Electrode Labeling</b>	Abbreviations and colors to comply with either IEC or AAMI (AHA) standards
<b>Display and Operating Console</b>	Dependent on PC (supplied by user)
<b>Gain/Sensitivity</b>	5, 10, 20 mm/mV
<b>Input Range</b>	±5 mV
<b>Acquisition sample rate</b>	1000 samples per second (compressed to 500Hz with peak picking and averaging algorithm)
<b>Heart Rate Range</b>	30 bpm - 200 bpm
<b>Frequency Response</b>	0.05 to 175Hz ± 3dB
<b>Defibrillator Protection</b>	Patient leads are isolated from system and operator, with 5 kV protection
<b>Common Mode Rejection</b>	-60dB (minimum)
<b>Safety Standards</b>	Complies with IEC 60601-1, IEC 60601-1-2, and IEC 60601-2-25
<b>Accuracy</b>	Accurate to IEC 60601-2-25 essential performance requirements.
<b>Leads Off Indicators</b>	Connection status for each lead is shown on Acquisition screen
<b>Power Source</b>	Powered by PC via USB interface
<b>Supply Voltage</b>	5V DC
<b>Supply Current</b>	<17mA DC
<b>Permanent Filters</b>	High Pass: 0.05Hz 2nd order Low Pass: 175Hz 2nd order Baseline Wander: Baseline reset by adaptive zeroing algorithm
<b>Notch filter (Mains Noise Rejection)</b>	50Hz 4th order Butterworth 60Hz 4th order Butterworth
<b>Low pass (Muscle Artifact Filter)</b>	35Hz 4th order
<b>Report Capabilities</b>	User selectable Report formats
<b>Environmental Conditions</b>	Operating Temperature: 10 to 35° C (50 to 95° F) Storage Temperature: -20 to 70° C (-4 to 158° F) Operating Relative Humidity: 20 to 85% (non-condensing) Storage Relative Humidity: 10 to 85% (non-condensing) Atmospheric Pressure: 70 kPa – 106 kPa

## Orbit Portable Spirometer Specifications

<b>Weight</b>	226.8 grams (0.5 lb.)
<b>Dimensions</b>	109.2 mm x 94.0 mm x 43.2 mm (4.3" x 3.7" x 1.7")
<b>Communication Port</b>	USB
<b>Software Compatibility</b>	Office Medic Version 5.5 (or later)
<b>Storage Conditions</b>	Temperature: -15 to 50° C (5 to 122° F) Relative Humidity: < 90% (non-condensing) Atmospheric Pressure: 700 to 1060 hPa
<b>Power Supply</b>	5 Vdc ±5% 100 mA or less from the host PC USB Port
<b>Operating Conditions</b>	Temperature: 15 to 40° C (59 to 104° F) Relative Humidity: 10 to 90% (non-condensing) Atmospheric Pressure: 700-1060 hPa
<b>Spirometry Measurement Principle</b>	The pressure is converted to flow. Volume measurement by flow integration.
<b>Measurement Time</b>	FVC – 60 sec.; SVC – 60 sec.; MVV – 15 sec.
<b>Sampling Rate</b>	125 Hz
<b>Range (BTPS)</b>	FLOW: ±14 liters/second VOLUME: 0.5 – 8.0 liters
<b>Accuracy (BTPS)</b>	FLOW: <ul style="list-style-type: none"> <li>• FEF 25-75: ±5% of indication or ±200 ml/sec, whichever is greater</li> <li>• PEF: ±10% of indication or ±300 ml/sec, whichever is greater</li> </ul> VOLUME: ±3% of indication or ±50 ml, whichever is greater for FVC and FEV1 <ul style="list-style-type: none"> <li>• FVC and FEV1: ±3% of indication or ±50 ml, whichever is greater</li> <li>• MVV: ±10% of indication or ±15 L/min, whichever is greater</li> </ul>
<b>Precision (BTPS)</b>	FLOW: PEF: ±5% of indication or 150 ml/sec, whichever is greater VOLUME: FVC and FEV1: ±3% or 50 ml, whichever is greater
<b>Minimum Tracing Size</b>	FLOW VOLUME: Flow (vertical): 5 mm/L/S; Volume (horizontal): 10 mm/L VOLUME TIME: Volume (vertical): 10 mm/L; Time (horizontal): 20 mm/S
<b>Calibration</b>	ATS 3-speed or standard calibration check
<b>Predicted Normals</b>	ADULT FVC: Crapo (1981), Cherniack (1972), Morris (1971/73), Knudson (1983), Roberts (1991), ECCS/ERS/Quanjer (1993), NHANES III (1999), GLI (2012) PEDIATRIC FVC: Hsu (1979), Knudson (1983), Polgar (1971), Warwick (1977), NHANES III (1999), Zapletal (1987), Wang (1993), Quanjer (1995), GLI (2012) ADULT MVV: Cherniack (1972) PEDIATRIC MVV: Polgar (1971), Zapletal (1987)
<b>Interpretation</b>	ATS/ERS 2005, BTS-NICE 2004-2005, NLHEP 2000, Enright 1987
<b>Report Format</b>	Pre-test overlay with full page graphs Pre/Post test overlay with full page graphs
<b>Parameters Measured</b>	FVC, FEV0.5, FEV1, FEV6, FEV1/FEV6, FEV3, FEV1/FVC, FEV3/FVC, PEFR, PEFT, FEF25%, FEF50%, FEF75%, FEF25-75%, FIVC, FIV0.5, FIV1, FIV3, FIV1/FIVC, FIV3/FIVC, PIFR, FIF50%, FIF 25-75%, FIF.2-1.2, FVC/FIVC, Extrapolated Volume (Ext. Vol. BEV), EOTV, FET, MVV, RR, MTV, SVC