


COINFYCARE<sup>®</sup>

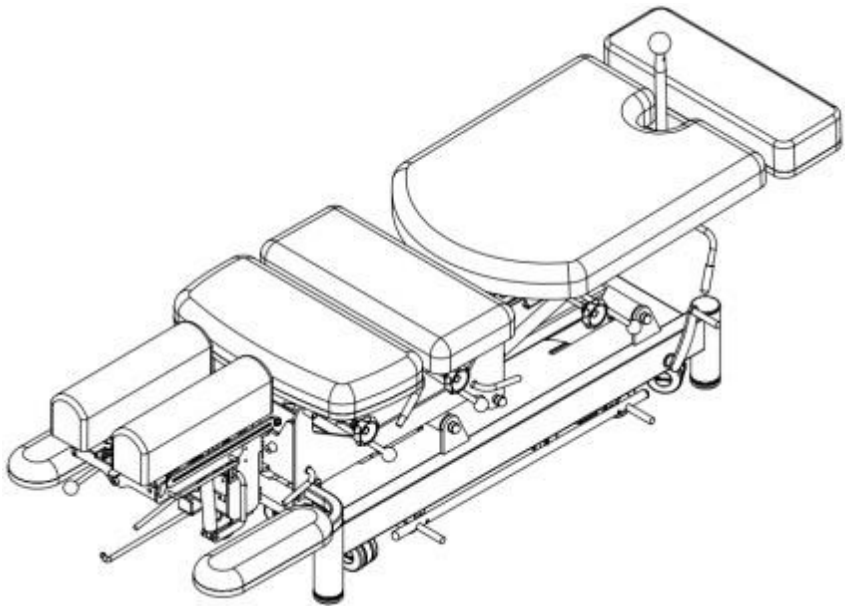

**EL07BPRO**  
**Manipulation Chiropractic Table**  
**USER GUIDE**

## **SAFETY PRECAUTIONS**

1. During adjustment of the table, do not put any part of the body into any gap or near any moving parts.
2. Please operate with both hands when adjusting the angle of back, foot or head sections.
3. When the patient is on the table, it is not recommended to adjust the angle of the table top.
4. Before using the product, ensure the four feet of

the table frame touch the ground at the same time and check the stability of the table frame.

5. Before connecting to the power supply, please check whether the incoming power line to the controller is intact.

6. A regular inspection (at least once a year) is needed for the treatment table.

7. Do not exceed the table weight capacity of 250 kg. The safe working load of the arm rest is 15 kg.

8. Always clean the table after each use with a soft cloth dampened with water and a mild antibacterial detergent. Do not use abrasive cleaners.

9. Disconnect the table from the power source before attempting any maintenance, installation, removal or replacement procedures to prevent electrical shock and possible damage to the table.

10. Please DO NOT make any changes to the table. Replacement parts or labor can be provided by

CoinfyCare, the selling distributor or a service technician certified by the manufacturer.

## **NORMAL OPERATING CONDITIONS**

Temperature: 0 °C - 40 °C

Relative humidity: 10% - 95%

Air pressure range: 860-1060hpa

Power supply: AC 100-240V, 50/60Hz

## **CONTENTS OF CARTON**

Electric Chiropractic Table: 1 unit

User Guide: 1 unit

## REGULATION AND PRODUCT STANDARDS

This table complies with the European regulation of medical devices (EU) 2017/745 and with the following standards:

- EN 60601-1:2006+A1:2013+AC:2014

Medical electrical equipment. Part 1: General Requirements for Basic Safety and Essential Performance.

- IEC/EN 60601-2-52:2009+A1:2015 clause 201.9.8.3.2

Medical electrical equipment. Part 2-52: Particular requirements for basic safety and essential operation of medical tables.

- EN 60601-1-6:2010+A1:2015

Medical electrical equipment. Part 1-6: General Requirements for basic safety and essential

performance. Collateral standard: Possibility for use

- EN 60601-1-2:2015

Medical electrical equipment. Part 1-2: General requirements for basic safety and essential

performance. Collateral standard:

Electromagnetic disturbances. Requirements and tests.

- EN ISO 13485:2016

Medical devices - Quality management systems - Requirements for regulatory purposes

(ISO 13485:2016)

DIN EN ISO 13485:2016

## PRECAUTIONS AND SAFETY INDICATIONS

CoinfyCare is not liable for any material or personal damage due to misuse or incorrect use of this table or for any purpose other than described in this user guide.

Inspect the condition of the table before using it for the first time and on a recurring basis as indicated in the "MAINTENANCE" section.

Keep the table out of high moisture environments.

Do not transport the table over rough or outdoor surfaces. The retractable caster system could become damaged and inoperable.

This table should be operated, transported, and

stored in temperatures between -10° C to + 50° C, with max humidity of 95%.

DO NOT operate this table in an environment

where other devices are being used that

intentionally radiate electromagnetic energy in an unshielded manner. Portable and mobile

communications equipment can affect Medical Electrical Equipment.

This table generates, uses, and can radiate radio

frequency energy and, if not installed and used in

accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. Harmful interference to other devices can be determined by turning this table on

and off. Try to correct the interference using one or

more of the following: reorient or relocate the receiving device, increase the separation between the

equipment, connect the table to an outlet on a different circuit from that which the other device(s) are

connected and consult CoinfyCare or your local dealer for help.

Unplug the table from the power supply before starting cleaning or any type of maintenance operations.

The table should be kept out of the reach of children when there is no adult supervision.

The table is designed for discontinuous interval

operation, so the continuous running time of the motor should never exceed 2 minutes. Should this happen, damage may occur due to overheating of the

electronics.

Always use original accessories and spare parts provided by your local dealer.

Always cover the table with a towel, disposable paper or a cover and change it for each user, thus ensuring correct hygiene and extending the useful life of the

upholstery.

Only connect the table to a properly installed and grounded outlet. The local mains voltage must

correspond to the technical data of the device.

The power outlet should be accessible in case

the table needs to be quickly unplugged at any

given time. Always pull the plug and not the mains cable.

Unplug the table from the power supply before starting cleaning or any types of maintenance operations.

## TECHNICAL SPECIFICATIONS

### Product dimensions

#### EL07BPRO Manipulation Chiropractic Table

Length: 1850-2000mm

Width: 550mm table surface, 650mm with armrests

Lifting range of bed surface: 560-900mm

Included angle between head plate and horizontal plane:  $-20^{\circ} \leq A \leq 20^{\circ}$ .

Included angle between chest plate and horizontal plane:  $-13^{\circ} \leq B \leq 15^{\circ}$ .

Included angle between leg plate and horizontal:

$-40^{\circ} \leq C \leq 25^{\circ}$ .

Included horizontal swing angle of leg plate:

$-20^{\circ} \leq D \leq 20^{\circ}$ .

Included horizontal swing angle of head plate:

$-15^{\circ} \leq E \leq 15^{\circ}$ .

## **PLACEMENT IN THE WORKPLACE**

The table has wheels to facilitate movement to the place where it will be used.

If the electrical supply cable is defective,

disconnect the table immediately and inform your dealer or a specialized technician.

Position the power cord properly so that it cannot be tripped over.

Unplug the power cord from the mains if the table is not going to be used for a long period of time.

If you detect any abnormality in the operation of the table, consult your dealer or a specialized technician. Never manipulate any electrical part or open or

repair any part of the table.

Only your dealer or a specialized technician should repair the table.

Never touch the table or the power cord with wet hands.

Never place your hands or any other part of your body in any hole or void on the table while it is in operation.

Make sure there are no objects in the space under the table that could cause collisions or entrapment during operation.

Make sure that your feet are not resting on the

peripheral foot bar or foot control during treatment and especially when sitting near the table. This

could create a dangerous situation where your leg is caught between the section and the peripheral foot bar or floor surface.

Take into account the weight distribution on the

table when working. The weight must be distributed along the table without concentrating on its ends to maintain the correct stability.

You should not sit on the head and foot sections. If

you do, there is a danger of tipping and table damage.

## EL07B PRO OPERATION

To activate or deactivate this system, you must press either of the two levers with your foot, as shown in

the following figures.

Figure 1: Table in working position.



Figure 1

Figure 2: Table in moving position.

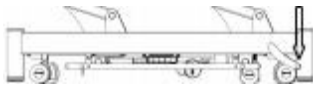


Figure 2

When the table top is lowered the bed is in a stable status.

The column at the front side of the base frame is equipped with height adjustable feet.

After moving, please confirm whether the adjustable feet are in full contact with the ground, otherwise rotate the feet for adjustment.



Figure 3 Adjustable feet of the table

## OPERATION

Connect the bed to an electrical outlet using the power cord.

### Lifting the table

- Peripheral foot bar for height adjustment

- Before adjusting, make sure there are no objects in the space below the table that could cause collisions or entrapment.
- Press the peripheral foot bar with your foot to one side or the other side to adjust the height of the table.



Figure 4 Foot bar

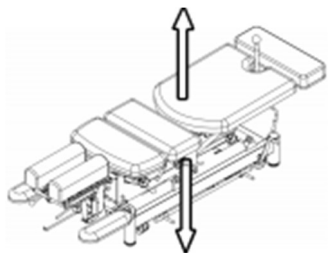


Figure 4

In order to reduce the risk of injury caused by the patient rolling off the bed when unattended, the bed should be placed at the lowest level.

### Traction stroke of lower limbs section

The traction stroke of lower limbs section can be electrically adjusted by a button on the table.

Press on the left or right button respectively to adjust the stroke.

When the required stroke is reached, release the button to stop. (Figure 5)



Button on table


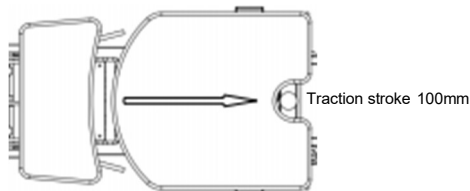


Figure 5

### Head section adjustment

Pull the control lever (in the direction indicated by the arrow) to lift the head section up and down. (Figure 6)

Lift up and down

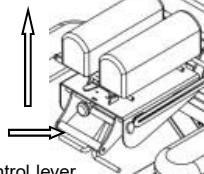


Left control lever

Figure 6

### Neck angle adjustment

As shown in Figure 7, pull the control lever (in the direction shown by the arrow) to adjust the neck angle of the head section.

Lift up and down

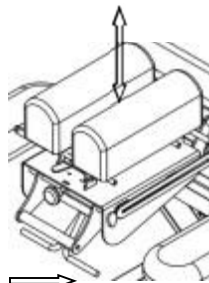

Right control lever

Figure 7

### Adjusting head section cushion width and drop operation

As shown in Figure 8, adjust the control lever to adjust the width of the head cushions.

Lift the pull rod with one hand to activate the drop and then press down on the head section to release the drop.

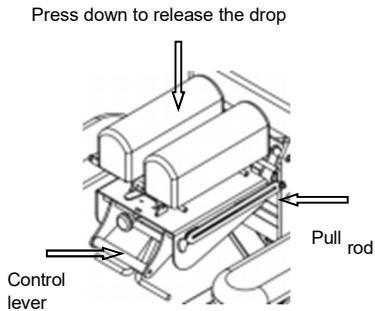


Figure 8

### Drop Tension adjustment

Use the black knob on the side as indicated below. The tension direction is indicated on the knob to either loosen or tighten. (Figure 10)

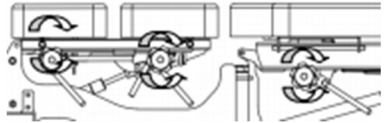


Figure 10

### Adjustment of chest section angle

The chest section of the bed can be raised and adjusted by pull rod. (Figure 11)

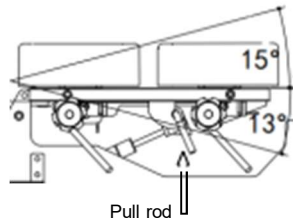


Figure 11

### Drop operation

Lift the pull rod with one hand to activate the drop (Thoracic, Lumbar & Pelvic) and then press down on the bed section to release the drop. (Figure 9)

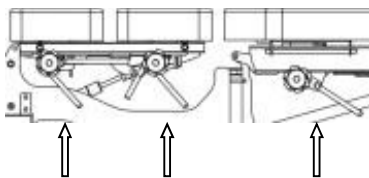


Figure 9

### Angle adjustment of lower limbs section

- As shown in the figure, holding the control lever, the lower limb plate can be adjusted by horizontal swing.

The horizontal swing angle range is  $-20^{\circ}$  to  $20^{\circ}$ . (Figure 12)

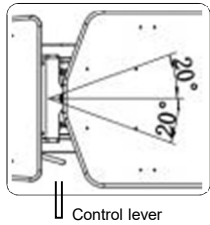


Figure 12

- The lower limb plate can also be adjusted vertically and locked at required angle by control lever.

The vertical lift angle range is  $-40^{\circ}$  to  $25^{\circ}$ . (Figure 13)

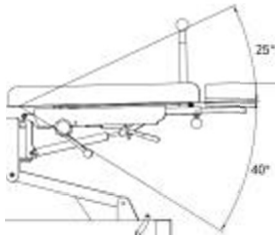


Figure 13

- Holding the center ball to achieve a  $360^{\circ}$  rotation of the lower limb section. (Figure 14)

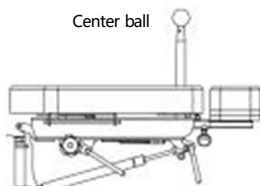


Figure 14

### Adjustment of extendable footrest

- As shown loosen the knob (in the direction shown by the arrow) to extend the footrest from 0~150mm. (Figure 15)

- When the desired position is reached, tighten the knob to affix it in place.

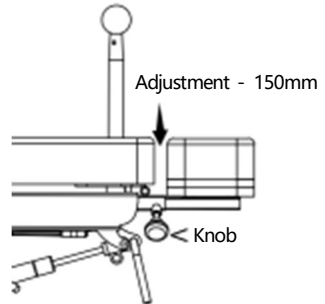


Figure 1

## TROUBLESHOOTING

If, during the life of the table you detect any malfunction or require repair, contact your dealer and do not use the table until you have received a diagnosis or the table has been professionally repaired.

## MAINTENANCE

Carry out a visual inspection at least once a month

to check the general condition of the table.












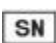

Moreover, you must do the following:

- Check the correct adjustment of the different sections.
- Check the proper functioning of the motors and their movements.
- Check and clean the gas struts
- Check and tighten screws (if necessary).
- Grease any parts with friction.
- Check the condition of the electrical wiring and the socket.
- Check the wheel immobilization system.

## RECYCLING

If you need to dispose of the table or any of its parts or accessories, consult and respect the laws and regulations applicable.

### Legend to symbols and information on the product label.

- **Model:** Product identification.
- **Model Num.:** Product version.
-  : CE conformity marking.
-  : Medical Device
-  : Unique serial number identifying the product.
-  : Date of manufacture.
- **Product Net Weight** : Table weight in kg.
- **INPUT** : Necessary values of the electrical mains supply.
- **ON: 2 min / OFF: 18 min:** Indicates the discontinuous operation of the device according to IEC/EN 60601-1, which indicates that 2 minutes of continuous operation of the motor or motors should not be exceeded.
-  : Consult the instructions for use or the user guide.
-  : Maximal load capacity that the table can support.
-  : Maximum patient weight.
-  : Waste from Electrical and Electronic Equipment. Selective waste collection of electrical and electronic equipment is required.
-  : Classification of medical electrical equipment - Double Insulation.
-  : Electrical Safety Classification - Type B.
- **IPX4** : Protection against splashing water or liquids.
-  : Manufacturer identification.
-  : Authorized representative in the EU.
-  : Importer.

## WARRANTY

This warranty shall remain in effect for 5 years

(60 months) for the structure and 3 years

(36 months) for the base frame and motor from the date of original consumer purchase.

An original sales receipt or proof of purchase may be required in order to redeem the warranty.

All repairs to the Product must be performed by a service center certified by CoinfyCare.

Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

This statement of guarantee prevails over any other statement specified or implied by the seller.

## CLEANING AND DISINFECTION

Before cleaning the table, unplug it from the mains. Clean the table after each use.

Dust the table with a soft, dry cloth.

To clean the upholstery, apply a solution of warm water and a mild detergent (for example, neutral soap) with a well wrung soft cloth.

Wipe the table over once cleaned.

Disinfect upholstery and other surfaces with a suitable bactericide for this purpose.

Apply the disinfectant on a soft cloth, never directly on the table.

Never use a mixture of disinfectants or a mixture of disinfectant and soap.

Wipe the table upholstery over once cleaned & disinfected with a cloth and water only. This

will remove any residue from the upholstery.

Avoid the accumulation of moisture on the moving parts of the metal structure, as they could rust.

Since we do not know which disinfectant will be used, we cannot guarantee that it will not cause damage to the upholstery. Read the instructions and warnings for use of the disinfectant product carefully before its purchase or use.

## TRANSPORTATION AND STORAGE CONDITION

### Transport condition

The products can be transported by general means of transportation. They should be protected from the sun and rain, and should not be mixed with toxic, harmful and corrosive materials.

### Storage

The packed electric table should be stored in a dry and well ventilated environment without corrosive gas.

This table should be operated, transported, and stored in temperatures between -10° C to + 50° C, with max humidity of 95%.

## SHUTDOWN PROCEDURE

Adjust the bed to a suitable position to ensure that the patient is safely left, then turn off the power.

### Contraindication

Please note that our beds are not sterile.

Please do not use them according to aseptic standards.

## CONTACT

Applicant / company name / after sales service unit:

Huangshan Jinfu Medical Equipment Co., Ltd  
Domicile / production address: No. 17,

Qiliting, Qimen County, Huangshan City,  
Anhui Province China

Tel: 0086 571 87006218

[www.coinfyCare.com](http://www.coinfyCare.com)



European legal representative:

LOTUS NL B.V.

Koningin Julianaplein 10, 1e Verd 2595AA,  
The Hague, Netherlands

COINFYCARE<sup>®</sup>

JF-EMN-2025V.01